**People First**

People First is our promise to empower people to communicate freely, interact naturally and participate actively


**INSTRUCTIONS FOR USE**

**MICROPHONE**

**Start / turn on / resume sound from Microphone**

**1. Make sure Streamer is on**

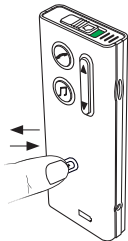Short press on connect button

Blue light slowly pulsating on Streamer

**2. Start / turn on Microphone**

Short press on on/off

Orange light slowly pulsating

**3. Sound from Microphone to Streamer**

Orange light Streamer on and orange light Microphone on

**Pause sound**

**a. Pause sound from Microphone**

Short press on on/off button


Orange light on Microphone turns off

Orange light on Streamer turns off

**b. Pause sound from Streamer**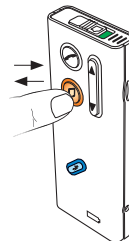

Short press on Streamer audio button

Orange light Streamer off

Orange light Microphone turns off

**Resume sound from Streamer**


**1. Microphone has to be on**

(green light on)

**2. start sound from Streamer**

Two short presses

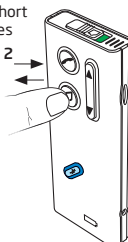two short presses on audio button

Orange light Streamer slowly pulsating

**3. Sound from Microphone to Streamer**


After 2 seconds, orange light Microphone turns on

**Check sound**

Tap the top of the microphone to check that the sound is heard in the hearing instruments

**Turn off Microphone**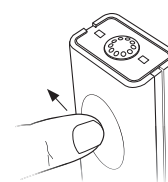

Press and hold two seconds, until green light turns off

People First

People First is our promise to empower people to communicate freely, interact naturally and participate actively


INSTRUCTIONS FOR USE

MICROPHONE


105688UK/12.10


What's in the box?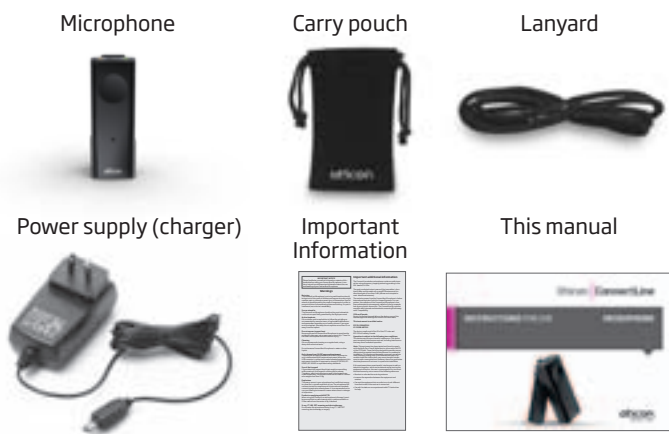


Product overview

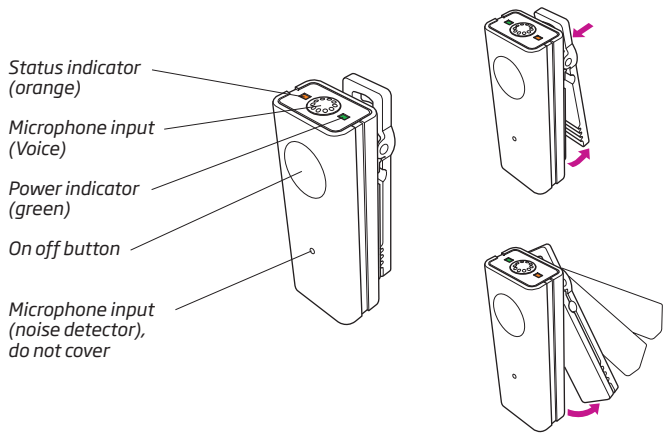How the Microphone works

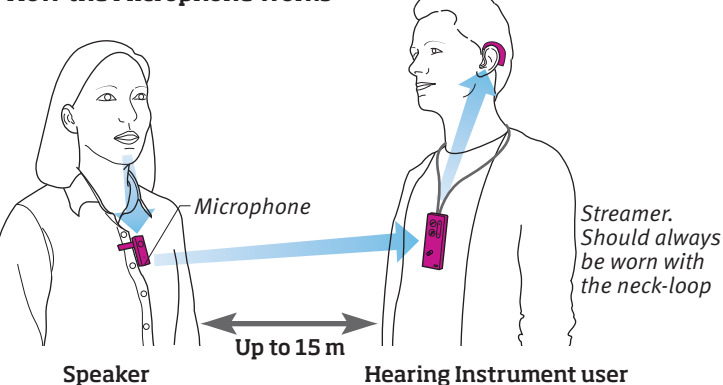
The Microphone will transmit the speech from the Speaker directly to the Hearing instrument user.  
 Note: A Microphone will only work with one Streamer at a time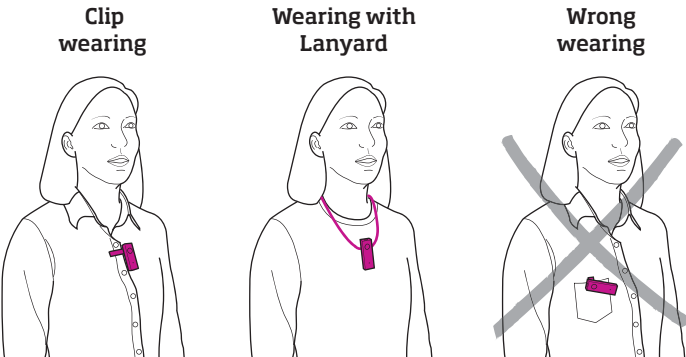


The microphone should point towards the speakers mouth with in a distance of 10 to 20 cm.

Charging / power indicator

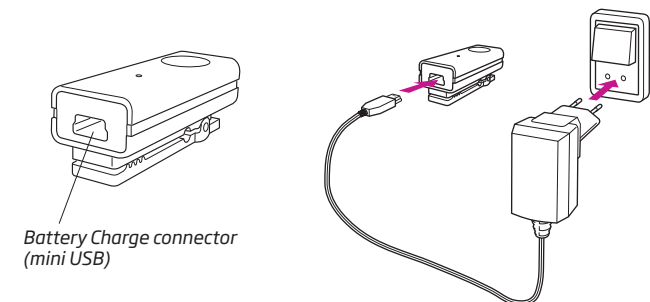
To charge the microphone, connect the power supply cord to the mini USB connector placed at the bottom of the microphone.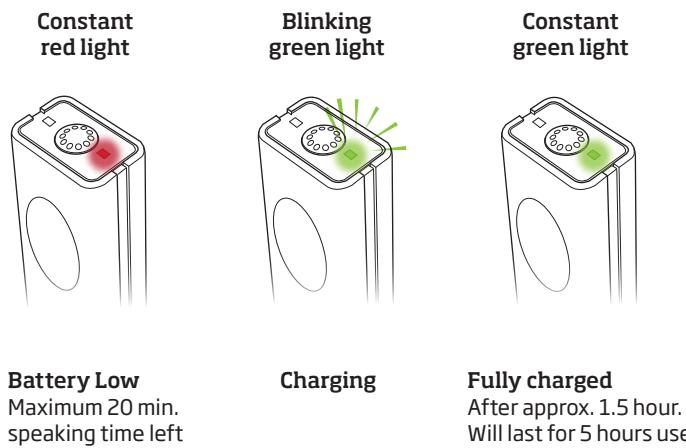

**Battery Low** Maximum 20 min. speaking time left  
**Charging**  
**Fully charged** After approx. 1.5 hour. Will last for 5 hours use

Pairing / intial setup (only needed once)

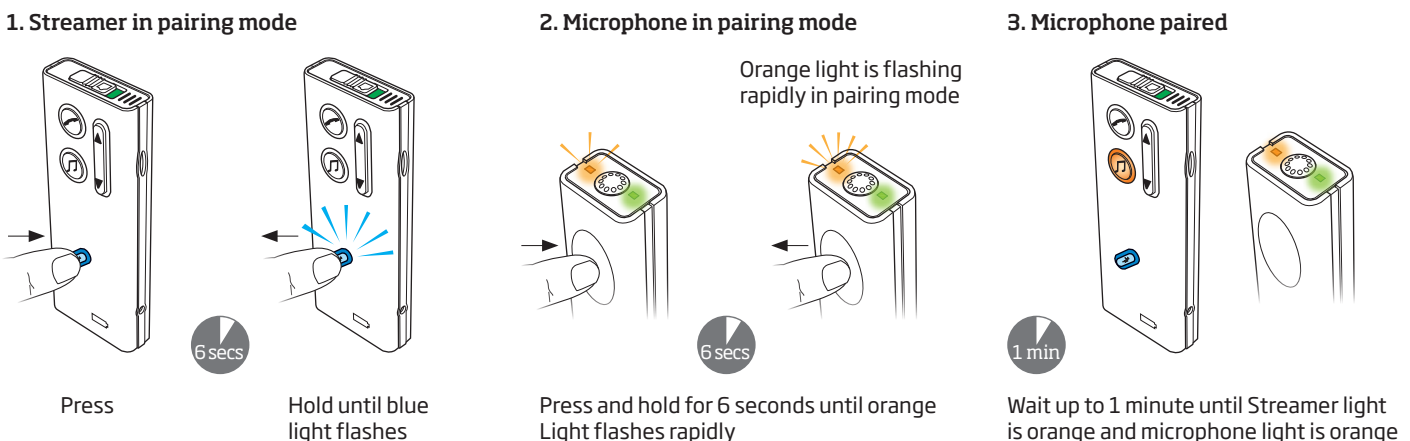Turn page

## IMPORTANT NOTICE

Please familiarize yourself with the entire contents of this booklet before using your Connectline Microphone. It contains instructions and important information about the use and handling of your Connectline Microphone.

## Warnings

### Not a toy

The Connectline Microphone is not a toy and therefore should be kept out of the reach of children and anyone else who might swallow parts or otherwise cause injury to themselves. Special attention should be paid to the smaller components in order to prevent children from swallowing them and choking. If a part is swallowed, see a doctor immediately.

### Use on aircrafts

The Connectline Microphone should not be used onboard aircrafts unless specifically permitted by the flight personnel.

### Active Implants

Oticon shows caution and advises following the guideline as recommended by manufacturers of implantable defibrillators and pacemakers regarding use of mobile phones. If you wear an active implant, then keep the microphone more than 15 cm away from the implant.

### Do not expose to extreme heat

Never expose the Connectline Microphone to extreme heat by putting it in the oven, micro wave oven or burning it. There is a risk that it will explode and cause serious injury.

### Cleaning

Oticon recommends cleaning on a regular basis, using a damp cloth without solvents.

Do not immerse Connectline Microphone in water or other liquids.

### Only charge from UL/IEC approved equipment

The safety of recharging batteries with the USB connector only is determined by the external equipment. When the USB connector is connected to mains operated equipment, this equipment should be UL approved or comply with IEC 60950, IEC-60065, IEC-60601 or equivalent safety standards.

### Use of the Lanyard

If by accident the lanyard should get caught on something while wearing it, the lanyard is constructed to release. Therefore, never try to shorten or modify the lanyard in an unauthorized way. The lanyard should not be used for children who weights less than 11 kg.

### Explosives

The power source in your microphone has insufficient energy to cause fire in normal conditions of use. The microphone has not been tested for compliance with international standards concerning explosive atmospheres. It is recommended not to use your microphone in areas where there is a danger of explosions.

### Products complying with R&TTE:

When using the ConnectLine microphone in Norway it must be ensured that no radio transmission is performed within a 20 km radius from the center of Ny-Aalesund.

### X-ray, CT, MR, PET scanning and electrotherapy

Do not wear the microphone during X-ray, CT / MR / PET scanning, electrotherapy or surgery.

## Important additional information

The ConnectLine wireless microphone contains a radio transmitter using a frequency hopping technology working in the ISM-band at 2.4 GHz.

The peak conducted output power of the transmitter is less than 6 dBm and the antenna is a small PCB antenna with a directional gain of less than 2 dBi (dB over an isotropical or omni-directional antenna).

The emission power from the Connectline Microphone is below international emission limits for Human Exposure. For comparison, the radiation of the Connectline Microphone is less than unintended electromagnetic radiation from, for example, hair dryers, electric shavers, etc. The Connectline Microphone complies with international standards concerning Electromagnetic Compatibility.

### USA and Canada

Due to the limited size available on the device, many of the relevant approval markings are found in this document.

### This instrument is certified under:

**FCC ID: U28MIC110**  
**IC: 1350B-MIC110**

The device complies with Part 15 of the FCC rules and RSS-210 of Industry Canada.

### Operation is subject to the following two conditions:

This device may not cause harmful interference. This device must accept any interference received, including interference that may cause undesired operation.

### Caution:

Any changes or modifications made to the equipment which are not expressly approved by Oticon may void the user's authority to operate the equipment.

***Note:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.*

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

# International Warranty

The manufacturer, Oticon, issues a one-year limited warranty to the original consumer of this product.

This warranty comes into effect upon the date of original purchase of this equipment, and will remain in effect for one calendar year from that date.

Please notice that extended warranties may apply in your country. Please contact your local Hearing Care Professional for more information.

## What is covered by this warranty?

Any electronic component which, because of workmanship, manufacturing or design defects, fails to function properly under normal use during the life of this warranty will be replaced or repaired at no charge for parts or labor when returned to the point of purchase. Transportation is paid by the customer. If it is determined that repair is not feasible, the entire unit may be replaced with an equivalent unit upon mutual agreement of the Hearing Care Professional and customer.

## What is not covered by this warranty?

This one-year limited warranty does not cover:

- Malfunctions resulting from abuse, neglect or accident.
- Peripheral accessories as itemized within the product brochure, when such items are later than 90 days from the original purchase.
- Batteries.
- Instruments connected, installed, used or adjusted in a manner contrary to the instructions provided.
- Consequential damages and damages resulting from delay or loss of this instrument. The exclusive remedy under this warranty is strictly limited to repair or replacement as herein provided.
- Products damaged in transit, unless investigated by the shipper and returned to the warrantor with the investigation report.

The manufacturer reserves the right to make changes in the design or construction of any of its instruments at any time without incurring any obligation to make any changes whatsoever on units previously purchased. This warranty is in lieu of all other expressed warranties by the manufacturer. No representative or person is authorized to represent or assume for the manufacturer any liability in connection with the sale or use of this product other than as set forth above.

If your ConnectLine Microphone requires service under the terms of this warranty, carefully package the instrument to prevent damage in transit and return it to the point of purchase. Include a detailed description of the problem, your full name, billing and shipping address and telephone number.

The above warranty does not affect any legal rights that you might have under applicable national legislation governing sale of consumer goods. Your point of purchase may have issued a warranty that goes beyond the clauses of this limited warranty. Please consult your point of purchase for further information.

Oticon hereby declares that this Connectline Microphone is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC. Declaration of conformity is available at:

Oticon A/S  
Kongebakken 9  
DK-2765 Smørum  
Denmark  
[www.oticon.com](http://www.oticon.com)

Waste from electronic equipment must be handled according to local regulations.

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