HUMANTECHNIK

earis

DE

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Seite 2

EN

User manual

Page 25

FR

Mode d'emploi

Page 47

NL

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Pagina 69

ES

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Página 91

IT

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Pagina 113

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Seite

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Index	Page
Important safety notices	25
Safety notes on the lithium-polymer rechargeable battery	26
Introduction	26
Contents of package	27
Product overview	28
Connecting the transmitter to an audio source.....	31
Connecting the transmitter to digital-out (S/PDIF / Toslink).....	31
Connecting the transmitter to RCA sockets	32
Connecting the transmitter to a headphone jack socket	32
Connecting the transmitter to a loudspeaker	33
Connecting the transmitter to a SCART socket.....	33
Powering the transmitter.....	34
Using the rechargeable battery	34
Earbuds.....	35
Using “earis”	36
Using the “earis” underchin receiver.....	36
Audio source selection	37
Adjusting the volume	37
Adjusting the balance	37
Using “dynamic sound HS” to improve your sound experience.....	38
Communication amplification with the receiver.....	39
Pairing transmitter and receiver.....	40
Important notes	41
Warranty.....	41
Environmental information / disposal.....	42
Troubleshooting.....	43
Specifications	45

Important safety notices

Before setting up and using the device, please read these instructions carefully and thoroughly.

- Keep this user manual in a safe place so that it can be accessed by other users at all times.
- Do not use the device if it is obviously damaged.
- Only use the device in locations where 2.4 GHz radio technology is permitted.
- The device and in particular the power supply unit are only intended for operation in dry indoor areas. Operation is only guaranteed within room temperatures of 5°C to 35 °C.
- Keep the product, packaging and accessories away from children and pets to avoid accidents.
- Always handle the device and accessories carefully and do not place them in dirty or dusty areas.
- Clean the transmitter and receiver regularly. For hygienic reasons, also clean the soft silicone earbuds regularly. Replace the earbuds for the ones available as accessories.
- Paints and furniture polishes can chemically attack the transmitter's rubber feet and cause surface damage to your furniture.
- Use only the provided power supply unit to power the transmitter.
- In the event of several days' absence, we recommend unplugging the power supply unit from the mains electricity socket.

Attention:

Your receiver can be set to a very high volume. To protect your own hearing or the hearing of people with regular hearing capabilities, especially children, from damage caused by excessive volume, please observe the following instructions:

1. Set the receiver to a low volume level before using it.
2. During use, adjust the volume to a level that is comfortable for you.
3. If you have a pacemaker, please check with your physician or the manufacturer of the pacemaker before using this product.
4. If you use other medical devices, use the device only in accordance with the safety instructions of your physician or the manufacturer.

Safety notes on the lithium-polymer rechargeable battery

The battery used in the “earis” is a lithium-polymer battery manufactured in accordance with the latest standards and safety regulations. Misuse or improper treatment can damage the battery, in which case the Humantechnik Group assumes no liability.

In extreme cases there is a risk of:

- development of heat
- development of fire
- explosion
- smoke or gas emission

- Do not expose the battery to temperatures above 70°C, e.g. by sunlight or throwing it into fire.
- Do not expose the battery to water and condensation.
- Do not short-circuit the battery.
- Do not disassemble or deform the battery.
- Store the device / battery in a cool and dry place at room temperature.
- Only use the chargers provided for the battery.
- When not in use for a prolonged time, recharge the battery in the unit regularly (every 6 months for approx. 1 hour).
- Charge the battery only at an ambient temperature of 10 °C to 40 °C.
- If the device is obviously defective, do not recharge the battery.
- Refer to the information regarding warranty and disposal.

Introduction

This digital wireless headset uses the latest digital radio technology and enables sound amplification.

The new “dynamic sound HS” technology included in this unit adjusts to your personal listening needs. Enjoy crystal clear stereo- and TV sound anywhere in the house.

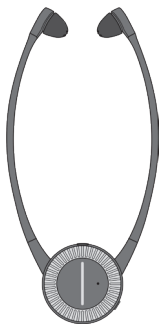
This digital wireless headset is intended to be used as assistive listening that can be used help the hearing impaired.

Please read this user manual carefully in order to set up the device correctly and to become familiar with all the possibilities it offers.

Contents of package


1 "earis" receiver

1 "earis" transmitter

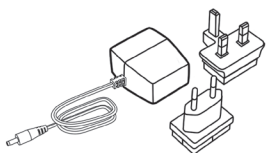
1 rechargeable battery


1 power supply with country adapters

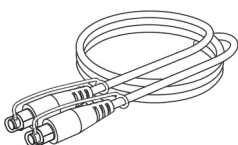
1 pair of small earbuds

1 Toslink cable

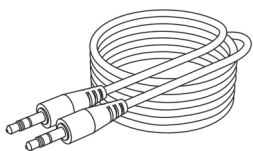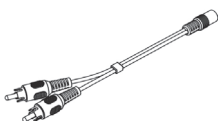1 audio connection lead with 3.5 mm
stereo jack plugs

1 audio-adaptor RCA to 3.5mm stereo jack

1 quick guide

1 user manual

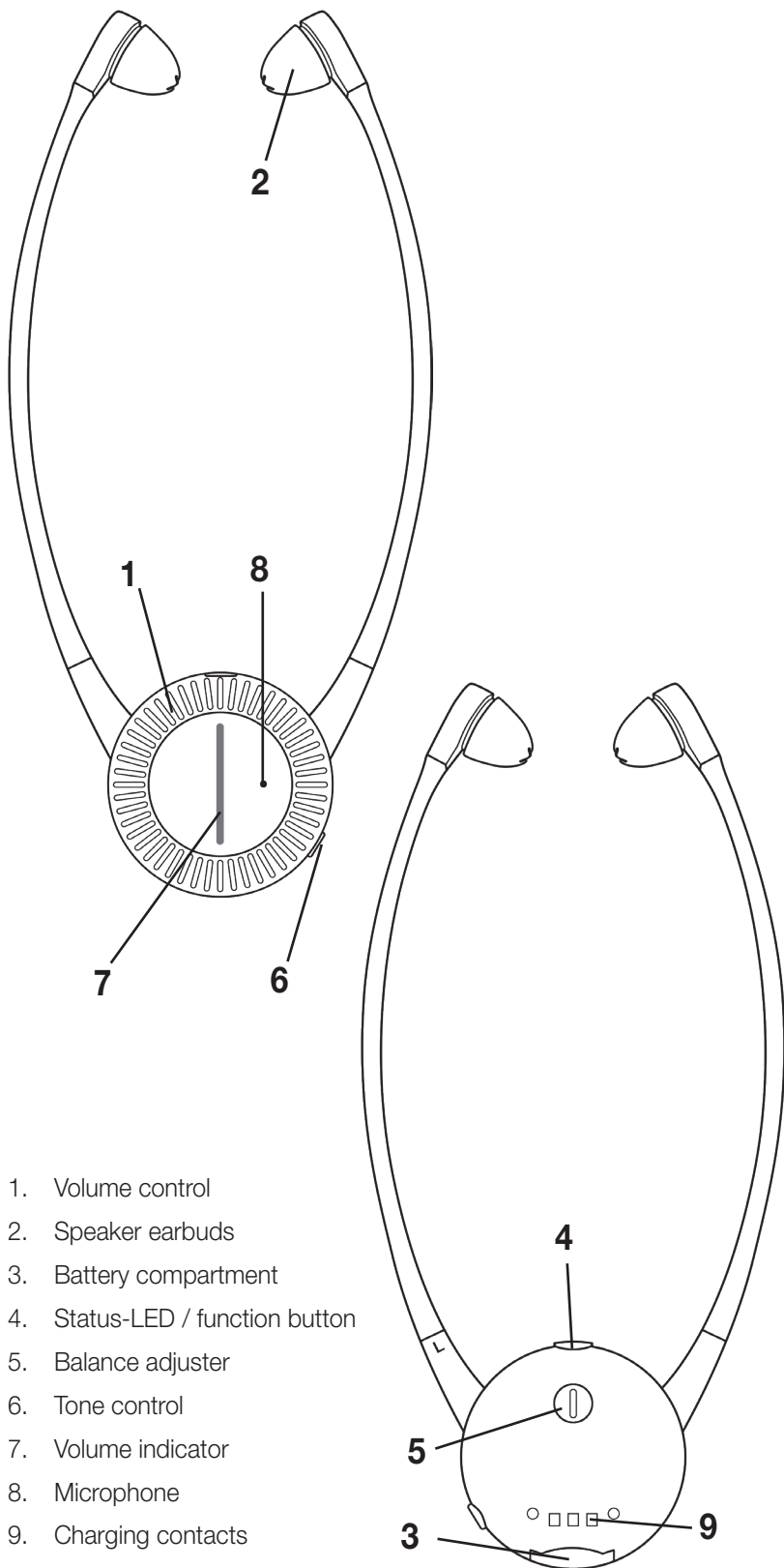
1 warranty card


Product overview

Overview: Underchin receiver

Overview: Transmitter

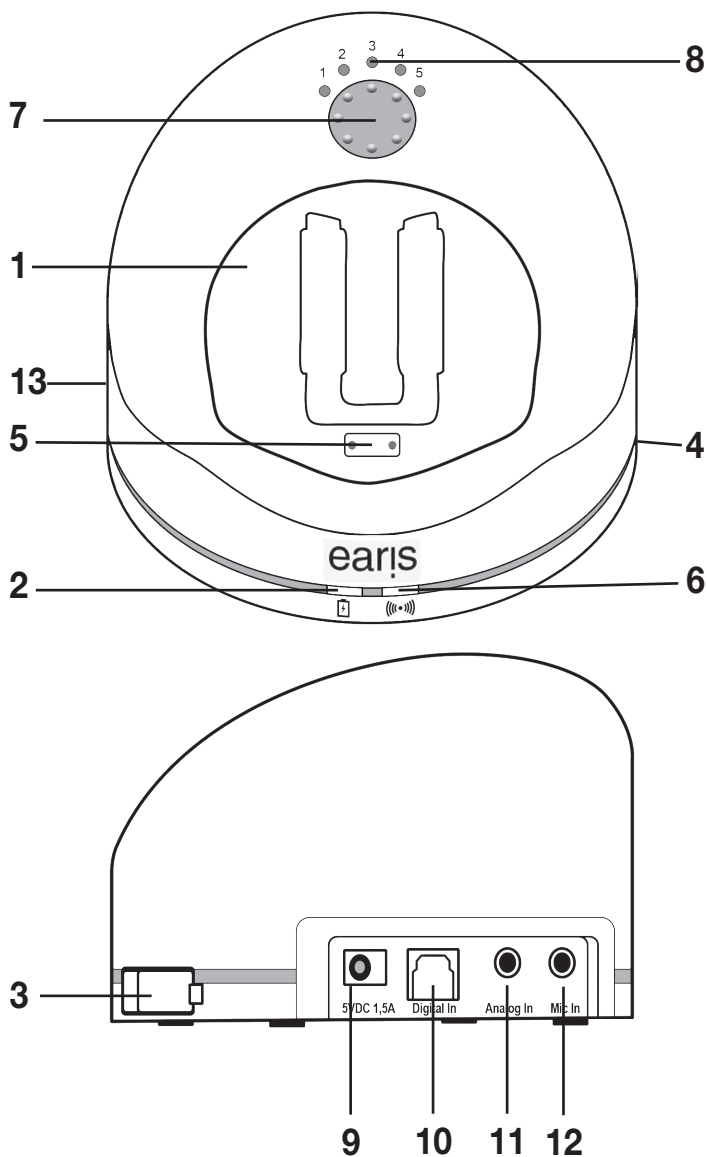1. Charging dock
2. Status LED for receiver in charging dock
3. Charging compartment for spare rechargeable battery
4. Status LED for spare rechargeable battery
5. Charging contacts with support magnet
6. Status LED
7. Tone control / pairing button
8. Tone profile setting indicator
9. Connection socket for power supply
10. Optical-digital Toslink connector
11. Jack socket for connecting analogue audio sources (blue)
12. Jack socket for connecting a microphone (green)
13. Analogue / digital selector switch

Overview: Status LED on transmitter

Status LED

dark
white light
white light flashes
flashes twice in white

Transmitter.....

... is switched off
... is switched on and transmitting
... is in pairing mode
....has no PCM signal at digital input

Overview: Status LED on receiver

Status LED

dark
constant green light
flashing green light

Receiver.....

... ist switched off
... is switched on and receiving
... is in microphone mode

volume indicator bar flashes
rapidly

... is in pairing mode

Overview of audible alerts

Audible alert

during use: 1 beep
every 30 seconds

when altering the volume:
1 beep

when altering the volume:
2 beeps

during use: 1 beep
every 20 seconds

Signification

battery is almost depleted

maximum volume reached

minimum volume reached

receiver is out of transmission range of
the transmitter

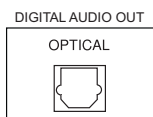
Connecting the transmitter to an audio source

Turn off the audio source before connecting the transmitter. Place the transmitter near the audio source. Do not place the transmitter in the immediate vicinity of thick reinforced concrete walls and other radio-transmitting devices, as the transmission range could be reduced.

You have several possibilities to connect “earis” to a sound source (e.g. TV, stereo set etc.).

Check the options of connection of your audio source.

Connecting options:

-S/PDIF digital output (Toslink)

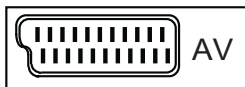
- RCA sockets

- Headphone jack

- Loudspeaker


- SCART socket

Connecting the transmitter to digital-out (S/PDIF / Toslink)

For best audio quality, connect the transmitter to your audio source using the Toslink optical (digital) cable.

Please note that the digital input can only transmit audio sources in PCM / Stereo standard.

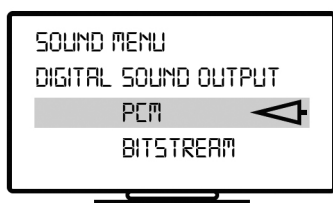
Refer to the manual of the connected audio source to learn how to set up the digital output.

Audio playback is not possible if a combined digital signal (bitstream) is used. The status LED on the transmitter will flash twice, with a pause in between, if there is no usable digital signal input.

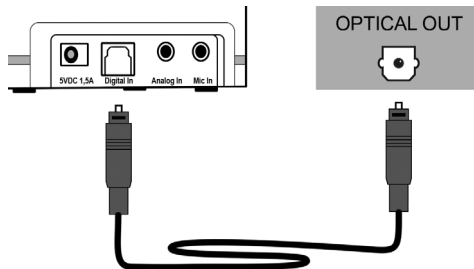Remove the protective caps on the two Toslink connectors on the cable. Plug the Toslink optical cable into the transmitter's Digital In connector and connect it to your audio source. The analog/digital switch on the transmitter's left side must be set to "Digital".


Connecting the transmitter to RCA sockets

Good audio quality can be achieved by connecting the transmitter to your audio source using the analogue audio connection lead.

Plug the supplied audio connection cable into the transmitter's blue "Analog In" socket. Connect the RCA adapter cable audio connection lead. Plug the RCA connectors into the RCA sockets of your audio source. Make sure that the white connector is connected to the white socket of the audio source and the red connector is connected to the red socket of the audio source.

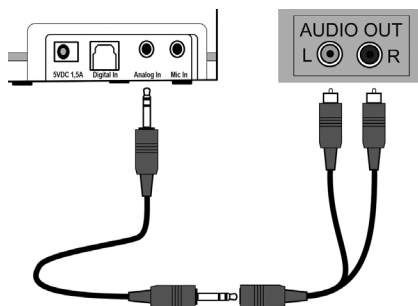The analogue/digital switch on the transmitter's left side must be set to "Analog"

Connecting the transmitter to a headphone jack socket

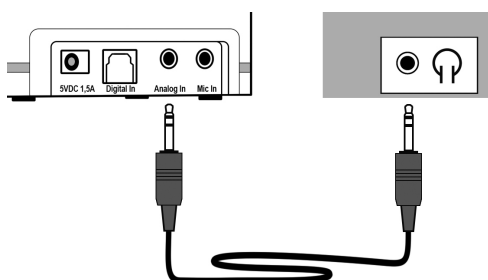
Plug the supplied audio connection lead into the transmitter's blue "Analog In" socket. Plug the audio connection lead into the headphone jack of your audio source.

Set the volume of your sound source to a medium level, where possible. There are TV sets that mute the speakers when devices are connected to their headphone jack. On some television sets, this automatic muting can be turned off via the TV settings menu. Alternatively, use another option for

connection. The analog/digital switch on the transmitter's left side must be set to "Analog".


Connecting the transmitter to a loudspeaker

The transmitter can pick up sound from a speaker via a microphone. This microphone is not included and must be purchased separately from your dealer. Only the "earis" microphone may be used.

Plug the microphone into the transmitter's green "Mic In" connector. Attach the microphone to the speaker grill of your sound source.

Please note that the volume of your sound source has a direct influence on the volume on the receiver. Please also note that this connection method provides the least audio quality of all possible options.

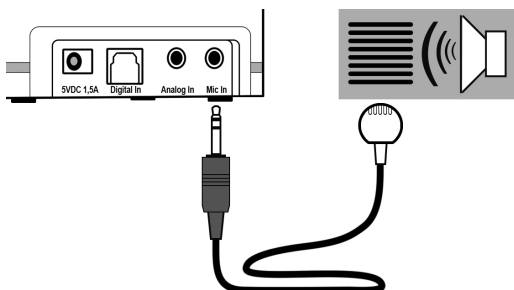The analog/digital switch on the transmitter's left side must be set to "Analog".

Connecting the transmitter to a SCART socket

The SCART adapter is not included in the scope of delivery and can be purchased from a specialist dealer.

Plug the supplied audio connection lead into the transmitter's blue "Analog In" socket. Connect the SCART adapter with the audio connection lead.

Plug the SCART adapter into the SCART socket of your audio source. The analog/digital switch on the transmitter's left side must be set to "Analog".

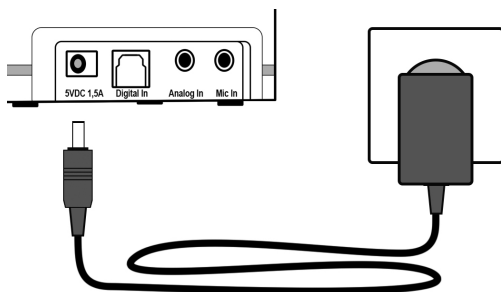
Powering the transmitter


Select the appropriate country adapter and insert it into the power supply unit until it clicks into place.

Connect the power supply unit to the transmitter with the yellow “5V DC 1.5A” power socket. Then plug the power supply unit into an electrical mains outlet.

Using the rechargeable battery

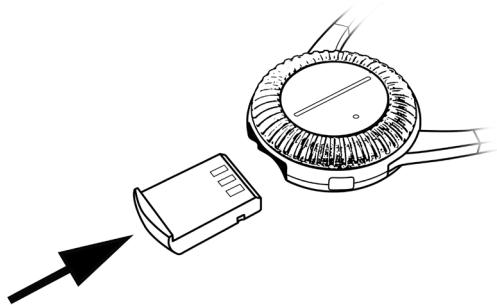Insert the supplied battery into the receiver with the slants facing upwards until it clicks into place.


Charge the receiver completely and without interruption before first use. A complete charge of 100% takes about 4 hours.

When the battery is low, the status LED on the receiver flashes and you will hear a beep every 30 seconds. Then, after a few minutes, the receiver will automatically switch off.

To charge the battery, place the receiver in the charging dock of the transmitter. The LED bar must point upwards. Magnets keep the receiver in the correct charging position.

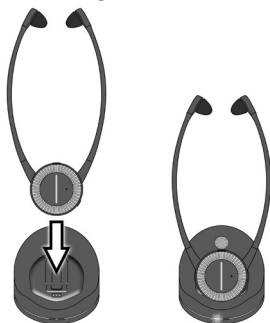The charging status LED on the front of the receiver starts flashing red. The charging status LED switches to green when the battery is fully charged.


We recommend that you always place the receiver in the charging dock of the transmitter if the device is not used. This keeps the battery in optimum operating condition at all times. The intelligent charging circuit in the receiver prevents overcharging of the battery.

A further battery can be charged in the transmitter. Additional batteries are available from your dealer.

The replacement battery can be inserted into the charging slot on the right side - please remove the cover above the charging slot first. Make sure that the slanted front edge points upwards and insert the battery without using force. During charging, the charging status LED for the spare battery lights red and switches to green when fully charged. The battery is thus ready for use.

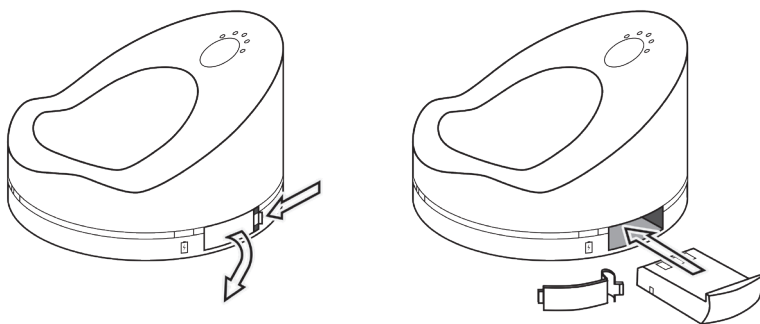If you have a second battery, we recommend that you swap the batteries once a week. These are then both used regularly and reach their maximum lifetime.


Earbuds

As supplied, the larger silicone earbuds are fitted to the receiver's speaker arms. If these buds are not comfortable for you, you will find a pair of smaller earbuds in the delivery.

To change the earbuds, gently pull them off the speaker arms.

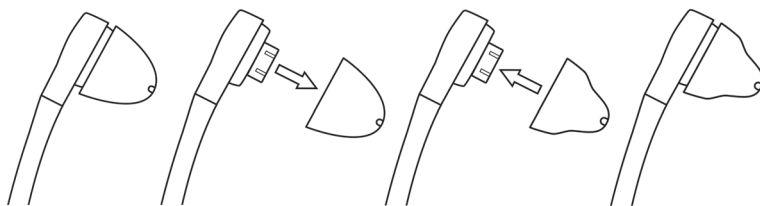Then carefully insert the other two earbuds. Make sure that the earbuds fit well and firmly.

From time to time, remove the earbuds from the receiver and clean them with mild soapy water. Dry the earbuds thoroughly before replacing them. In case of heavy soiling or damage, you should replace the earbuds. Replacements are available from your specialist dealer.

Using “earis”

Attention:

Your receiver can be set to a very high volume. To protect your own hearing or the hearing of people with regular hearing capabilities, especially children, from damage caused by excessive volume, please observe the following instructions:

1. Set the receiver to a low volume level before using it.
2. During use, adjust the volume to a level that is comfortable for you.
3. Do not expose yourself to high volume for an extended amount of time.

Using the “earis” underchin receiver

To use “earis”, proceed as follows:

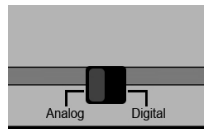
1. Switch your audio source on (e.g. TV). If “earis” is connected via the headphone jack or is operated with an optional microphone, you should set the volume of the sound source to a medium level. This is not necessary if “earis” is connected via Toslink or RCA.
2. The receiver switches on automatically when spreading the speaker arms. Use the volume control on the receiver to adjust the volume to a low level to protect your hearing. A low level of volume is indicated when only a few LEDs on the volume indicator are lit. The more LEDs on the volume indicator light up, the higher the set volume.
3. Adjust the earbuds so that they sit comfortably and without pressure directly in front of your auditory canals. Look for the markings “L” and “R” on the top of the speaker arms. The large silver volume wheel should face forward for easy operation. Put the receiver on. The green status LED at the top of the receiver indicates whether the receiver is switched on.
4. The white status LED on the transmitter lights up white as soon as the transmitter is in contact with the receiver and a transmission is established.
5. Adjust to the desired volume.
6. When you remove the receiver, it is automatically switched off. We recommend to always return the receiver to the transmitter’s charging dock so that the battery is always charged. If no audio signal reaches the transmitter for more than 15 minutes, it will switch off automatically.


Audio source selection

You have the possibility to connecting a maximum of 2 audio sources to the transmitter, for example an analogue stereo system and a digital TV set. Use the “Analog/Digital” input selection switch on the left side of the receiver to select the desired audio source.


Adjusting the volume

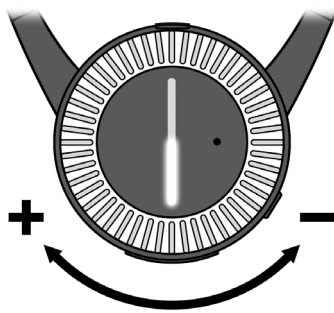Use the silver volume control on the front of the receiver to easily adjust the desired volume by turning it. The LED bar indicates the set volume. A low level of volume is indicated when only a few LEDs on the volume indicator bar are lit. The more LEDs on the volume indicator light up, the higher the set volume.

Adjusting the balance

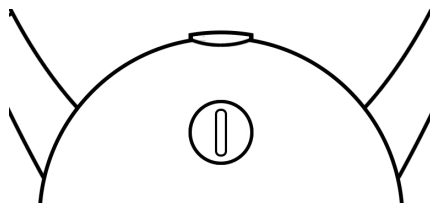The balance control allows the volume distribution between the right and left ear. Adjust the balance so that you hear equally well in both ears. The balance control is located on the rear side of the receiver and can be turned with the help of a coin.

Using “dynamic sound HS” to improve your sound experience

With “dynamic sound HS”, you have the possibility to find the ideal setting for your hearing experience. The integrated “dynamic sound HS” provides five selectable dynamic hearing profiles. These always calculate the sound dynamically according to the respective settings and are specially designed for users with hearing loss. “dynamic sound HS” is based on an algorithm developed by the German Fraunhofer Institute for Digital Media Technology IDMT in Oldenburg, which represents the latest in intelligent soundscape generation. The input signal is divided into different bands and each band is changed in its dynamic range. This change depends on the volume and sound profile selected on the transmitter. Finally, the sound is individualized on the separate frequency bands. After the separate frequency bands have been combined to form an overall signal, the system reproduces the sound. This is calculated dynamically and continuously in real time, so that the ideal adjustment is guaranteed for you at all times. The speech reproduction is calculated differently than, for example, music.

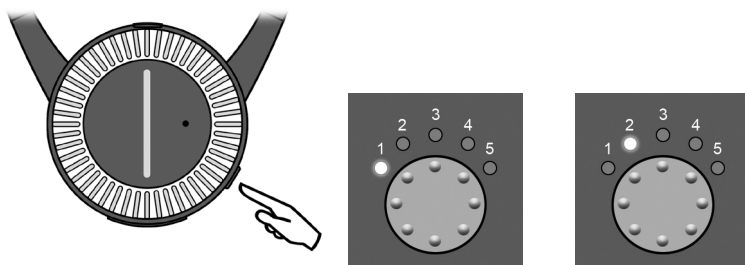
You can choose the most convenient sound profile for you from a choice of 5. The selected profile is saved.

The following profiles can be selected:

1. strong sound alteration in the low frequency range
2. minor sound alteration in the low frequency range
3. no change
4. minor sound alteration in the high frequency range
5. strong sound alteration in the high frequency range




There are two ways to select preferred sound profile:

1. Press the tone control button on the bottom left of the receiver to select the desired profile. You can identify the set profile on the transmitter by the 5 sound profile indicator LEDs.

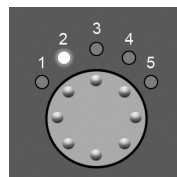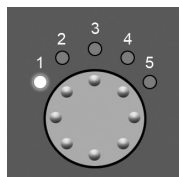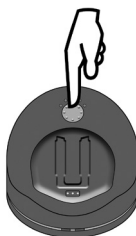1. Press the tone control button on the top of the transmitter to select the desired profile. You can identify the set profile on the transmitter by the 5 sound profile indicator LEDs.

You can connect a maximum of two receivers to one “earis” transmitter. The transmitter calculates the sound profile, while the adjusted sound profiles are stored in the receiver.

The transmitter always calculates the set sound profile of the receiver that is switched on first.

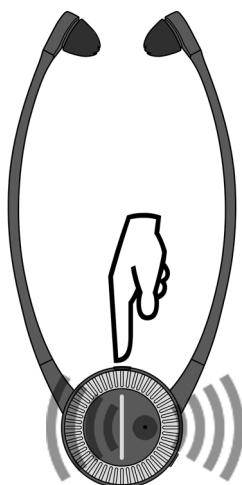
Communication amplification with the receiver

The audio stream of the transmission can be interrupted at any time, for example if you want to talk to a person in the room. The receiver is ready to use this function 2 seconds after switching on.

To do this, press the function button at the top of the receiver once. The receiver's integrated microphone now picks up the sounds of the environment and amplifies these via the receiver. The volume can be adjusted as usual. To end the function, press the function button again to resume the previous transmission.

Please note that the volume setting of the microphone is independent of the volume setting of the previous transmission. The volume you have set for the microphone is saved and used the next time you activate it.

“dynamic sound HS” is not activated when the microphone is switched on.

Pairing transmitter and receiver

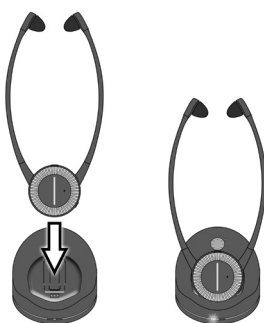
For “earis” to function properly, transmitter and receiver must be paired. As a rule, this has already been done during production and is carried out as a standard on delivery.

However, if you want to use another receiver or perhaps have to replace one later, it must first be paired with the transmitter.

A maximum of 2 receivers can be used simultaneously with one transmitter.

There are 2 possibilities to pair a receiver with the transmitter:


1. When inserting a receiver into the charging dock of the transmitter for at least 15 seconds, the according receiver and the transmitter will be paired automatically.

2. Press and hold the tone control button until the status LED on the front of the transmitter starts flashing rapidly.

Underchin receiver: Wear the underchin receiver to switch it on.

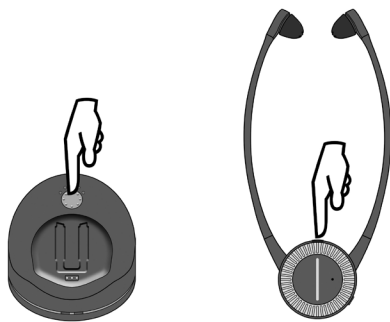Press the function button on the top of the receiver until the flashing status LED on the transmitter lights up constantly. This indicates that transmitter and receiver are now paired.


Important notes

The equipment is designed for listening within domestic environments. Other uses are outside the scope of use for this product. Damage or contamination caused to the equipment, caused by use not in accordance with these instructions, is excluded from the warranty.

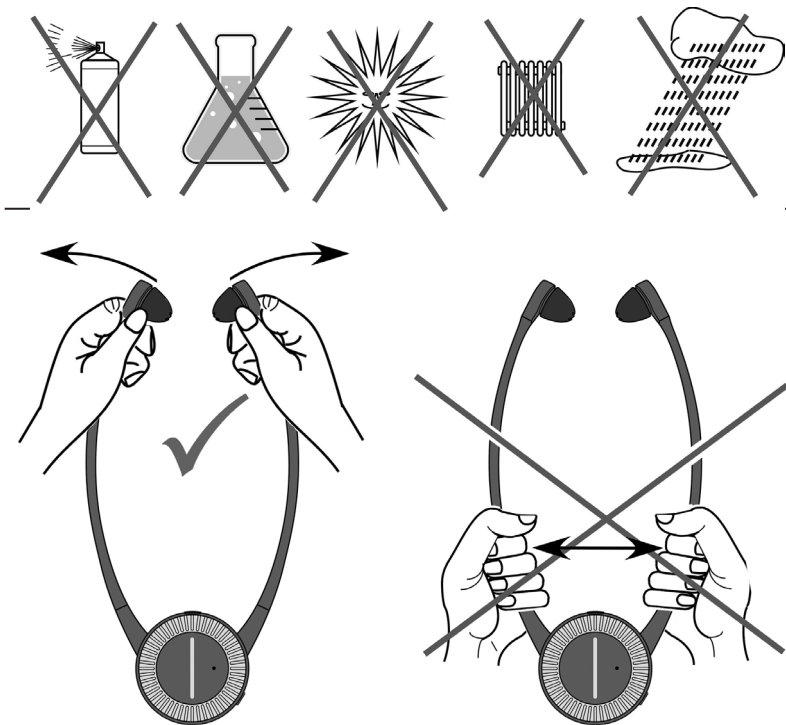The devices are not waterproof and should be kept away from all liquids. Do not place them near any heat sources. Do not place any items on top of the devices which may cause them to malfunction.

Warranty

“earis” is a highly reliable product. Should a fault occur, contrary to proper use, please refer to the troubleshooting guide. If this does not resolve the problem, please contact your retailer or the manufacturer directly for further help before returning any items.

The warranty covers free repair as well as free return shipment, where applicable.

We recommend to return the items in the original packaging, so please keep it for the length of the warranty period.

This warranty does not cover misuse, improper handling or attempted repair by unauthorised persons (broken device seal). Warranty repairs will only be carried out if proof of purchase is provided to the retailer or manufacturer.

Always specify the product number in any event.

Environmental information / disposal

Disposal of used electrical and electronic devices (to be applied in countries of the European Union and other European countries with a separate collection system for these devices).

The symbol on the product or on its packing indicates that this product must not be treated as normal household waste, but that it must be returned to a recycling point of acceptance for electrical and electronic devices or that it must be disposed in a Yellow Bin. You protect the environment and the health of your fellow man by contributing to the correct disposal. Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw materials. Additional information about recycling of this product can be obtained from your community, the municipal disposal companies or the shop where you bought the product.

Battery

The provided battery can be recycled. Please dispose of the battery through battery containers or a specialist shop.

Please dispose only discharged batteries to guarantee environmental protection.

These devices comply to the requirements of the following EC directives:

- 2011/ 65/EU RoHS-directive
- 2012/19/EU WEEE-directive
- 2014/53/EU RED-directive
- EN 301 489-17 V3.2.0 : 2017B
- EN 301 489-1 V2.2.0 : 2017
- EN 62479 : 2010
- EN 62368-1 : 2014 - EN300 328 V2.1.1:2016-11
- EN55032/AC:2016, - EN55024:2015,
- EN61000-3-2:2014 - EN61000-3-3:2013

Conformity with the above-mentioned directives is confirmed by the CE mark on the device. CE declarations of conformity are available on the Internet at **www.humantechnik.com/service**

This devices adheres to the requirements of the following IC directives:
RSS-247 Issue 2 (2017-2)

Technical changes reserved.

Troubleshooting

Symptoms	Cause	Solution
Charging status LEDs do not light up, despite batteries inserted in the transmitter charging stations.	The transmitter is without power from the power supply unit.	Check that the power supply unit is properly connected and that the mains electricity outlet is working properly.
Charging status LED does not light up, despite a battery being inserted.	No contact between battery and charging station in the transmitter.	Insert the batteries correctly and completely into the transmitter. Check if the contact areas are clean.
Charging status LED for the receiver does not light up, despite the receiver being put in the charging dock.	No contact between the receiver battery and the charging dock of the transmitter.	Make sure that the receiver is correctly placed in the charging dock.
The receiver does not work, no sound transmission.	No battery placed in the receiver.	Insert the battery in the receiver.
	The transmitter is not or not properly connected to the TV- or stereo set.	Check the cables and connectors between the transmitter and the audio source.
	The TV- or stereo set is not switched on or the output has not been configured properly.	Switch the audio source on or check the settings.
	The pairing has been lost	Pair the transmitter and the receiver again.
The receiver does not work, no sound transmission.	The receiver is not switched on. (Pocket Version)	Switch the receiver on, using the ON/OFF switch on the side of the device. (Pocket Version)

Symptoms	Cause	Solution
	The receiver battery is depleted or not charged.	Charge the battery or insert a charged battery.
Sound transmission is poor, noisy or disturbed.	The receiver is too far from the transmitter.	Bring the receiver closer to the transmitter.
Sound transmission seems distorted, muted or disturbed.	The battery is almost depleted.	Charge the battery or insert a charged battery.
The sound transmission is briefly interrupted and then works again	The receiver is too far from the transmitter. The surroundings hinder transmission (walls, objects).	Bring the receiver closer to the transmitter.
The battery lasts only a short while.	Battery is not fully charged or defective and therefore no longer able to work.	Charge the battery, insert a fully charged battery or a new fully charged battery.

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For other service-partners
in Europe please contact:

Humantechnik Germany
Tel.: +49 (0) 76 21/ 9 56 89-0
Fax: +49 (0) 76 21/ 9 56 89-70
Internet: www.humantechnik.com
e-mail: info@humantechnik.com

HUMANTECHNIK

RM41000A · 0219

USA

Federal Communication Commission Interference Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC Caution: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This device and its antenna(s) must not be co-located or operating in conjunction with any other antenna or transmitter.

FCC Declaration of Conformity (DoC)

FCC ID: U94EARIS1218-R, U94EARIS1218-T

FCC Declaration of Conformity (DoC)

FCC ID	U94EARIS1218-R	U94EARIS1218-T
Model Name	Earis receiver, Earis PR receiver	Earis transmitter

Federal Communication Commission (FCC) Radiation Exposure Statement

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. End user must follow the specific operating instructions for satisfying RF exposure compliance

HUMANTECHNIK

Model No: Earis transmitter, Earis receiver,
Earis PR receiver

We, Humantechnik GmbH, Im Wörth 25, DE-Weil am Rhein, Tel.: +49 (0) 76 21/ 9 56 89-0, declare the above comply with the requirements of Federal Communications Commission. This device complies with Part 15 of FCC rules. Operation is subjected to the following conditions:
1) This device may not cause harmful interferences, and
2) This device must accept any interferences received that may cause undesired operation.

Responsible Party: Daniel Müller

Canada Industry Canada Statement

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

1. This device may not cause interference.
2. This device must accept any interference, including interference that may cause undesired operation of the device.

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CAN ICES-3 (B)/NMB-3(B)

Radiation Exposure Statement:

The product comply with the Canada portable RF exposure limit set forth for an uncontrolled environment and are safe for intended operation as described in this manual. The further RF exposure reduction can be achieved if the product can be kept as far as possible from the user body or set the device to lower output power if such function is available.

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