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#### For the safety of your children

Please be aware that this product may include small parts. Ensure that the package and small parts are kept out of reach from your child.

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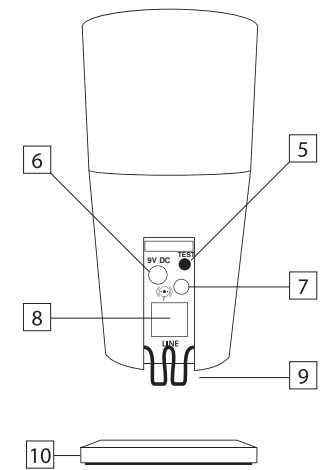
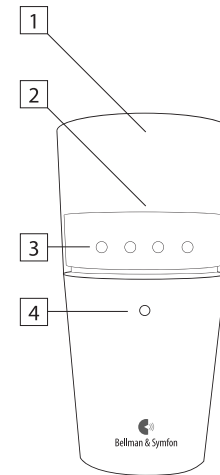
## Flash receiver

### Model BE1330

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**CE0681!**

For use in all EU countries

Hereby, Bellman & Symfon, declares that this BE1330 Flash receiver is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EG.

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## User manual, Bellman Visit Flash receiver

---

*Bellman Visit Flash Receiver is part of the Visit system and alerts with various light signals when someone is calling on the phone or rings the doorbell, or when the Baby-cry transmitter or Smoke alarm is activated. You can also connect the Bellman Visit Bed-shaker (accessory) and get alerted through vibrations. Read the manual carefully before using your Bellman Visit Flash Receiver for the first time. Also keep in mind not to place the unit near any hot surfaces (e.g. a hotplate or oven).*

### This is what to do!

- A. Connect the mains adapter to outlet (6) on the Flash and let it remain connected for at least 24 hours first time of usage.
- B. Place the unit where you want it. By turning the lens you can chose the direction of the flash. There is also a wallmount bracket available as an accessory (BE9075).
- C. Check the flash effect and radio range by pressing the test button on the transmitters in your system.

## Miscellaneous

### Flash (2)

The purpose of the flash in the Bellman Visit Flash Receiver is to visually call on your attention. There is a built-in light sensor that regulates the number of flashes emitted; 12 flashes during the day and 8 at night. When the phone rings, the unit will only flash more 5 times after the last ring signal, day or night.

### Signal Indicators (3)

Behind the lens are 4 coloured LEDs (3) that, upon delivery, indicates the following signals:

 (orange) Baby-cry    (green) Door    (yellow) Telephone    (red) Fire

### Range

The range between Bellman Visit Flash and the transmitters is more than 30 meters, but is affected by the thickness and material of the walls. Thick walls made of concrete will decrease the range.

### Radio channel

Upon delivery, all units are set on the same radio channel. If you need to change, or have changed already, the radio channel in your transmitters (see each transmitters own manual) you will need to change the channel of your Flash as well. For safety reasons, Bellman Visit Smoke alarm transmits on all radio channels.

- How you set the radio channel:

Place the unit near one of your transmitters. Press the test button (5) on the Flash Receiver and keep it pressed for approximately 10 seconds until the yellow and green LED (3) starts flashing. Release the test button on the Flash Receiver and press the test button on the transmitter instead. All LEDs (3) on the Flash Receiver will then start to flash in sequence. This confirms that the radio channel in the Flash Receiver has been changed.

Please note!

The flash may be uncomfortable to you at such a short distance. You have the option of turning the upper part of the receiver around to direct the flash.

### Power supply

Bellman Visit Flash Receiver is powered by a mains adapter. Connect it to the inlet (6) on the back of the unit. During normal usage, the mains indicator (4) will shine with a constant green light. When there is an AC power failure or when the Flash Receiver is not connected to the mains adapter, the mains indicator will flash with a green light. If it flashes with a red light, the battery needs to be charged - see section 'Battery'.

Please note! Only mains adapter Bellman model G090050D31 is to be used.

## Battery

Bellman Visit Flash Receiver has a built-in, rechargeable (NiMH) back-up battery which is automatically activated when the Flash is no longer powered through the mains adapter (the mains indicator (4) flashes green). This battery lasts up to a week during normal usage, which also allows for the Flash Receiver to be, for example, brought from one room to another or even outdoors. It is, however, recommended that the Flash Receiver remains connected to the mains adapter during normal usage. Let the Flash Receiver be connected for a minimum of 24 hours first time of usage to allow for the back-up battery to be fully charged. This will normally take approximately 16 hours to fully charge an empty battery. When the battery needs to be recharged, the battery indicator (4) will start to flash red. If the battery needs to be replaced, please contact your distributor.

### Battery/Mains indicator (4)

Below the lens there is an LED which indicates both the power and battery status. When the LED shines with a constant green light the unit is connected to the mains adapter and the battery is being charged.

When the LED flashes green, the unit is powered by the back-up battery.

When the LED flashes red the back-up battery is running low and you should recharge it by connecting the unit to the mains adapter.

### Test button (5)

The test button (5) is located on the back of the Bellman Visit Flash Receiver. By pressing the test button, you can check that the unit is working. Keep in mind to not look straight at the unit since the emitted flash may be uncomfortable to your eyes at such a short distance. If the Bellman Visit Bed-shaker (accessory) is connected, it too will signal but with vibrations.

The test button is also used for setting the radio channel - see section "Radio channel" or to cancel the flashes as well as the vibrations from the Bed-shaker (accessory) by pressing it shortly when the Flash signals.. For safety reasons the signaling can not be cancelled when there is a fire alarm.

### Telephone outlet (8)

There is a modular connector on the back of the Flash Receiver which offers a direct connection to the telephone line. This allows the Flash to signal when the phone rings without using the Bellman Visit Telephone transmitter. Order the telephone connection cord from your distributor.

## Troubleshooting

Follow the troubleshooting instructions below before calling for service.

Symptom	Procedure
Bellman Visit Flash does not receive signals from transmitters	<p>Make sure that the Flash receiver has power (the Battery/Mains indicator (4) should flash or shine with a constant green light.)</p> <p>Make sure the receiver and transmitter operate on the same radio channel (see section "Radio channel")</p> <p>Make sure the flash works by pressing the test button on the Flash Receiver.</p>
Bellman Visit Flash signals without being activated by any of my transmitters.	Change the radio channel in all the transmitters and in the Flash Receiver (see section "Radio channel").

### Product description:

Product description:

1) Lens

2) Flash

3) Signal indicators

4) Power indicator

5) Test button

6) Inlet for mains adaptor

7) Outlet for Bellman Visit Bed-shaker (accessory)

8) Modular connector for the telephone connection line (accessory)

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