

Set-up Instructions for a cordless monitor & the cordless pad transmitter(TM-11):

1. Battery cover needs to be opened with a screwdriver. Put 3 x AA batteries into the monitor and replace cover.
2. Make sure transmitters are switched into ON position and connected with Cordless Pressure Pad. *Transmitters are shipped in "storage mode". Activate by pressing and holding pad for 15s.*
3. Press "Program" button on rear of monitor. The monitor will beep and Pad1 LED will light up.
4. Press and hold pad with activated transmitter for 1-2 seconds and release. Monitor will beep once to indicate programming is successful, and Pad2 LED will light up. If a second pad will be used press it for 1-2 seconds and release. If only one pad will be used press "Reset" to exit programming.
5. Test the Cordless system by pressing down on the pad/s. The LED should flash several times, and the monitor should beep twice to indicate the system is ready. Release pressure from the pad and the monitor should alarm. Press reset to silence.
6. Make sure transmitters are safely secured inside the pocket provided in the pad. The Cordless system is now ready to be put into service.

Warning:

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Caution:

The user is cautioned that changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.