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
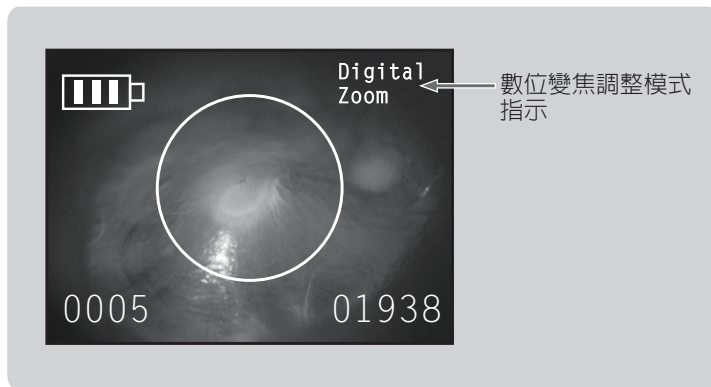
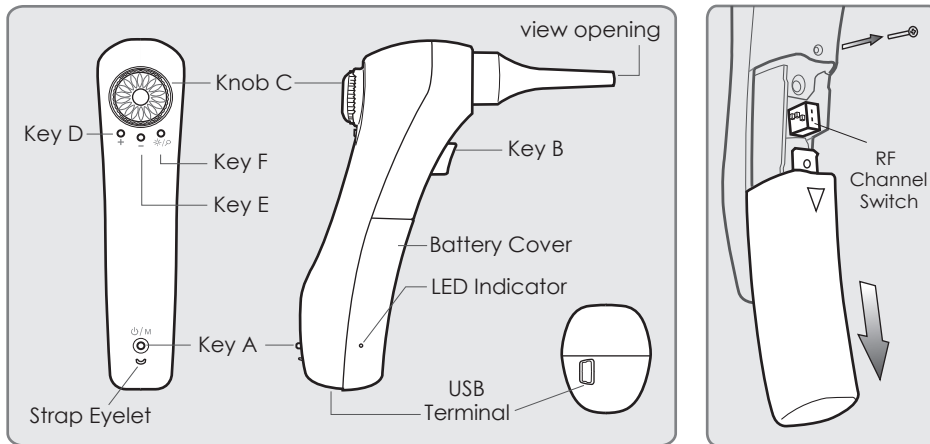
OT-2000

- | | |
|----------------------------------|---|
| Key A (ON/OFF) : ON/OFF switch | Key D (+) : Zoom in /Brightness+
Mode switch: Playback Mode |
| Key B (Shutter) : Shutter Button | Key E (-) : Zoom Out/Brightness-
Mode switch: Digital Zoom/Brightness mode |
| Knob C : Focus Adjustments | |

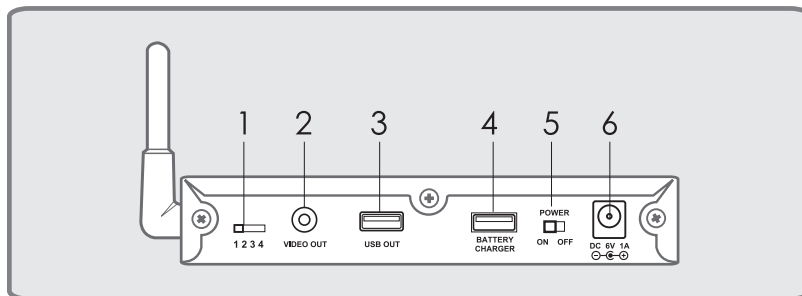RF Receiver


- | | |
|-------------------------------------|----------------------|
| 1. RF Channel Switch(1~4) | 4. USB Charging Port |
| 2. Video Line Out (for TV/monitors) | 5. ON/OFF Switch |
| 3. USB Connector (for PCs) | 6. DC IN |

Charging

The battery is not fully charged when the Otscope is purchased. Fully charge the battery before first use (8 hrs).



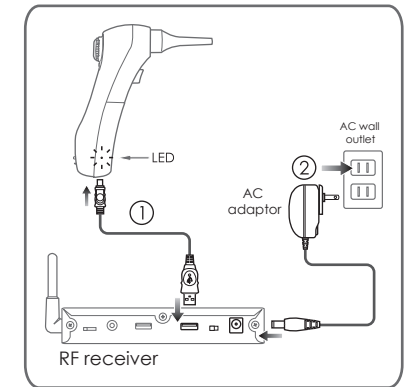
1. Connect the Otscope and the RF Receiver (USB charging port) with the USB cable, as shown in ①. Connect the AC Adaptor as shown in ②. Switch the RF Receiver ON.
2. The LED indicator flashes a green light while charging. Charging is finished when the green LED stays on.

Normal charging time **Approx. 5 hrs**

Switching On

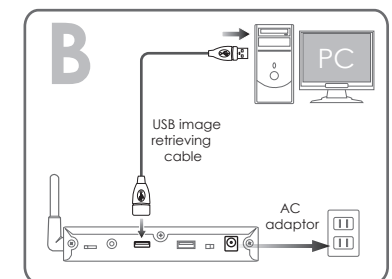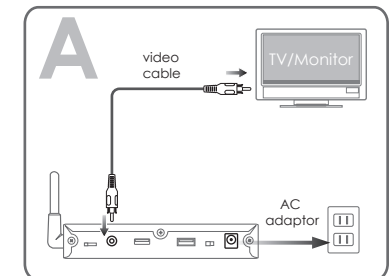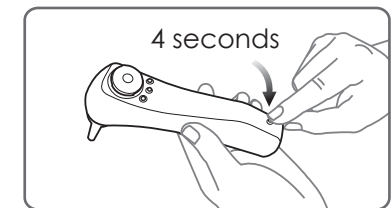1. Hold down Key A for 4 seconds.
2. A) TV/monitor connection:
Use the video cable to connect the RF Receiver(video line out) to the TV/monitor(video line in).
- B) PC connection:
(Please install the driver and software from the CD-ROM beforehand.)
Use the USB image retrieving cable to connect the RF Receiver(USB connector) to the PC(USB port).
3. Connect the AC Adaptor and switch the RF Receiver ON. Make sure the RF channel switch in selected on Channel 1.

Shutting Off

To shut down power of the Otscope, hold down Key A for 4 seconds just as in switching on.