



Instructions for Attaching Leg Tags:

**FCC Radio frequency interference statement

**This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

E.N.G.S Systems LTD is not responsible for any radio or communication interference caused by using other than specified or recommended cables and battery or by unauthorized changes or modifications to this equipment. Changes or modifications not expressly approved by the manufacturer could void the user's authority to operate the equipment.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.



It is recommended to make a list of the tag numbers you plan to attach to the cows, and keep the hard copy as a reference.

- Make sure to keep enough distance between you and the animal, in case the animal is trying to kick or move rapidly.
- Don't put your hands in the headlock or in areas that the animal can hurt you.
- Know ahead of time what leg to install the tag that will work with the designated applications such as: the parlor ID or feed ID.
- It is preferable to work in teams of two, one experienced person to keep the animal quiet and one to attach the tag.

1. Have the animal locked in a headlock to limit her movement and to avoid injuries.
2. Attach the strap of the tag only on one side of the leg tag by making a loop at the end of the strap, leaving a few centimeters of extra strap, as shown in the attached picture, and push it through one of the openings on the side of the tag. Insert one of the black stopper pins into the loop, and then pull both ends of the strap down, so that the pin sits firmly in the opening. Then pull again, sharply, so that the pin clicks into place in the opening. It should be sitting just below the surface level of the tag.





3. Hold the tag, with the Logo facing up, next to the outside of the cow's leg. Bring the strap around the leg and thread the pointed end of the strap through the opening. Then thread it back through, to make a loop, as you did with the first end of the strap.

4. Put the black stopper pin in the loop and pull gently on the strap to move the pin close to the opening. You can adjust the strap around the leg by pulling on the end of the strap. The pin will roll with the strap, allowing you to fit the tag securely on the leg. Leave two fingers width of space between the strap and the leg.

Put the black stopper pin in the loop and pull gently on the strap to move the pin close to the opening. You can adjust the strap around the leg by pulling on the end of the strap. The pin will roll with the strap, allowing you to fit the tag securely on the leg. **Leave two fingers width of space between the strap and the leg.**

5. Once the fitting is done, pull again firmly on the end of the strap, to click the pin into place.



Uninstall tag

1. Use a sharp cutting tool, (preferably with a rounded tip). Slide it inside the strap and cut away from the leg.

