



KONICA MINOLTA

DIRECT DIGITIZER

AeroDR SYSTEM

AeroDR 1717HQ



Operation Manual

Introduction

Chapter 1 Safety Precautions
& Warnings

Chapter 2 Product Overview

Chapter 3 General Operations

Chapter 4 Status (LED) Display

Chapter 5 Troubleshooting

Chapter 6 Error Codes

Chapter 7 Maintenance

Chapter 8 Specifications

CE 0197 CE ⚠

EN



Manufacturer:
KONICA MINOLTA MEDICAL & GRAPHIC, INC.

1 Sakura-machi, Hino-shi Tokyo 191-8511, Japan

CODE NO. A54T

Contents

Introduction 5

Introduction	6
Summary of usability specifications (for IEC/EN 60601-1-6, IEC/EN 62366)	7
Disclaimer	7
Trademark	8
AeroDR SYSTEM name correspondence table	8
Term description	8
Structure of pages	9

Chapter 1

Safety Precautions & Warnings . . 11

1.1 Symbols relating to safety	12
1.1.1 Safety alert symbol	12
1.1.2 Warning notice (signal words)	12
1.1.3 Description of graphic symbols	12
1.2 Warning labels	13
1.2.1 AeroDR Detector	13
1.2.2 AeroDR Battery Charger	13
1.3 Safety precautions	14
1.3.1 Precautions before usage	14
1.3.2 Precautions for usage	14
1.3.3 Precautions regarding electromagnetic waves	15
1.3.4 Precautions regarding wireless communication	19
1.3.5 Precautions for installing, moving, and storing	21
1.3.6 Precautions regarding maintenance	21
1.3.7 Precautions on service life	22
1.4 R&TTE DECLARATION of CONFORMITY	23

Chapter 2

Product Overview 25

2.1 Overview of this device	26
2.1.1 Functions	26
2.1.2 System configuration and connection examples	26
2.2 Component names and functions	28
2.2.1 AeroDR Detector	28
2.2.2 AeroDR Interface Unit	29
2.2.3 AeroDR Generator Interface Unit	30
2.2.4 AeroDR Battery Charger	31
2.2.5 AeroDR Access Point	32
2.2.6 AeroDR UF Cable	33
2.2.7 AeroDR I/F Cable	33

Chapter 3

General Operations 35

3.1 Startup and shutdown	36
3.1.1 Startup of this device	36
3.1.2 Shutdown of this device	37
3.2 Operation of AeroDR Detector	38
3.2.1 Exposure	38
3.2.2 AeroDR Detector orientation	39
3.2.3 Precautions for exposure	39
3.3 Charging of AeroDR Detector	41
3.3.1 Charging with AeroDR Battery Charger	41
3.3.2 Charging with the wired cable	42
3.3.3 Charging time guide	42
3.3.4 AeroDR Detector charging display	42
3.4 Registration and selection of the AeroDR Detector	43
3.4.1 Registration of the AeroDR Detector	43
3.4.2 Selection of the AeroDR Detector	43
3.4.3 Manual selection of the AeroDR Detector	43
3.5 Calibration	44
3.5.1 Offset calibration	44
3.5.2 Gain calibration	44
3.6 Position to affix AeroDR Detector identification label	45

Chapter 4

Status (LED) Display 47

4.1 LED display of respective devices	48
4.1.1 AeroDR Detector	48
4.1.2 AeroDR Interface Unit	49
4.1.3 AeroDR Generator Interface Unit	49
4.1.4 AeroDR Battery Charger	50
4.1.5 AeroDR Access Point	51

Chapter 5

Troubleshooting 53

5.1 Support flow during trouble	54
5.2 Various problems and countermeasures	55
5.2.1 AeroDR Detector	55
5.2.2 AeroDR Interface Unit	56
5.2.3 AeroDR Battery Charger	56
5.2.4 AeroDR Access Point	57
5.2.5 CS-7/Images	57

Chapter 6
Error Codes 59

- 6.1 Types of error codes and recognition methods 60
 - 6.1.1 Types of error codes 60
 - 6.1.2 Recognition method for error codes. . 60
- 6.2 Error codes and countermeasures 61
 - 6.2.1 AeroDR Detector errors 61
 - 6.2.2 AeroDR Generator Interface Unit errors 66

Chapter 7
Maintenance. 67

- 7.1 Maintenance and inspection items 68
 - 7.1.1 Maintenance schedule. 68
 - 7.1.2 Cleaning. 68
 - 7.1.3 Disinfection of the AeroDR Detector 69

Chapter 8
Specifications 71

- 8.1 Specifications 72
 - 8.1.1 AeroDR Detector 72
 - 8.1.2 AeroDR Interface Unit 73
 - 8.1.3 AeroDR Generator Interface Unit. 73
 - 8.1.4 AeroDR Battery Charger 74
 - 8.1.5 AeroDR Access Point. 74
 - 8.1.6 AeroDR UF Cable 75
 - 8.1.7 AeroDR I/F Cable. 75
 - 8.1.8 AeroDR XG Cable 75
 - 8.1.9 AeroDR S-SRM Cable 76
 - 8.1.10 General AeroDR SYSTEM. 76

Introduction

Introduction

Digital radiography AeroDR SYSTEM (hereafter referred to as this device) performs X-ray imaging of the human body using an X-ray planar detector that outputs a digital signal, which is then input into an image processing device, and the image acquired with a digital image acquisition device is then transmitted to a filing system, printer, and image display device as diagnostic image data. Diagnostic X-ray image data of this device does not provide mammographic images.

The X-ray planar detector contained in this device is the AeroDR 1717HQ (AeroDR P-21) detector, and it can be connected by wireless or wired connection.

The DIRECT DIGITIZER CS-7 (hereafter referred to as CS-7), which controls the receiving, processing, and output of image data of this device, is required for operation. For the operation of the CS-7, refer to the "CS-7 Operation Manual".

This operation manual provides instructions on the basic functions for operation of this device. Those operating this device for the first time should read this manual beforehand.

Also, store this manual close to this device after reading it through, so it can be used as a guide to allow optimum operating conditions.

*** If the pages of the operation manual are smudged and illegible, replace it with a new one. (There is a fee for this service.)**

Summary of usability specifications (for IEC/EN 60601-1-6, IEC/EN 62366)

- 1) Medical purposes
 - Provision and reading of disease and injury diagnostic images.
- 2) Patient groups
 - No patient population exists who uses or is in contact with the device.
 - Patient population for the X-ray images read is not specified.
- 3) Parts of body or organizations to which the device is mounted or that interact with the device.
 - AeroDR Detector contacts the body surface of a patient.
 - AeroDR Interface Unit contacts the body surface of an operator.
- 4) Operating principle
 - AeroDR Detector forms the still images according to the X-ray energy passing through the human and animal body; after digitizing the exposed image, it is transmitted to the console (CS-7) with wired or wireless communication by way of AeroDR Interface Unit.
 - AeroDR Interface Unit supplies the power to AeroDR Detector, AeroDR Generator Interface Unit, and AeroDR Access Point (radio communication device).
 - AeroDR Generator Interface Unit interfaces with an X-ray device.
 - AeroDR Access Point (radio communication device) performs a wireless communication with AeroDR Detector.
 - The console (CS-7) processes the image data into the diagnostic image, and then stores and outputs the images added with the patient information.
 - The AeroDR Battery Charger charges the AeroDR Detector. It registers the using cassette with the exposure room.
- 5) Significant physical characteristics
 - Refer to “8.1 Specifications”.
- 6) Significant performance characteristics
 - Refer to “2.1 Overview of this device”.
- 7) User of this device
 - An adult technician of sound body who understands the operation manual (physician, radiologist)

Disclaimer

- (1) This manual may not be reproduced in whole or in part without the permission of Konica Minolta Medical & Graphic, Inc.
- (2) The contents of this manual may be subject to change without prior notice.
- (3) Konica Minolta Medical & Graphic, Inc. is not responsible for any claims made for malfunction or damage caused by installation, relocation, modification, maintenance, and repair made by anyone except Konica Minolta and contractors designated by Konica Minolta.
- (4) Konica Minolta Medical & Graphic, Inc. is not responsible for any claims made for malfunction or damage to Konica Minolta products, caused by third-party products not installed by Konica Minolta.
- (5) Konica Minolta Medical & Graphic, Inc. is not responsible for any claims made for malfunction or damage caused by maintenance or repair using maintenance parts other than those specified by Konica Minolta.
- (6) Konica Minolta Medical & Graphic, Inc. is not responsible for any claims made for malfunction or damage caused by not observing the precautions and operation methods described in the operation manual.
- (7) Konica Minolta Medical & Graphic, Inc. is not responsible for any claims made for malfunction or damage caused by the environment, such as power source and installation environment, that is described in the Installation Requirement or operation manual, that is not suitable for the installation requirements of this device.
- (8) Konica Minolta Medical & Graphic, Inc. is not responsible for any claims for malfunction or damage caused by acts of nature such as fires, earthquakes, floods, or lightning strikes.
- (9) Konica Minolta Medical & Graphic, Inc. is not responsible for any claims for malfunction or damage caused by using this device for any purpose other than that specified for this device.
- (10) Diagnostic and treatment action is performed under the responsibility of the physician(s). Konica Minolta Medical & Graphic, Inc. is not responsible for any diagnostic/treatment conditions or diagnostic/treatment results.

Trademark

Company names and product names in this manual are trademarks or registered trademarks of their respective owners.

Please note that ©, ® and ™ marks are omitted hereafter.

Copyright © 2011 Konica Minolta Medical & Graphic, Inc. All Rights Reserved.

Term description

The meanings of terms used in this operation manual are as follows:

Terms	Explanation
AeroDR Detector	The collective term indicating the AeroDR 1717HQ is referred to as AeroDR Detector.
CS-7	The image processing workstation CS-7 is referred to as CS-7.
Calibration	Processing for calibrating the characteristics of the AeroDR Detector for each pixel.
Bucky	A wall stand or bucky table exposure stage for housing the AeroDR Detector.
PoE	An abbreviation for Power over Ethernet. Provides power using an Ethernet cable.
Wired cable	Collective term indicating AeroDR I/F Cable and AeroDR UF Cable.

Structure of pages

①

②

⑤



③

3.4 • Registration and selection of the AeroDR Detector

3.4.1 Registration of the AeroDR Detector

By registering the AeroDR Detector on the CS-7, it is possible to move it between different exposure rooms.

- Make sure that all the devices in the destination exposure room are usable.
- Insert the AeroDR Detector into AeroDR Battery Charger in the destination exposure room.
 - Registration process will start.

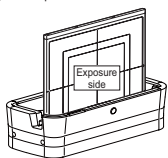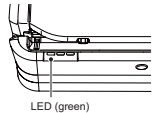- Once the registration is completed, a buzzer will sound on the AeroDR Battery Charger, and the LED (green) will change from slow flashing to lit.
 
- Confirm that the AeroDR Detector icon is displayed on the CS-7.

IMPORTANT

- Once it is registered in a new exposure room, it will not be usable in the previous exposure room. When returning to a previous exposure room, perform the registration operation again.

HINT

- Charging will also start when the AeroDR Detector is inserted into the AeroDR Battery Charger, and the AeroDR Battery Charger LED (blue) will light.

3.4.2 Selection of the AeroDR Detector

An AeroDR Detector is selected as follows depending on the number registered on each CS-7.

Number of registered AeroDR Detectors	Selection method
1	<ul style="list-style-type: none"> AeroDR Detector is selected automatically when that AeroDR Detector is ready to be selected. The AeroDR Detector is not selected if it is not set in a bucky that matches the order information.
Multiple	<ul style="list-style-type: none"> The AeroDR Detector that was selected immediately beforehand will be selected automatically if it is ready to be selected. If there is no AeroDR Detector in the bucky that matches the order information, no AeroDR Detector will be selected.


HINT

- The LED (green) of the selected AeroDR Detector is lit.

3.4.3 Manual selection of the AeroDR Detector

Selecting the AeroDR Detector manually is performed with the following procedure.

- Press the selection switch of the AeroDR Detector that will be used.

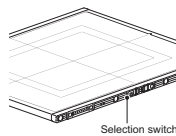


- After selection is completed, the LED (green) of the AeroDR Detector is lit.

Reference

- The AeroDR Detector can also be selected manually from the CS-7. For details, refer to the "CS-7 Operation Manual".

Chapter 3

43

Number	Item	Description	Icon
①	Item heading	Describes the titles of described content.	-
②	Operation procedure	The operating procedure is described in sequential numerical steps.	-
③	Hint	Describes important information.	
④	Reference	Describes reference items. Refer to these as necessary.	
⑤	Important items	Describes the important items for operation. Be sure to read them.	

9

Chapter 1

Safety Precautions & Warnings

This chapter describes precautions and warnings
to ensure safe use of this device.

1.1 • Symbols relating to safety

1.1.1 Safety alert symbol



This is a "safety alert symbol". This symbol alerts you to matters and/or operation potentially hazardous to yourself and other people. Read these messages and follow the instructions carefully.

1.1.2 Warning notice (signal words)

Signal words indicate the degree of potential hazards in the use of the product.

Signal words include the following three types, which are used according to risk of damage caused by danger and the severity of damage.

DANGER

Indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.

WARNING

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION

Indicates a potentially hazardous situation which, if not avoided, could result in minor or moderate injury. It may also be used to indicate hazardous situation where only physical damage is likely to occur.

1.1.3 Description of graphic symbols



Indicates the off position for the main power switch of this device.



Indicates the on position for the main power switch of this device.



Indicates the position where the ground is attached.



Indicates a B-type Applied Parts medical device.



Indicates that full caution is required when handling this device.



Indicates that it is necessary to read the User's Manual before use or operation of this device.



Indicates devices including radio frequency transmitters.

CE 0197

It means conformity with the Medical Devices Directive 93/42/EEC. 0197 is Notified Body number.

CE !

It means conformity with the Radio Equipment and Telecommunications Terminal Equipment Directive 1999/5/EC. And the alert sign (!) means that is categorized Class 2 radio equipment.

1.2 • Warning labels


Various warning labels are attached to this device on the locations shown below. Do not remove these labels from this device. Warning labels are there to make sure that the user recognizes potential hazards when operating this device.

- * If a warning label is too dirty or damaged to read, contact Konica Minolta technical representatives to have a new warning label attached, and redisplay by parts replacement. (There is a fee for this service.)

1.2.1 AeroDR Detector



1.2.2 AeroDR Battery Charger

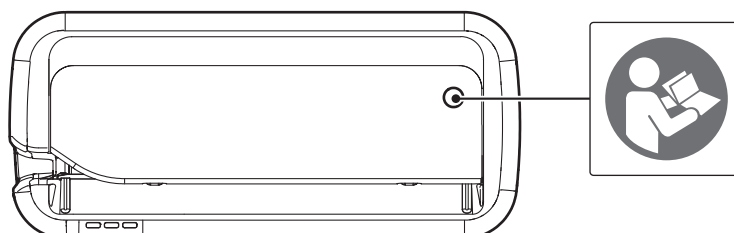

1.3 • Safety precautions

Read all safety precautions thoroughly before using this device.

Be sure to observe the safety precautions described in this section.

1.3.1 Precautions before usage

CAUTION

- The users (hospitals and clinics) hold responsibility for the usage and maintenance of this device. Do not use this device unless you are a physician or certified person under law.
- This device excluding CS-7 is suitable for use in the patient environment. (PC used for CS-7 is not suitable for use in the patient environment.)
- Confirm that this device is operating normally before using.
- When a problem occurs with this device, turn the power off, attach an appropriate sign, such as “malfunction”, on this device, and contact Konica Minolta technical representatives.
- This device is not explosion-proof, so do not use any flammable or explosive gas near this device.
- For the basic operation of the computer, display monitor, and optional parts for this device, refer to their operation manuals.
- Please follow the rules and regulations of your relevant authorities in the disposal of this product, accessories, options, consumables, media and their packing materials.



For EU member states only

This symbol means: Do not dispose of this product together with your household waste!

Please refer to the information of your local community or contact our dealers regarding the proper handling of end-of-life electric and electronic equipments.

Recycling of this product will help to conserve natural resources and prevent potential negative consequences for the environment and human health caused by inappropriate waste handling.

1.3.2 Precautions for usage

WARNING

- Take note of the following when using this device:
 - Do not subject the AeroDR Detector to strong shocks or excessive loads by dropping it, etc.
 - For the load applied to the AeroDR Detector, do not exceed an overall load of 150 kg or a partial load of 100 kg/80φ.
 - Do not disassemble or modify this device.
 - Do not connect any devices that were not purchased from Konica Minolta.
 - Do not turn the power switch off or pull out the power cable while the system is operating.
 - Be careful not to drop the AeroDR Detector on any part of a person's body by catching the AeroDR I/F Cable or AeroDR UF Cable on your feet.
- The AeroDR I/F Cable and AeroDR UF Cable are connected to the AeroDR Detector using magnetic force. When moving the AeroDR Detector, do not hold onto the cable, and always hold on the AeroDR Detector. Also, do not grasp and pull the AeroDR Detector forcefully.
- If there is any smoke, odor, or abnormal sound, it may cause a fire if use is continued, so immediately turn the power switch off, unplug the power plug from the wall outlet, and contact Konica Minolta technical representatives.
- Take note of the following to reduce the risk of fire, electric shock, or electrical leakage:
 - Use specified cables for the power cable, etc.
 - Use a wall outlet with the correct rating as a power source.
 - Connect the power plug to the wall outlet properly without any slack.
 - Use a grounded power source.
 - If you do not plan to use this device for an extended period of time, unplug the power plug.
 - The supplied power cable is dedicated for this device, so do not use it elsewhere.
 - Avoid exposure to liquid such as water.
 - Make sure that foreign material, such as pieces of metal or wire, does not get inside.
 - Do not allow any metal or conductive objects to come into contact with the spring connector of the AeroDR I/F Cable.
 - Do not handle the power plug with wet hands.
 - Do not let soil or dust accumulate on the power plug, AeroDR I/F Cable, or AeroDR UF Cable.
 - Do not use extension cables.
 - Do not connect many plugs to a single electrical outlet.
 - Do not damage the power cable, AeroDR I/F Cable, or AeroDR UF Cable. Also, do not use damaged cables.
 - Do not block the ventilation openings.

**WARNING**

- Do not lean on or put pressure on the AeroDR Battery Charger installed on a wall.
- If there is any abnormality in appearance such as deformation of the housing or a crack, stop using the device immediately and contact Konica Minolta technical representatives.

**CAUTION**

- People with an implanted pacemaker or implanted defibrillator may feel palpitation or dizziness.
- When using the AeroDR I/F Cable, observe the following:
 - Remove the cable by holding the connector housing.
 - Do not let the cable get pinched by doors and do not place heavy objects on it.
 - Do not bend or pull the cable excessively.
 - Make sure that the cable is properly connected to the AeroDR Detector without wobbling.
 - Do not connect the connector housing backwards.
- Use on a flat base. Use the recommended adapter when you need to perform exposure on a stretcher or a place where load is to be applied locally.
- The AeroDR Detector has wireless antennae positioned at 2 places. Do not block them with the body or metal because that will interfere with, disconnect or slow down wireless communication.
- Pressing the power switch causes each LED (green, orange, blue) to light temporarily or flash. After this, only green begins to flash in a slow cycle. Please check the lighting or flashing of each color once.
- Do not use devices that emit electromagnetic waves such as high-frequency therapy equipment, mobile phones, or pocket pagers, close to this device.
- Take note of the reception status for radios and TVs near this device, since interference may occur in them when this device is in use.
- The battery capacity is designed to allow the life period calculated based on the standard usage of the AeroDR Detector. It can be replaced with a new battery for a fee if replacement is necessary after the warranty period due to battery breakdown or decrease in capacity.
- Take note of the following if the AeroDR Detector is damaged and fluorescent medium or lead is exposed:
 - Immediately stop using the device, and contact Konica Minolta technical representatives.
 - If the fluorescent medium gets in your eyes, do not rub and instead wash with running water immediately.
 - If you have swallowed any of the fluorescent medium or if the fluorescent medium has gotten into your eyes, administer first-aid treatment immediately, and consult a doctor.
 - If the medium comes into contact with your skin directly, wash the affected area thoroughly with water.
 - Use and store in a location inaccessible to children.
- Use under the specified environmental conditions. Failure to do so may result in degradation of performance or malfunction.
- Be careful not to get your hand caught when setting the AeroDR Detector into the AeroDR Battery Charger.

1.3.3 Precautions regarding electromagnetic waves**EMC Statement**

This device has been tested and found to comply with the limits for medical devices in IEC 60601-1-2: 2007. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation. The device generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to other devices in its vicinity. However, there is no guarantee that interference will not occur in a particular installation.

Whether this device does cause harmful interference to other devices can be determined by turning this device off and on. If it causes harmful interference, the user is encouraged to try to correct the interference by 1 or more of the following measures:

- Reorient or relocate the receiving device.
- Increase the separation between the devices.
- Connect this device into a wall outlet on a circuit different from that to which the other devices are connected.
- Contact Konica Minolta technical representatives.

Supplementary information regarding IEC 60601-1-2: 2007

- (1) Take precautions against this device especially regarding EMC. Install and put into service according to the electromagnetic compatibility (EMC) information provided in the manual (Table 1 - Table 4).
- (2) Do not use mobile phones or pocket pagers in the vicinity of this device. Use of mobile phones or pocket pagers near this device can cause errors in operation due to electromagnetic wave interference, so such devices should be turned off in the vicinity of this device.
- (3) Cable list
 - Power cable (3 m/3-Wire/No Shielding)
 - Ethernet cable (max 100 m/100 BASE-TX)
 - Ethernet cable (PoE) (30 m/No Shielding)
 - Various AeroDR I/F Cables
 - Various AeroDR UF Cables
 - AeroDR Collimator Cable Set
 - Various AeroDR XG Cable Sets
- (4) The use of accessories, transducers and cables other than those sold by KONICA MINOLTA MEDICAL & GRAPHIC, INC. as internal components, may result in increased emissions or decreased electromagnetic immunity of this device.
- (5) Do not use this device adjacent to or stacked with other devices. If adjacent or stacked use is necessary, confirm normal operation in the configuration in which this device will be used.
- (6) Specifications regarding RF transmitters frequency:
 - Frequency: 5150 to 5350 MHz, 5470 to 5850 MHz
 - Modulation: OFDM
 - Maximum effective radiation power: +15 dBm
 - This device may be interfered with by other devices that conform to CISPR emission requirements.

1.3 Safety precautions


Table 1

Guidelines and manufacture's declaration - electromagnetic emissions		
This device is intended for use in the electromagnetic environment specified below. The customer or the user of this device should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidelines
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	This device is suitable for use in all establishments including the following: Domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Table 2

Guidelines and manufacturer's declaration - electromagnetic immunity			
This device is intended for use in the electromagnetic environment specified below. The customer or the user of this device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidelines
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact	± 6 kV contact	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%. Mains power quality should be that of a typical commercial or hospital environment.
	± 8 kV air	± 8 kV air	
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines	± 2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
	± 1 kV for input/output lines	± 1 kV for input/output lines	
Surge IEC 61000-4-5	± 1 kV differential mode	± 1 kV differential mode	Mains power quality should be that of a typical commercial or hospital environment.
	± 2 kV common mode	± 2 kV common mode	
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U_T (>95% dip in U_T) for 0.5 cycle	<5% U_T (>95% dip in U_T) for 0.5 cycle	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterrupted power supply or a battery.
	40% U_T (60% dip in U_T) for 5 cycles	40% U_T (60% dip in U_T) for 5 cycles	
	70% U_T (30% dip in U_T) for 25 cycles	70% U_T (30% dip in U_T) for 25 cycles	
	<5% U_T (<95% dip in U_T) for 5 sec	<5% U_T (<95% dip in U_T) for 5 sec	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
[NOTE] U_T is the AC mains voltage prior to application of the test level.			

Table 3

Guidelines and manufacturer's declaration - electromagnetic immunity			
This device is intended for use in the electromagnetic environment specified below. The customer or the user of this device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidelines
Conducted RF IEC 61000-4-6	3 Vms 150 kHz to 80 MHz	[3] V	Portable and mobile RF communications equipment should be used no closer to any part of this device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = [1.2] \sqrt{P}$ $d = [1.2] \sqrt{P}$ 80 MHz to 800 MHz $d = [2.3] \sqrt{P}$ 800 MHz to 2.5 GHz
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	[3] V/m	where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a , should be less than the compliance level in each frequency range ^b . Interference may occur in the vicinity of equipment marked with the following symbol: 
[NOTE] At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
[NOTE] These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which this device is used exceeds the applicable RF compliance level above, this device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating this device.			
b Over the frequency range 150 kHz to 80 MHz, field strength should be less than [3] V/m.			

Continued on the next page

1.3 Safety precautions

Table 4

Recommended separation distance between portable and mobile RF communications equipment and the device			
This device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of this device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and this device as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of the transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d=[1.2] \sqrt{P}$	80 MHz to 800 MHz $d=[1.2] \sqrt{P}$	800 MHz to 2.5 GHz $d=[2.3] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	8
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
[NOTE] At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
[NOTE] These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

1.3.4 Precautions regarding wireless communication



CAUTION

- This device is equipped with a communication function that operates via wireless LAN. Conformance is required with the relevant regulations defined by the countries in which this device is to be used.
- Inappropriate usage may cause interference in radio communication. Also, if this device is modified, approval and warranty according to the radio law of the applicable government will be voided.
- It may affect aeronautical systems, so do not use on-board airplanes.
- This device may be affected by other wireless devices. Make sure that the environment is free of wireless communications.
- AeroDR Detector has been confirmed to comply with the relevant regulations of the following countries:

United States & Canada

Federal Communications Commission Statement / Canadian Department of Communications

- This device complies with Part 15 of FCC Rules and RSS-Gen of IC Rules. Operation is subject to the following 2 conditions:
(1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of this device.

- Le présent appareil est conforme aux la partie 15 des règles de la FCC et CNR d'Industrie Canada applicables aux appareils radio exempts de licence. L'exploitation est autorisée aux deux conditions suivantes : (1) l'appareil ne doit pas produire de brouillage, et (2) l'utilisateur de l'appareil doit accepter tout brouillage radioélectrique subi, même si le brouillage est susceptible d'en compromettre le fonctionnement.

- This transmitter must not be co-located or operated in conjunction with any other antenna or transmitter.
- 5.15-5.25 GHz band is restricted to indoor operations only.

- La bande 5 150-5 250 MHz est restreints à une utilisation à l'intérieur.

- Frequency Tolerance: +/-20ppm.

- The available scientific evidence does not show that any health problems are associated with using low power wireless devices. There is no proof, however, that these low power wireless devices are absolutely safe. Low power Wireless devices emit low levels of radio frequency energy (RF) in the microwave range while being used. Whereas high levels of RF can produce health effects (by heating tissue), exposure of low-level RF that does not produce heating effects causes no known adverse health effects. Many studies of low-level RF exposures have not found any biological effects. Some studies have suggested that some biological effects might occur, but such findings have not been confirmed by additional research. AeroDR System has been tested and found to comply with FCC/IC radiation exposure limits set forth for an uncontrolled environment and meets the FCC radio frequency (RF) Exposure Guidelines in Supplement C to OET65 and RSS-102 of the IC radio frequency (RF) Exposure rules.

- Les connaissances scientifiques dont nous disposons n'ont mis en évidence aucun problème de santé associé à l'usage des appareils sans fil à faible puissance. Nous ne sommes cependant pas en mesure de prouver que ces appareils sans fil à faible puissance sont entièrement sans danger. Les appareils sans fil à faible puissance émettent une énergie radioélectrique (RF) très faible dans le spectre des micro-ondes lorsqu'ils sont utilisés. Alors qu'une dose élevée de RF peut avoir des effets sur la santé (en chauffant les tissus), l'exposition à de faibles RF qui ne produisent pas de chaleur n'a pas de mauvais effets connus sur la santé. De nombreuses études ont été menées sur les expositions aux RF faibles et n'ont découvert aucun effet biologique. Certaines études ont suggéré qu'il pouvait y avoir certains effets biologiques, mais ces résultats n'ont pas été confirmés par des recherches supplémentaires. Aero DR system a été testé et jugé conforme aux limites d'exposition aux rayonnements énoncés pour un environnement non contrôlé et respecte les règles d'exposition aux fréquences radioélectriques (RF) CNR-102 de l'IC.

FCC WARNING

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Continued on the next page

European Union

European Union Directives Conformance Statement

AeroDR Detector conforms with the Declaration of Conformity (DoC) to Directive 1999/5/EC (to R&TTE Directive). This device purposely connects to an access point of a 5 GHz network.

AeroDR Detector is used for the following countries indicated by country code (ISO3166 2-letter code).

AT	BE	BG	CY	CZ	DK
EE	FI	FR	DE	GR	HU
IE	IT	LV	LT	LU	MT
NL	PL	PT	RO	SK	SI
ES	SE	GB	IS	LI	NO
CH					

Taiwan

Observe the following based on the regulations stated in Article 10 of Administration Measures for Low-power Radiation Electric Machines.

Article 12

As for type-approval qualified low-power radio-frequency generator, no company, firm, or user may arbitrarily change the frequency, increase the power or alter the function or characteristics of the original design without prior permission.

Article 14

The use of low-power radio-frequency electric machines shall not affect air flight safety or disturb lawful communication. If disturbance is found, the use of such electric machines shall be suspended immediately and such use shall not be resumed until the disturbance is eliminated. Lawful communication mentioned in the preceding paragraph refers to wireless communication operation in accordance with the provisions of the Telecommunication Law.

Low-power radio-frequency electric machines shall tolerate disturbance from lawful communication or from radio-wave radiation electric machine for industrial, scientific or medical use.

China

- Clearly indicate the technical indicators and the scope of usage in the accompanying document, and explain the usage of all controls, adjustments, and switches, etc.
 - Operating frequency range: 5725-5850 MHz
 - Transmit power: ≤ 500 mW and ≤ 27 dBm
 - Equivalent isotropic radiated power (EIRP): ≤ 2 W and ≤ 33 dBm
 - Maximum radiation power density: ≤ 13 dBm/MHz and ≤ 19 dBm/MHz (EIRP)
 - Frequency Tolerance limit: 20 ppm
 - Transmit power at the band edge (EIRP): ≤ 80 dBm/Hz (≤ 5725 MHz or ≥ 5850 MHz)
 - Spurious emissions:
 - ≤ -36 dBm/100 kHz (30-1000 MHz)
 - ≤ -40 dBm/1 MHz (2400-2483.5 MHz)
 - ≤ -40 dBm/1 MHz (3400-3530 MHz)
 - ≤ -33 dBm/100 kHz (5725-5850 MHz)
 (Note: Other than ± 2.5 -times supported channel bandwidth)
 - ≤ -30 dBm/1 MHz (Other 1-40 GHz)
- Do not arbitrarily change frequency or increase transmit power (including the added radio frequency amplifier). In addition, do not mount an antenna without permission, or change to another transmission antenna.
- When in use, do not cause harmful interference in other legal wireless communication services. If you find interference to be occurring, discontinue use immediately. After taking measures to eliminate interference, you may continue using the device.
- If you use low power radio equipment, you must accept interference from various wireless services or radiation interference from industrial, scientific and medical application equipment.
- Do not use near a plane or an airport.

South Korea

Because this wireless equipment may be affected by radio interference, it cannot provide services related with life safety.

Thailand

AeroDR Detector conforms to NTC technical requirement.

Brazil

This equipment operates secondarily, that is, it doesn't benefit from protection against harmful interference, even from stations of the same type, and cannot cause interference to systems that operate primarily.

AeroDR Detector has been approved by ANATEL in compliance with the procedures regulated by Resolution 242/2000 and complies with the applicable technical requirements.

For more information, please visit:

<http://www.anatel.gov.br>

Mexico

This equipment operates on a secondary basis; consequently, you must accept harmful interferences from equipment of the same type, and cannot cause interferences to systems operating on a primary basis.

1.3.5 Precautions for installing, moving, and storing**! WARNING**

- Take note of the following when moving this device not including the AeroDR Detector:
 - Do not subject to shocks or vibration.
 - Do not start moving until the power is turned off, and operation has stopped completely.
 - Do not move with the power cable or any other cable connected.
- When installing the AeroDR Battery Charger on a table top, observe the following:
 - Install on a base that can withstand maximum patient weight because load is inserted when setting the AeroDR Detector.
 - The contact area for the AeroDR Battery Charger must be larger than the area of the AeroDR Battery Charger so that it will not fall down when the AeroDR Detector is set or when people touch it by accident.
 - Install on a horizontal and stable surface.
 - Do not use a base that has a surface made of slippery material such as teflon.
 - Connect power cable and wires so that no one will trip.
 - Install in a place easy to set the AeroDR Detector.
 - Install so that the vent on the back is not blocked.

! CAUTION

- Because connections of the X-ray device can only be made by Konica Minolta or its designated contractors, contact Konica Minolta or its designated contractors.
- Contact Konica Minolta or dealers specified by Konica Minolta to install or move this device.
- For the X-ray devices enabled to connect, contact Konica Minolta technical representatives.
- Since it is required to meet the specification provided by the X-ray device manufacturer to connect with an X-ray device, contact Konica Minolta or dealers specified by Konica Minolta.
- Take note of the following when installing or storing this device.
 - Do not install or store in a location where it may be adversely affected by atmospheric pressure, temperature, humidity, ventilation, sunlight, dust, salt-air, or air containing sulfur.
 - Do not install or store in a location where it is not stable, ventilation is insufficient, the difference in light-dark is great, electromagnetic waves are generated, or where subject to vibration or shock.
 - Do not install or store in a location where chemical agents are used or stored.
 - Do not install this device facing up or upside down.
 - Do not install the AeroDR Interface Unit and the AeroDR Generator Interface Unit stacked on each other.

 **CAUTION**

- Connect the AeroDR Generator Interface Unit to an X-ray device that conforms to IEC 60601 or equivalent standard.
- Contact Konica Minolta technical representatives when installing or moving this device.
- Do not use access points other than those on this device.

1.3.6 Precautions regarding maintenance

 **WARNING**

- Perform the maintenance and inspection periodically. In addition to the user periodical maintenance that needs to be performed, periodical maintenance by a service engineer is also required.
- If there are stains such as body fluids, clean and disinfect.

 **CAUTION**

- Based on the warranty, parts that are no longer under warranty (1 year) can be replaced for a fee.
- Turn off the power and disconnect the power plug from the wall outlet before cleaning or maintaining this device.
- Securely connect the power cable, AeroDR I/F Cable, and AeroDR UF Cable after cleaning and maintenance.
- Clean the dirt from between the protective cover and the exterior, and the terminal using a commercial plastic brush. Do not clean with sharp or hard objects.
- Take care regarding the following when disinfecting the AeroDR Detector.
 - Use ethanol for disinfection, isopropanol for disinfection, or commercial chlorine bleach, or 0.5% hypochlorite (10-fold dilution of household bleach) when disinfecting. However, bleach and hypochlorite are corrosive, so wash the bleach off well to avoid corrosion.
 - Dampen a lint-free, soft cloth with disinfecting solution, and use after wringing it thoroughly. Do not apply disinfecting solution onto the wired connection connector and LED when cleaning.
 - Disinfecting solution is a chemical agent, so follow the precautions of the manufacturer.
- Fully charge battery once a month even if the AeroDR Detector has not been used for a long time.

1.3.7 Precautions on service life

 **CAUTION**

Service life

Name	Service life
AeroDR Detector	6 years
AeroDR Interface Unit	6 years
AeroDR Generator Interface Unit	6 years
AeroDR Battery Charger	6 years
AeroDR Access Point	6 years
AeroDR I/F Cable	6 years
AeroDR UF Cable	6 years

- The above service life is valid only if the product has been properly operated while following the precautions for use and performing the specified maintenance. (By self certification <our data>)
- The service life may differ depending on usage conditions and environment.
- Some component parts of this device are commercially available parts that have a short cycle of model changes, therefore, it might not be possible to supply service parts even within the service life. In addition, related component parts may need to be replaced to maintain compatibility at the time of model change.

1.4 • R&TTE DECLARATION of CONFORMITY

Chapter 2

Product Overview

This chapter describes the overview of this device.

2.1 • Overview of this device

This section describes the functions of this device as well as a system configuration and connection examples using this device.

2.1.1 Functions

This device consists of AeroDR Detector, AeroDR Interface Unit, AeroDR Generator Interface Unit, AeroDR Battery Charger, AeroDR Access Point, etc. With AeroDR Detector, diagnostic X-ray digital image data is generated by irradiation signal from an X-ray device and sent to the CS-7.

Furthermore, the AeroDR Detector can be connected with (or inserted in) an AeroDR Interface Unit or AeroDR Battery Charger or removed from them.

2.1.2 System configuration and connection examples


The system configuration and connection examples are as follows.

● Basic configuration example


Number	Name	Functions
①	AeroDR Detector	<ul style="list-style-type: none">AeroDR Detector is of B-type Applied Parts.
②	AeroDR Interface Unit	<ul style="list-style-type: none">Supplies power to the AeroDR Generator Interface Unit and the AeroDR Access Point, and supplies power to and charges the battery of the AeroDR Detector when an AeroDR I/F Cable or AeroDR UF Cable is used. It also has a hub function.4 AeroDR Detectors can be connected via wireless connection and 2 via wired connection.The expansion AeroDR Interface Unit is required for connecting the third and fourth AeroDR Detectors via wired connection.
③	AeroDR Generator Interface Unit *	Relays signals between the X-ray device, the AeroDR Detector, and the CS-7.
④	AeroDR Battery Charger *	Charges the AeroDR Detector. It also has the registration function for the AeroDR Detector.
⑤	AeroDR Access Point *	Used for wireless connection with the AeroDR Detector.
⑥	AeroDR I/F Cable	Used for wired connection with the AeroDR Detector.
⑦	AeroDR UF Cable *	Used when the AeroDR Detector is set in the bucky.
⑧	CS-7	Controls the reception, management, and output of this device's image data.
⑨	AeroDR XG Cable *	Performs signal relay between the X-ray device and the AeroDR Generator Interface Unit.
⑩	AeroDR Collimator Cable *	Performs exposure field signal relay between the X-ray device and the AeroDR Generator Interface Unit.
⑪	AeroDR S-SRM Cable *	Performs signal relay between the X-ray device operation panel and the AeroDR Generator Interface Unit.
⑫	S-SRM *	Synchronizes AeroDR Generator Interface Unit and X-ray device simply. Exposure is performed using the AeroDR Generator Interface Unit.

* Optional product.

● Basic connection example

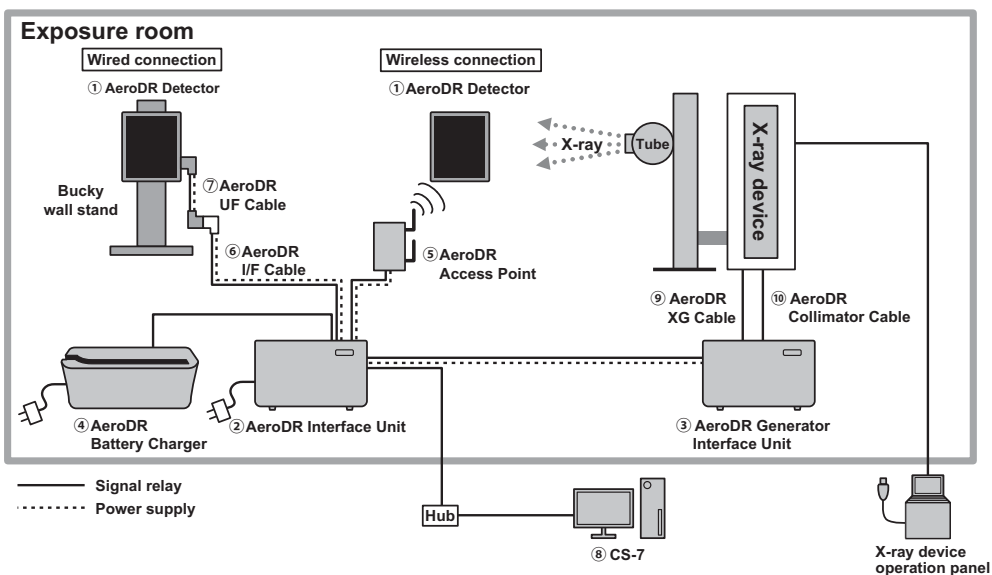


● S-SRM connection example

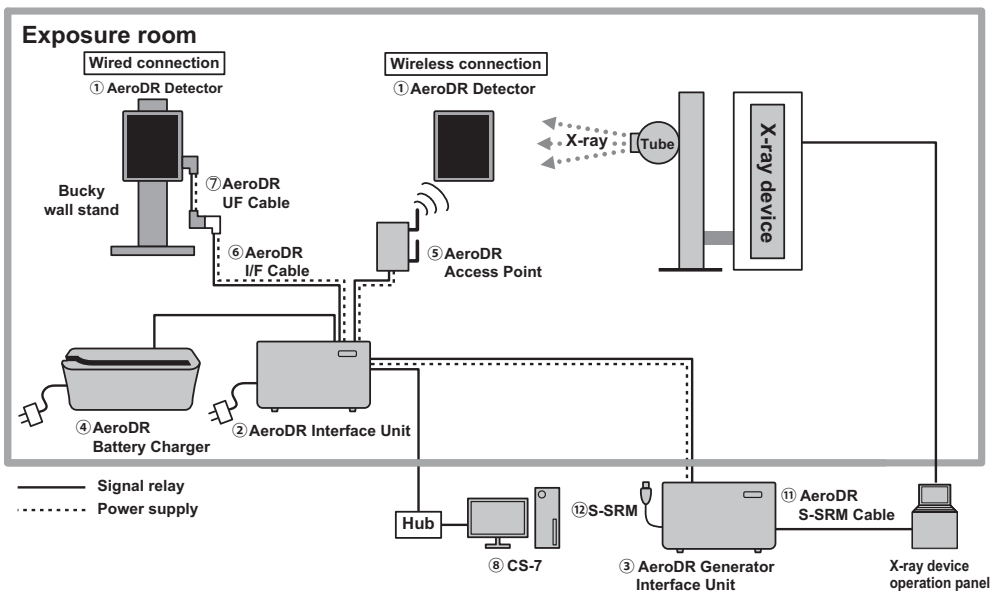



2.2 • Component names and functions

2.2.1 AeroDR Detector

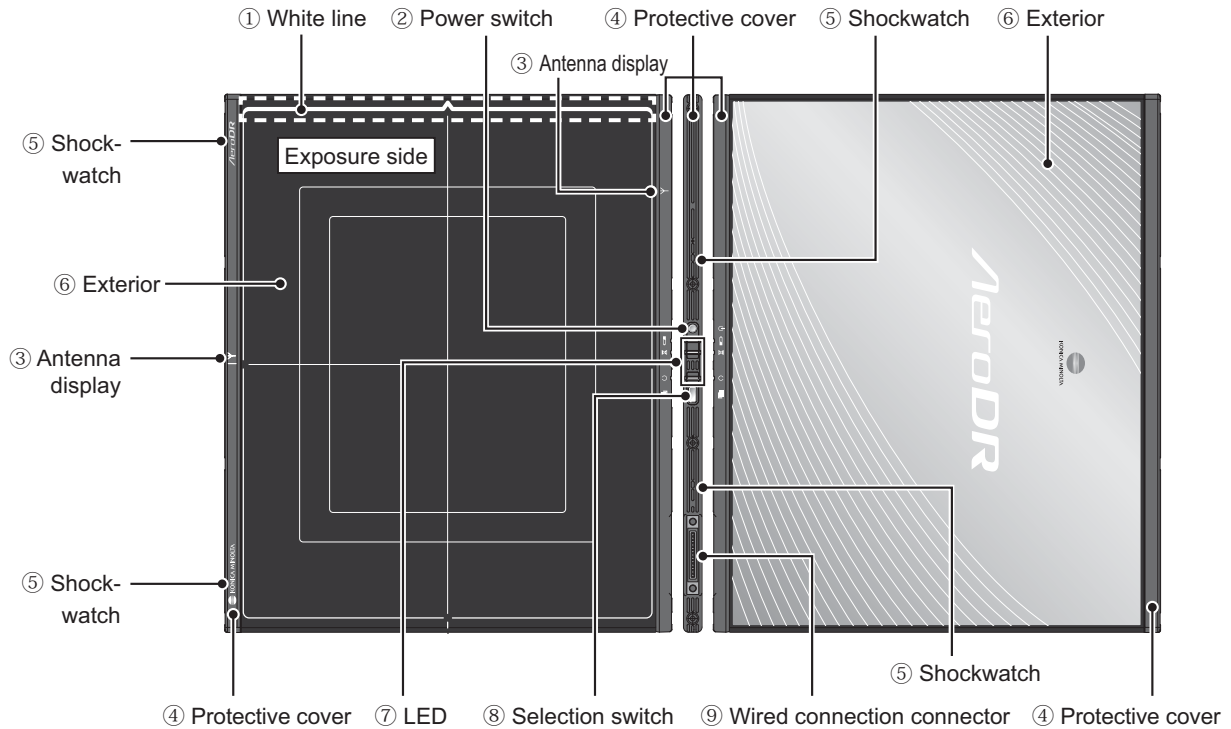
The component names and functions of the AeroDR Detector are as follows.




Number	Name	Functions
①	White line	Indicates the direction to place the AeroDR Detector in during exposure. When exposing in portrait, place the white line upward. When exposing in landscape, place the white line to the left or right. (Left and right are set during installation according to the exposure environment.)  Reference <ul style="list-style-type: none"> For the placement of the AeroDR Detector during exposure, refer to "3.2.2 AeroDR Detector orientation".
②	Power switch	Used to turn the AeroDR Detector on/off.
③	Antenna display	Displays the place where a wireless antenna is attached.
④	Protective cover	Absorbs external shocks.
⑤	Shockwatch	Becomes red if the AeroDR Detector is subjected to a certain amount of shock.
⑥	Exterior	Protects the internal parts.
⑦	LEDs	Displays the status of the AeroDR Detector.  Reference <ul style="list-style-type: none"> For the display patterns and status of the LEDs, refer to "4 Status (LED) Display".
⑧	Selection switch	Notifies the CS-7 that this AeroDR Detector will be used for the exposure.
⑨	Wired connection connector	Connects to the AeroDR Battery Charger, AeroDR I/F Cable, and AeroDR UF Cable.

2.2.2 AeroDR Interface Unit


The component names and functions of the AeroDR Interface Unit are as follows.

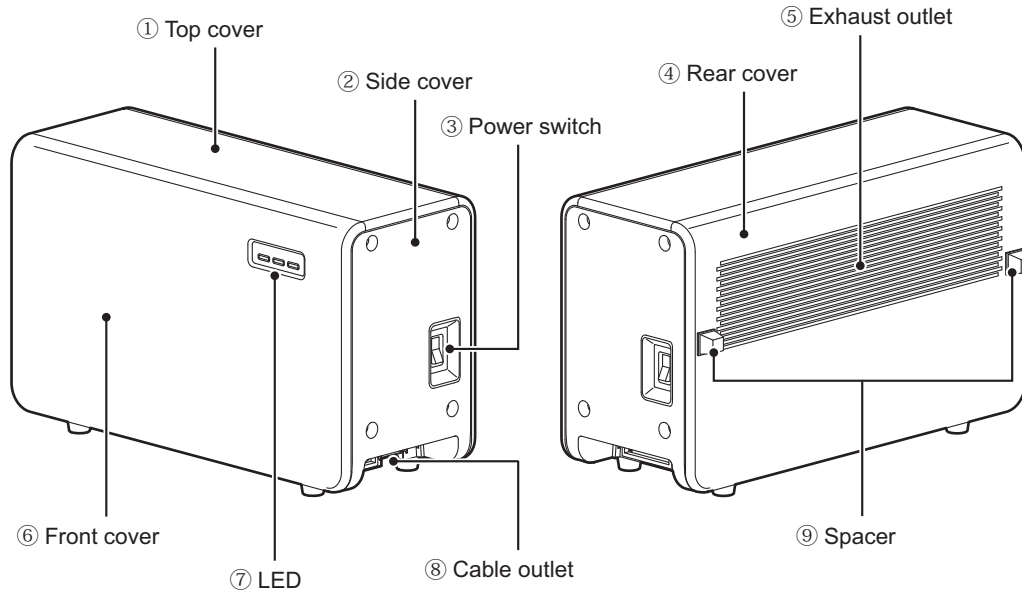


Number	Name	Functions
①	Top cover	Protects the internal parts.
②	Side cover	Protects the internal parts.
③	Power switch	Turns the AeroDR Interface Unit on/off.
④	Rear cover	Protects the internal parts.
⑤	Exhaust outlet	Exhausts internal heat.
⑥	Front cover	Protects the internal parts.
⑦	LEDs	Displays the status of the AeroDR Interface Unit.  Reference • For the display patterns and status of the LEDs, refer to "4 Status (LED) Display".
⑧	Cable outlet	Outlet for various cables.
⑨	Spacer	Prevents exhaust outlet from being blocked after installation.

2.2.3 AeroDR Generator Interface Unit


The component names and functions of the AeroDR Generator Interface Unit are as follows.

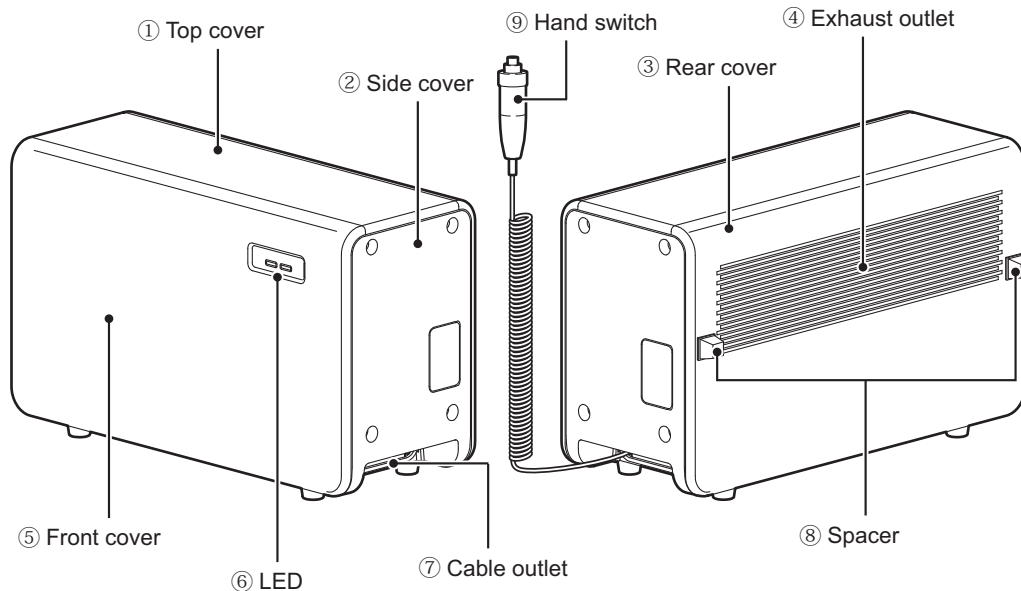

Number	Name	Functions
①	Top cover	Protects the internal parts.
②	Side cover	Protects the internal parts.
③	Rear cover	Protects the internal parts.
④	Exhaust outlet	Exhausts internal heat.
⑤	Front cover	Protects the internal parts.
⑥	LEDs	Displays the status of the AeroDR Generator Interface Unit.  Reference • For the display patterns and status of the LEDs, refer to "4 Status (LED) Display".
⑦	Cable outlet	Outlet for various cables.
⑧	Spacer	Prevents exhaust outlet from being blocked after installation.
⑨	Hand switch	When S-SRM connection is adopted, a hand switch is installed in the AeroDR Generator Interface Unit.

2.2.4 AeroDR Battery Charger


The component names and functions of the AeroDR Battery Charger are as follows.

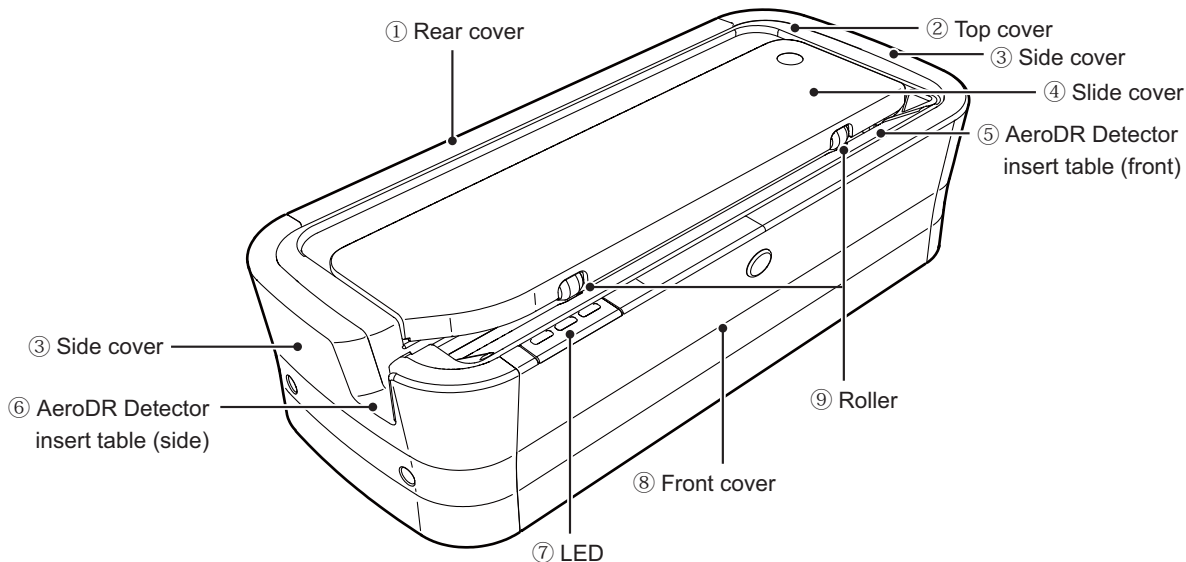

Number	Name	Functions
①	Rear cover	Protects the internal parts.
②	Top cover	Protects the internal parts.
③	Side cover	Protects the internal parts.
④	Slide cover	Protects internal parts, and prevents dust from getting inside the AeroDR Battery Charger.
⑤	AeroDR Detector insert table (front)	Guide for inserting the AeroDR Detector from the front.
⑥	AeroDR Detector insert table (side)	Guide for inserting the AeroDR Detector from the side.
⑦	LEDs	Indicates the status between the AeroDR Detector and the AeroDR Battery Charger. <div style="border: 1px solid black; padding: 2px; display: inline-block;"> Reference • For the display patterns and status of the LEDs, refer to "4 Status (LED) Display". </div>
⑧	Front cover	Protects the internal parts.
⑨	Roller	Prevents external friction when you insert the AeroDR Detector.

2.2.5 AeroDR Access Point


The component names and functions of the AeroDR Access Point are as follows.

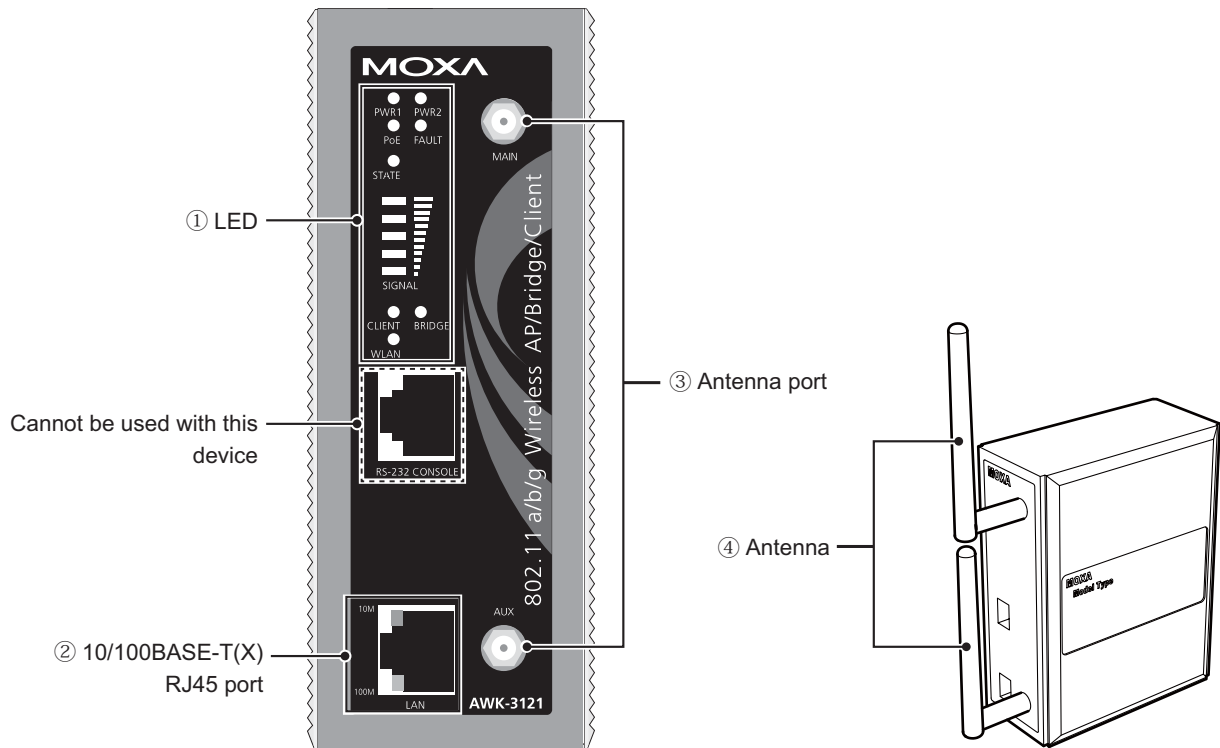

Number	Name	Functions
①	LEDs	Displays the status of the AeroDR Access Point. <div style="border: 1px solid black; padding: 2px; display: inline-block;"> Reference </div> <ul style="list-style-type: none"> For the display patterns and status of the LEDs, refer to "4 Status (LED) Display".
②	10/100BASE-T(X) RJ45 port	Used for connection to the AeroDR Interface Unit.
③	Antenna port	Port to attach antenna.
④	Antenna	Receives wireless signal.

2.2.6 AeroDR UF Cable


The component names and functions of the AeroDR UF Cable are as follows.

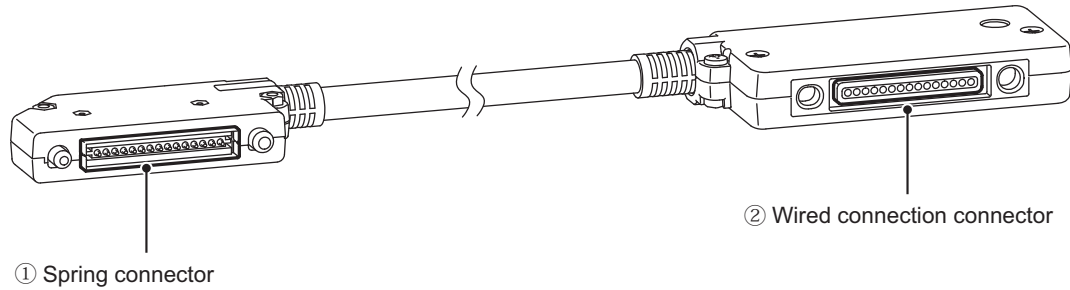

Number	Name	Functions
①	Spring connector	Connects to the wired connection connector of the AeroDR Detector.
②	Wired connection connector	Connects to the spring connector of the AeroDR I/F Cable.

2.2.7 AeroDR I/F Cable

The component name and function of the AeroDR I/F Cable are as follows.

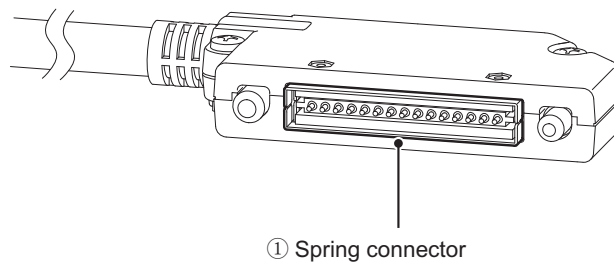

Number	Name	Functions
①	Spring connector	Connects to the wired connection connector of the AeroDR Detector or AeroDR UF Cable.

Chapter 3

General Operations

This chapter describes general operation methods of this device.

3.1 • Startup and shutdown

Normally, this device is used with the power on. When not used for more than 1 day, operate the start-up/shutdown as follows.

Reference


- Refer to the “CS-7 Operation Manual” regarding on/off for the CS-7.

3.1.1 Startup of this device

Startup of this device is performed with the following procedure.


1 Start the AeroDR Interface Unit.

- Turn the power switch of the AeroDR Interface Unit on, and confirm that the LED (green) lights.

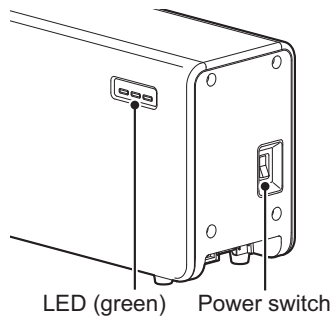


The AeroDR Generator Interface Unit is started.

- When the power switch of the AeroDR Interface Unit is turned on, power is supplied to the AeroDR Generator Interface Unit, and the LED (green) will flash.

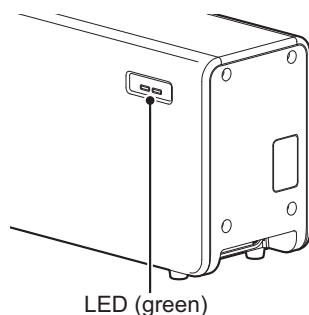

The AeroDR Access Point is started.

- When the power switch of the AeroDR Interface Unit is turned on, power is supplied to the AeroDR Access Point, and the LED (umber) on the AeroDR Access Point will light.

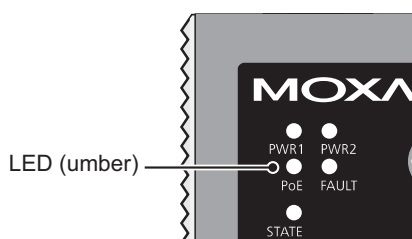


2 Start the CS-7.

- Start the CS-7 by turning the power switch of the CS-7 on.


3 Start the AeroDR Detector.

- If the AeroDR Detector is inserted into the AeroDR Battery Charger with a wireless connection, the AeroDR Detector should be removed.
- For a wired connection, confirm that the AeroDR UF Cable or the AeroDR I/F Cable is connected securely to the wired connection connector of the AeroDR Detector.
- Next, press the power switch of the AeroDR Detector for 2 seconds and turn it on, and confirm that the LED (green) is slowly flashing or lit.

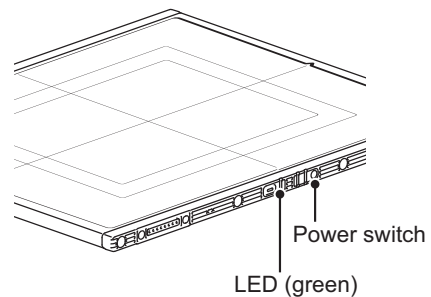

4 Start the AeroDR Battery Charger.

- When the power cable is connected to the wall outlet, power of the AeroDR Battery Charger is turned on. Confirm that the LED (green) is slowly flashing.

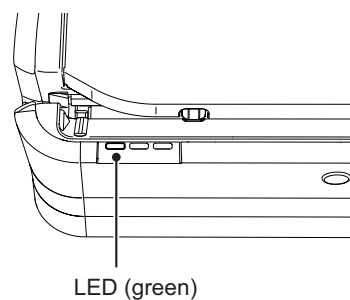


5 Confirm that the AeroDR Detector is ready for use on the CS-7.

3.1.2 Shutdown of this device

Shutdown of this device is performed with the following procedure.

1 Shut down the AeroDR Detector.


- Press the power switch of the AeroDR Detector for 5 seconds to turn it off, and confirm that the LED (green) is turned off.

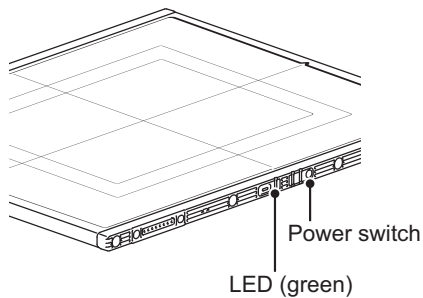

2 Shut down the CS-7.


3 Shut down the AeroDR Interface Unit.

- Turn the power switch of the AeroDR Interface Unit off, and confirm that the LED (green) is turned off.

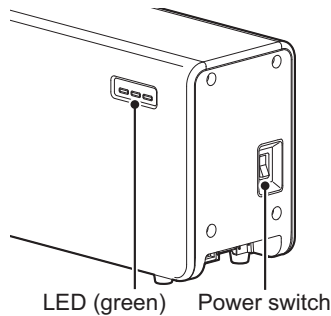


The AeroDR Generator Interface Unit shuts down.

- When the power switch of the AeroDR Interface Unit is turned off, the power supply to AeroDR Generator Interface Unit is terminated. The LED (green) will turn off on the AeroDR Generator Interface Unit.

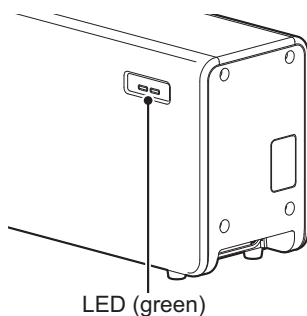


The AeroDR Access Point shuts down.

- When the power switch of the AeroDR Interface Unit is turned off, the power supply to AeroDR Access Point is terminated. The LED (umber) on the AeroDR Access Point will turn off.

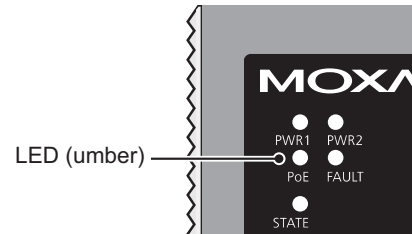

4 Shut down the AeroDR Battery Charger.

- When the power cable is removed from the wall outlet, the power of the AeroDR Battery Charger is turned off and the LED (green) is turned off.

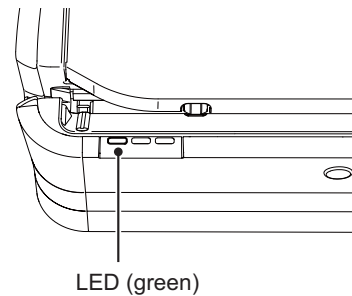

3.2 • Operation of AeroDR Detector

3.2.1 Exposure


Exposure with this device is performed with the following procedure.

● Exposure under the basic connection example

- 1 Perform examination registration with CS-7.
- 2 Check that this device is ready to expose images, and then prepare for the exposure.
- 3 Push the exposure switch of the X-ray device to perform the exposure.
 - When the exposure is completed, images are stored in the AeroDR Detector and will then be converted to digital data and sent to the CS-7 sequentially.
- 4 Check that the exposed image is displayed on the CS-7.


● Exposure under the S-SRM connection example

- 1 Perform examination registration with the CS-7.
- 2 Check that this device is ready to expose images, and then prepare for the exposure.
- 3 Push the S-SRM's hand switch to the first stage.
 - The exposure preparation signal is sent to the X-ray device.

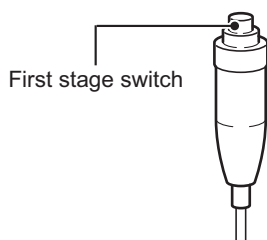

4 Push the S-SRM's hand switch up to the second level to perform exposure.

- Exposure is performed from the X-ray device to produce X-ray images.
- When the exposure is completed, images are stored in the AeroDR Detector and will then be converted to digital data and sent to the CS-7 sequentially.

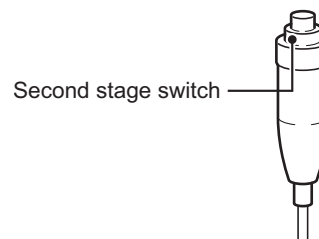

5 Check that the exposed image is displayed on the CS-7.

HINT

- If the AeroDR Detector remains unused for a long time (time can be set) it transitions to the sleep mode.
- When the CS-7 is ready to expose, it recovers from the sleep mode.


Reference

- Regarding the operation of CS-7, refer to the "CS-7 Operation Manual".


3.2.2 AeroDR Detector orientation

Change the orientation of the AeroDR Detector according to the exposure body part. Expose with the white line upward when exposing in portrait, and with the white line to the side when exposing in landscape.

● When exposing in portrait

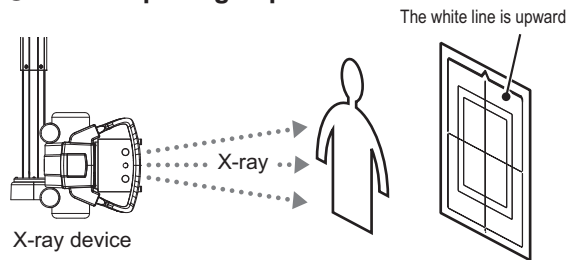

● When exposing in landscape

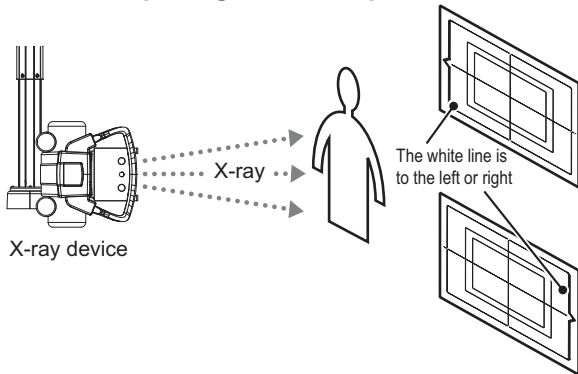

IMPORTANT

- The side with the white line is the exposure surface.
- When exposing in landscape, the direction of the white line is set according to exposure environment.

3.2.3 Precautions for exposure

During exposure give adequate attention to the following items.

● High-dose exposure

- When high-dose exposures are performed continuously, afterimages of the last exposure will occasionally be visible. The affect of afterimages in this device can be resolved by leaving longer intervals between exposures, so make the intervals between exposures longer than usual when performing high-dose exposure.
- During high-dose exposure, continual use of a lead or other marker at the same position can cause burning in of the afterimage. Therefore, avoid using it in the same position. Also, if image burn-in is visible, run gain calibration.

● Exposure time

- Set exposure time to within 700 msec including X-ray irradiation delay time. Otherwise, sometimes it cannot expose normally.

● Exposure area

- The white line of AeroDR Detector indicates the exposure size.
- During exposure, place the exposure body part within the white line.

● Grid

- Use the following grid when exposing.

	AeroDR 1717HQ	
Grid density	40 lp/cm or 34 lp/cm	
Grid ratio	Variety	
Convergence distance	Variety	
Angle error	40 lp/cm: 1.0° or less 34 lp/cm: 1.7° or less	

IMPORTANT


- Capped grid is recommended when laying the grid over the AeroDR Detector on the table top.

● S-SRM connection

- When S-SRM connection is adopted, make sure to perform exposure using the hand switch connected to the AeroDR Generator Interface Unit.
- If exposure is performed using the hand switch on the operation panel of the X-ray device, exposure synchronized to the AeroDR SYSTEM is not possible.

● **Wired connection**

- When performing exposure with a wired connection, fasten the wired cable horizontally to the wired connection connector of the AeroDR Detector. If connected at an angle, transverse (noise) sometimes gets into acquired images after exposure.

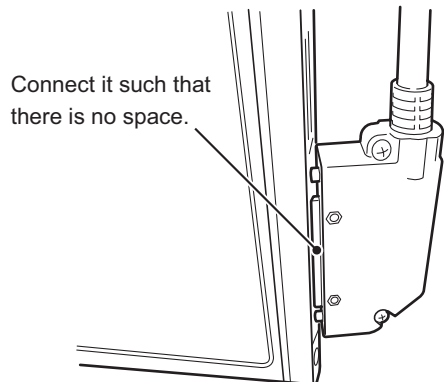

● **Wireless communications environment**

- With a wireless environment, it is possible that errors such as no wireless connection, wireless communication terminates, and lengthened exposure cycle time might occur.

 **HINT**

- Problems with a wireless communications environment can occur in the following conditions:
 - Installed location of the AeroDR Access Point is not good.
 - When inserted in the bucky or stretcher, the opening in the bucky is too small and there is no passage for the radio waves.
 - The radio waves are not emitted due to metal parts close to the antenna, which changes the antenna characteristics.
 - For exposures where the body touches the AeroDR Detector directly, the radio waves are not emitted if the body completely covers the antennas positioned in 2 places.
 - Other devices use the same radio band, and cause interference.

.....

3.3 • Charging of AeroDR Detector

Charge the AeroDR Detector when the LED (blue) on the AeroDR Detector lights or flashes, or when the battery level gets low in the status display of the AeroDR Detector on the CS-7.

IMPORTANT

- During charging, if the AeroDR Detector should become hot, stop charging immediately.
- If charging errors occur repeatedly, contact Konica Minolta technical representatives.

HINT


- The AeroDR Detector can be charged when the power is either on or off.
- The AeroDR Detector can be used while stopping charging in progress.
- To charge the AeroDR Detector with the AeroDR Battery Charger when you do not use it for a long time such as during the night, charge the AeroDR Detector with its power turned off.
- Even if you use the battery equipped with this device for repeated quick charging and use, deterioration of battery life is small compared to a lithium-ion battery.

3.3.1 Charging with AeroDR Battery Charger

Inserting the AeroDR Detector into the AeroDR Battery Charger performs charging of the AeroDR Detector.


- 1 Confirm that the LED (green) of the AeroDR Battery Charger is slowly flashing.
- 2 Slowly insert the AeroDR Detector with its exposure side pointed toward the operator until the buzzer sounds. When inserted, charging of the AeroDR Detector begins.

Insert from the side

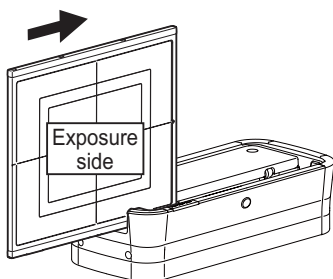


Slide slowly

Insert from the front (top)

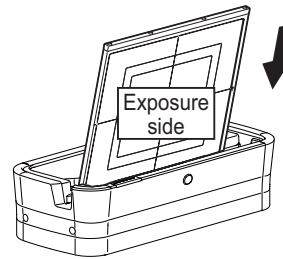

Load slowly

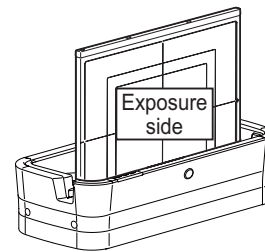

Inserted

- 3 Once the AeroDR Detector is inserted correctly and charging starts, the LED (blue) on the AeroDR Battery Charger will light.
- 4 Once charging of the AeroDR Detector is completed, the LED (blue) on the AeroDR Battery Charger will turn off.

IMPORTANT

- Handle the AeroDR Detector with extreme care when inserting it into the AeroDR Battery Charger.
- The LED of the AeroDR Detector is not visible when the AeroDR Detector is inserted into the AeroDR Battery Charger.
- The wired connection connector of the AeroDR Detector may become warm right after charging on the AeroDR Battery Charger. This often occurs during charging and is not a malfunction.

HINT


- If there is any problem during charging, the LED (orange) on the AeroDR Battery Charger will light. Also, charging will stop when an error occurs.

3.3.2 Charging with the wired cable


Connecting the AeroDR UF Cable or AeroDR I/F Cable to the AeroDR Detector performs charging of the AeroDR Detector.

- 1 Confirm that the LED (green) of the AeroDR Interface Unit lights.
- 2 Securely connect the AeroDR I/F Cable or AeroDR UF Cable to the wired connection connector on the AeroDR Detector. Once it is connected, the AeroDR Detector will start charging.

When the AeroDR I/F Cable is connected

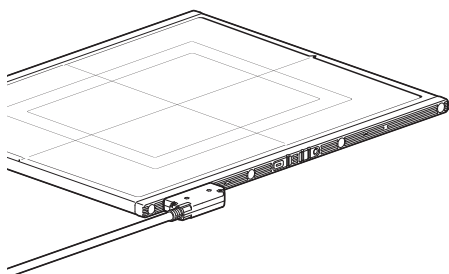

When the AeroDR UF Cable is connected

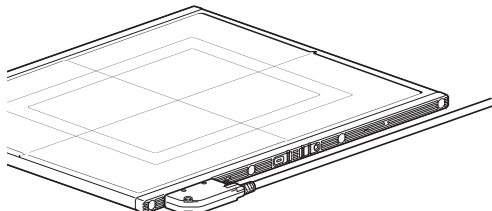

- 3 Once the charging of the AeroDR Detector is higher than 10%, the LED (blue) on the AeroDR Detector will go out.

HINT

- Confirm completion of full charge and the level of battery power with the CS-7.
 - If there is any problem during charging, the LED (orange) on the AeroDR Detector will light. Also, charging will stop when an error occurs.
-

3.3.3 Charging time guide

To fully charge the AeroDR Detector requires the following charging time.

Charging status	Charging time of the AeroDR Detector when the power is off
Via the AeroDR Battery Charger	Approx. 30 min
Via wired cable	Approx. 60 min

IMPORTANT

- When the AeroDR Detector is on, the charging time will be slightly longer as it depends on the operation status.
-

3.3.4 AeroDR Detector charging display

The AeroDR Detector LED (blue) changes according to the level of battery power.

Battery level	LED display
Less than 3% (exposure not possible)	Lit (blue)
Less than 5%	Fast flashing (blue)
Less than 10%	Slow flashing (blue)
10% or more	Off

IMPORTANT


- When the battery runs down completely, all the LEDs go out. Confirm that the LED (green) either lights or flashes when you perform exposure.
-

3.4 • Registration and selection of the AeroDR Detector


3.4.1 Registration of the AeroDR Detector

By registering the AeroDR Detector on the CS-7, it is possible to move it between different exposure rooms.

- 1 **Make sure that all the devices in the destination exposure room are usable.**
- 2 **Insert the AeroDR Detector into AeroDR Battery Charger in the destination exposure room.**
 - Registration process will start.

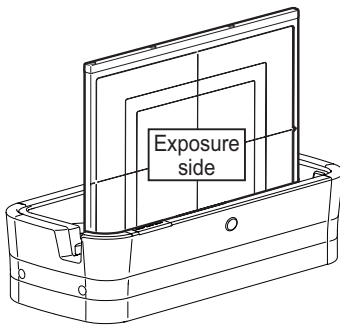

- 3 **Once the registration is completed, a buzzer will sound on the AeroDR Battery Charger, and the LED (green) will change from slow flashing to lit.**

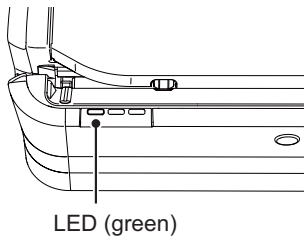

- 4 **Confirm that the AeroDR Detector icon is displayed on the CS-7.**

IMPORTANT

- Once it is registered in a new exposure room, it will not be usable in the previous exposure room. When returning to a previous exposure room, perform the registration operation again.

.....

HINT

- Charging will also start when the AeroDR Detector is inserted into the AeroDR Battery Charger, and the AeroDR Battery Charger LED (blue) will light.

.....

3.4.2 Selection of the AeroDR Detector

An AeroDR Detector is selected as follows depending on the number registered on each CS-7.

Number of registered AeroDR Detectors	Selection method
1	<ul style="list-style-type: none"> • AeroDR Detector is selected automatically when that AeroDR Detector is ready to be selected. • The AeroDR Detector is not selected if it is not set in a bucky that matches the order information.
Multiple	<ul style="list-style-type: none"> • The AeroDR Detector that was selected immediately beforehand will be selected automatically if it is ready to be selected. • If there is no AeroDR Detector in the bucky that matches the order information, no AeroDR Detector will be selected.

HINT


- The LED (green) of the selected AeroDR Detector is lit.

.....

3.4.3 Manual selection of the AeroDR Detector

Selecting the AeroDR Detector manually is performed with the following procedure.

- 1 **Press the selection switch of the AeroDR Detector that will be used.**

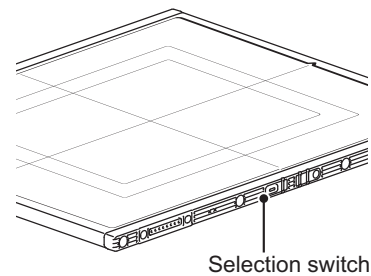

- 2 **After selection is completed, the LED (green) of the AeroDR Detector is lit.**

Reference

- The AeroDR Detector can also be selected manually from the CS-7. For details, refer to the "CS-7 Operation Manual".

.....

3.5 • Calibration

In order for the AeroDR Detector to provide optimal images, perform the following calibration. The calibration is performed with the CS-7.

IMPORTANT

- It is necessary to perform the gain calibration periodically to compensate for changes over time or changes in the exposure environment.

Reference

- For the calibration, refer to the “CS-7 User Tool Operation Manual”.

3.5.1 Offset calibration

Every 3 months, acquire 1 unexposed image and perform calibration.

IMPORTANT

- Perform the offset calibration before the gain calibration operation.
- Perform by waiting at least 10 minutes after the previous exposure.

3.5.2 Gain calibration

Every 3 months, acquire multiple exposed images and perform calibration.

3.6 • Position to affix AeroDR Detector identification label

When using more than 1 AeroDR Detector and affixing identification labels (stickers) to the outside of the AeroDR Detectors, it is recommended to affix the labels to the 2 places (① , ②) shown as follows.

Position to affix AeroDR Detector identification label



IMPORTANT

- Affix the labels only in the recommended places. Failure to do so may cause the labels to come off or image unevenness to occur.

HINT

- Use of commercially available vinyl (Tepa, etc.) labels are recommended for the identification labels (stickers). When possible, use labels that do not easily peel off.
- It is recommended to write the name and identification number registered in the CS-7 on the label.

Chapter 4





Status (LED) Display

This chapter describes the LED display patterns and the status of the respective devices.


4.1 • LED display of respective devices

Status of the respective devices can be confirmed with LEDs.
Check the status of the respective devices, referring to the “LED display pattern”.





LED display pattern

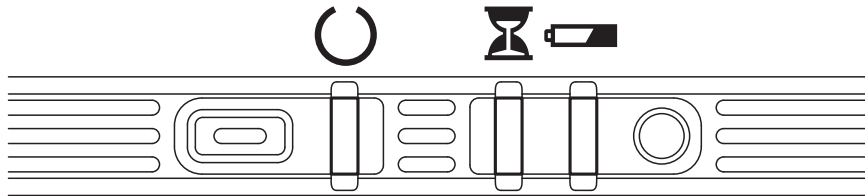Notation	Display pattern
	Off
	Slow flashing
	Fast flashing
	On

4.1.1 AeroDR Detector






: Status LED (green)

Display pattern	Status
	Shutdown condition
	Standing by
	AeroDR Detector being selected
	Exposing





* During startup/shutdown processing, it also flashes fast and is lit.

: Busy/error LED (orange)

Display pattern	Status
	Shutdown condition or standing by
	Exposing or performing maintenance
	Error occurred


* During startup/shutdown processing, it also flashes fast and is lit.

: Battery LED (blue)

Display pattern	Status
	Shutdown condition or battery level is 10% or above
	Battery level is less than 10%
	Battery level is less than 5%
	Battery level is less than 3%

* During startup/shutdown processing, it also flashes fast and is lit.

4.1.2 AeroDR Interface Unit

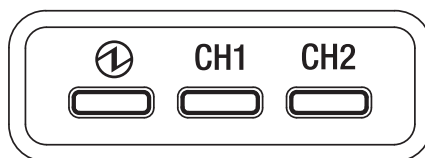

① : Power LED (green)

Display pattern	Status
	Shutdown condition
	Operating


CH1 : Feeding1 LED (blue)

Display pattern	Status
	Shutdown condition or feeding1 is not connected to the AeroDR Detector
	The AeroDR Detector is connected to feeding1

CH2 : Feeding2 LED (blue)

Display pattern	Status
	Shutdown condition or feeding2 is not connected to the AeroDR Detector
	The AeroDR Detector is connected to feeding2

4.1.3 AeroDR Generator Interface Unit

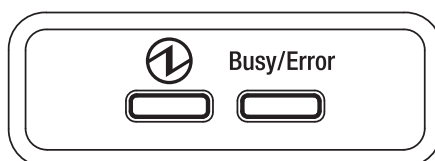


① : Power LED (green)

Display pattern	Status
	Shutdown condition
	Operating and not connected to the CS-7
	Operating and connected to the CS-7

Busy/Error : Busy/error LED (orange)

Display pattern	Status
	Shutdown condition or standing by
	Exposing or performing maintenance
	Error occurred

4.1.4 AeroDR Battery Charger

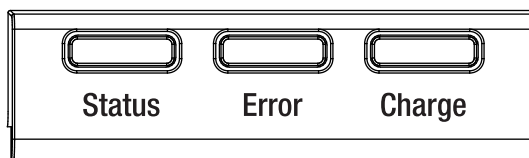

Status : Status LED (green)

Display pattern	Status
	Shutdown condition
	Operating
	Registration processing of the inserted AeroDR Detector is complete

* Because the charger dedicated for charging does not perform registration processing of the AeroDR Detector, the status LED always flashes slowly.


Error : Error LED (orange)

Display pattern	Status
	Shutdown condition or operating normally
	Error occurred

Charge : Feeding LED (blue)

Display pattern	Status
	Shutdown condition or standing by for insertion of the AeroDR Detector Battery charged during AeroDR Detector insertion
	AeroDR Detector battery charging

4.1.5 AeroDR Access Point

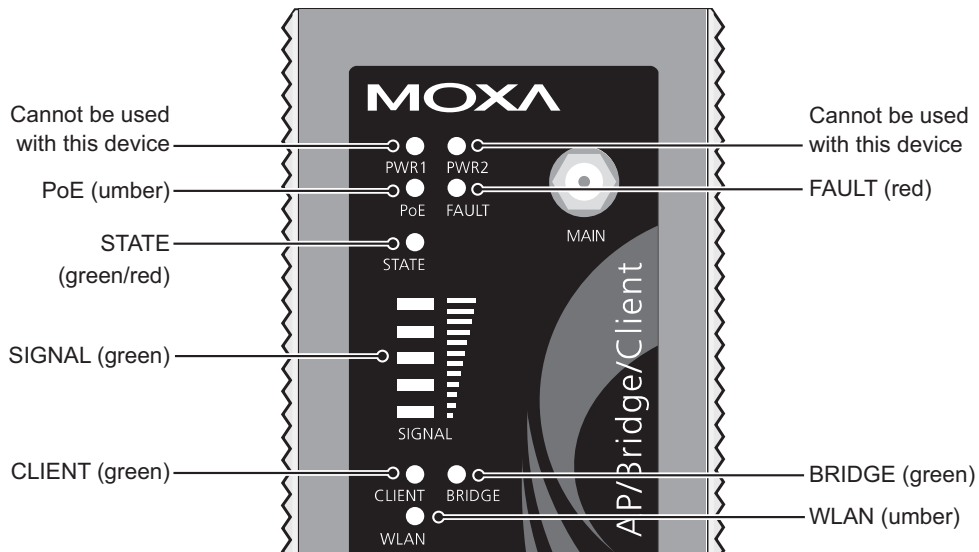

LED	Lighting color	Lighting pattern	Status
PoE	Umber	On	Power is being supplied.
FAULT	Red	Flashing	Error is occurring.
STATE	Green/ Red	Green/on	Lights when operation preparation is completed.
		Red/on	Error is occurring.
SIGNAL	Green	On	Signal strength of wireless. (Only in client mode)
BRIDGE	Green	On	Operating in bridge mode.
CLIENT	Green	On	Operating in client mode.
WLAN	Umber	On	Operating in wireless LAN mode. (Normal)


Chapter 5

Troubleshooting

This chapter describes the countermeasures when the problems occur.

5.1 • Support flow during trouble

If the following problems occur with any of these devices, consult the respective references for countermeasures.

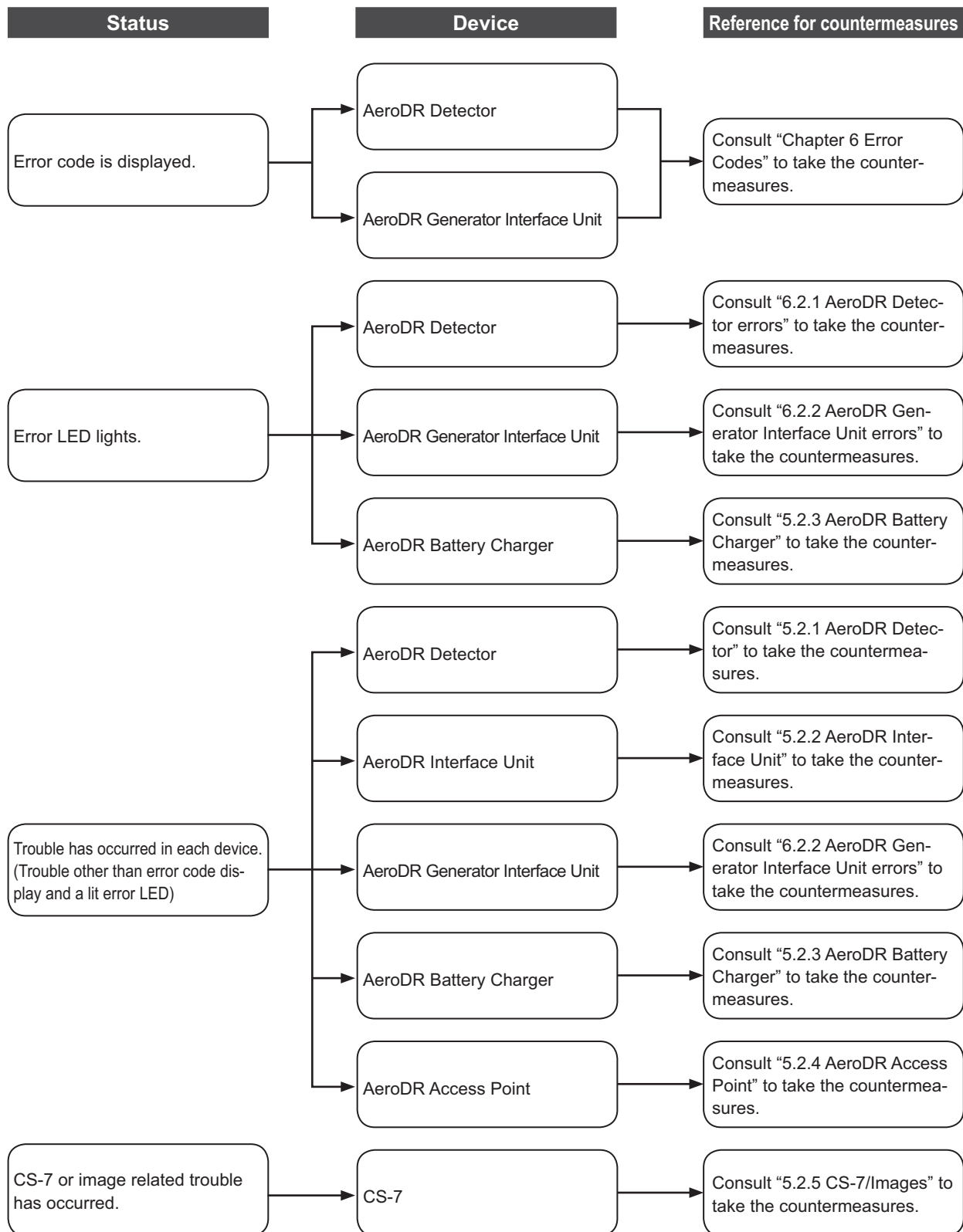

5.2 • Various problems and countermeasures

If the following problems occur with any of these devices, consult the respective references for countermeasures.



- After performing countermeasures, if the problem does not go away, contact Konica Minolta technical representatives.

5.2.1 AeroDR Detector

Status	Error description	Corrective actions
The AeroDR Detector does not start up.	Power does not go on even when the power switch is pressed for 2 seconds or longer.	The battery might be out. Charge it by inserting the AeroDR Detector into the AeroDR Battery Charger for more than 2 minutes, or connect it to a wired cable for more than 6 minutes. Then start it.
The AeroDR Detector does not shut down.	Power does not go off even when the power switch is pressed for 5 seconds or longer.	It is not possible to shut down while exposing. Shut down after exposure is ended.
The status LED (green) is lit, and the busy/error LED (orange) flashes rapidly. ("Ready" is not displayed on the CS-7)	System error is occurring.	If a busy/error LED (orange) continues to flash after 10 minutes has elapsed, shut down the AeroDR Detector. Or, when "Ready" is not displayed on the CS-7, restart it.
When the AeroDR Detector is placed on a smooth surface, the AeroDR Detector is not stably attached to the smooth surface.	AeroDR Detector is warped.	Contact Konica Minolta technical representatives.
	The protective cover is deformed.	
The AeroDR Detector will not go into the bucky.	AeroDR Detector is warped. The protective cover is deformed.	
	The protective cover is deformed.	
The AeroDR Detector cannot be inserted in the AeroDR Battery Charger.	Foreign material is in the wired connection connector of the AeroDR Detector.	Refer to "7.1.2 Cleaning" and remove the foreign material.
	Foreign material is in the bottom of the AeroDR Battery Charger.	
The AeroDR I/F Cable cannot be connected to the AeroDR Detector.	Part of the wired connection connector of the AeroDR Detector is deformed.	Contact Konica Minolta technical representatives.
	The spring connector of the AeroDR I/F Cable is deformed.	
	Foreign material is in the wired connection connector of the AeroDR Detector.	Refer to "7.1.2 Cleaning" and remove the foreign material.
	Foreign material is in the spring connector on the AeroDR I/F Cable.	
The shockwatch changes color to red.	Possible excessive shock to AeroDR Detector.	Contact Konica Minolta technical representatives.
Only wired connection with the AeroDR Detector cannot be used.	The wired cable is not connected properly.	Check that the wired cable is properly connected to the AeroDR Detector.
Only wireless connection with the AeroDR Detector cannot be used.	There is an error in the AeroDR Access Point.	Check that the Ethernet cable is properly connected to the AeroDR Access Point.
	The AeroDR Detector and AeroDR Access Point are being used under poor conditions. <ul style="list-style-type: none"> • Wireless does not connect • Wireless communication terminates • Cycle time is extended 	Check the installation location of AeroDR Detector and AeroDR Access Point. When performance is noticeably lower than at the initial installation of this device, it is possible that the installation environment/usage environment have changed.

Continued on the next page

5.2 Various problems and countermeasures

Status	Error description	Corrective actions
AeroDR Detector wired connection connector is heated up.	AeroDR Detector wired connection connector is heated up immediately after charging with AeroDR Battery Charger.	This is caused by charging and is not a malfunction.
Charging sometimes takes longer.	Charging takes longer when the battery is over discharged.	It takes time for internal components to start. Because it is not abnormal, wait a bit.
Battery LED (blue) flashes fast.	Usage time with the battery has gotten shorter.	It could be that the charging function of the battery has deteriorated. It can be replaced with a new battery for a fee if necessary.
	The number of images that can be exposed has decreased.	
	The charging time has gotten shorter.	



- If the exterior or protective cover is damaged extensively, or in the event of the AeroDR Detector being dropped or subjected to shock, contact Konica Minolta technical representatives.

5.2.2 AeroDR Interface Unit

Status	Error description	Corrective actions
Power LED (green) does not light.	AC power not supplied.	Make sure the power cable is connected correctly.
Feeding1/2 LED (blue) does not light.	Feeding1/2 LED (blue) does not light even when connecting the AeroDR Detector.	Make sure that the AeroDR I/F Cable is connected correctly.
Communications are down between devices, and power LED (green) is off.	The power switch is not on. Or, the power cable is not connected.	Make sure the power switch of the AeroDR Interface Unit is on. Or, make sure the power cable is connected correctly.
The AeroDR Detector is being used with wired connection, but it is not recognized on the CS-7, and feeding1/2 LEDs (blue) are off.	The AeroDR I/F Cable is not connected.	Make sure that the AeroDR I/F Cable is connected to the AeroDR Detector properly.
Battery is not charging while the AeroDR Detector is used with wired connection (the battery level on the CS-7 battery level display does not increase), and feeding1/2 LEDs (blue) are off.		

5.2.3 AeroDR Battery Charger

Status	Error description	Corrective actions
Status LED (green) does not light.	AC power not supplied.	Make sure that the power cable is connected correctly.
Error LED (orange) is on.	-	Contact Konica Minolta technical representatives.
When the AeroDR Detector is set, the error LED (orange) lights.	AeroDR Detector was inserted the wrong way.	Check the insertion orientation of the AeroDR Detector.
	The AeroDR Detector is not set correctly.	Make sure that the AeroDR Detector is set correctly.
Although the AeroDR Detector is inserted, charging does not start, and the status LED (green) is out.	AC power not supplied.	Make sure that the power cable is connected correctly.
Although the AeroDR Detector is inserted, it cannot be registered, and the status LED (green) is flashing.	The Ethernet cable is disconnected.	Make sure that the Ethernet cable is connected correctly.

5.2.4 AeroDR Access Point

If a problem occurs in the AeroDR Access Point, contact Konica Minolta technical representatives.

5.2.5 CS-7/Images

Status	Error description	Corrective actions
Transverse (noise) has gotten into all images acquired from the AeroDR Detector.	The wired cable is not connected properly.	Connect the wired cable spring connector unit horizontally to the wired connector of the AeroDR Detector.
Operation is normal, but problems can be seen with exposure images.	It has been occurring frequently since a certain time.	Perform calibration.
	There is trouble in only 1 image.	Check the exposure method and image processing.
Cannot expose with the CS-7.	CS-7 does not become Ready.	Check the CS-7 start up.
	The icons of devices used on the CS-7 are not displayed.	Confirm that the icon of the device to be used is displayed on the system monitor screen. When the icon of the device is not displayed, check that the device to be used is started. Or, check that the Ethernet cable is properly connected.

Chapter 6

Error Codes

This chapter describes the error codes and countermeasures.

6.1 • Types of error codes and recognition methods



6.1.1 Types of error codes

Error messages are all displayed in a dialog. The types and description of error messages are as follows.

Types	Description
Communication errors	An error has been caused in communications.
Software errors	An error has been caused in the CS-7.
AeroDR Detector errors	An error has been caused in the AeroDR Detector.
AeroDR Generator Interface Unit errors	An error has been caused in the AeroDR Generator Interface Unit.
REGIUS errors	An error has been caused in the REGIUS series.

6.1.2 Recognition method for error codes

The initial letters have been decided for each type of error code. When an error message has been displayed, identify the type of error in accordance with the flow and check the error description and the countermeasures in each item.

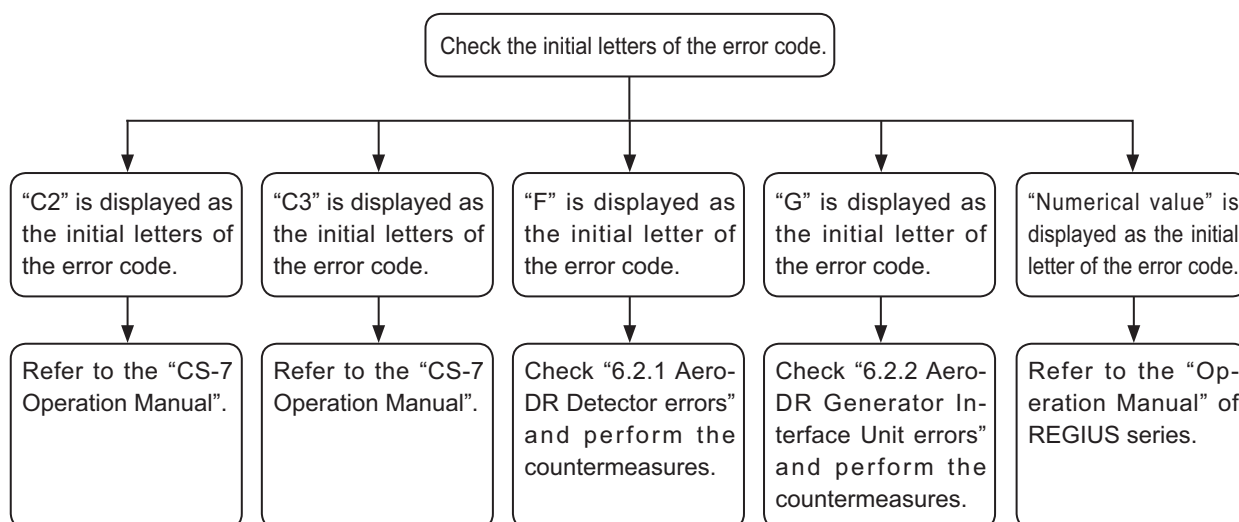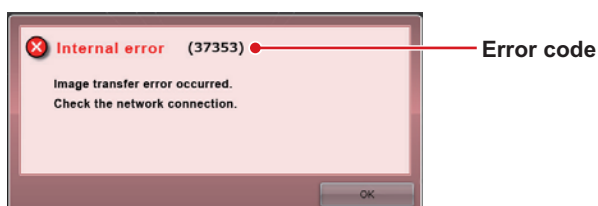

6.2 • Error codes and countermeasures

If the following error codes appear on the CS-7, take the respective countermeasures.

IMPORTANT

- After performing countermeasures, if the problem does not go away, contact Konica Minolta technical representatives.

HINT

- The “*” at the tail of the error code is indicating a character from 0 to 9 or from A to Z.
- For operation of the CS-7, refer to the “CS-7 Operation Manual”.

6.2.1 AeroDR Detector errors

Error codes starting with “F” are indicating AeroDR Detector errors.

Error code	Displayed message	Status (LED) display	Problems/countermeasures	
F01005	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At shutdown of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out	Contact Konica Minolta technical representatives.	
F01012	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out		
F0109H	[AeroDR Detector name] version update error It has failed to update the version. Contact a service representative.	Busy/error LED (orange): lit		
F0200*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out		
F0201*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit		
F0202H F0203H	[AeroDR Detector name] version update error It has failed to update the version. Contact a service representative.			
F0300*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.			
F0301H F0400H	[AeroDR Detector name] version update error It has failed to update the version. Contact a service representative.			
F090** F091** F092** F093**	[AeroDR Detector name] wireless error Wireless communication has malfunc- tioned. Contact a service representative.	LED displayed at that point in time.		Wireless connection cannot be used, but continued operation is possible with the wired connection.
F09402	[AeroDR Detector name] wireless LAN error Wireless LAN has malfunctioned. Contact a service representative.			

Continued on the next page

6.2 Error codes and countermeasures

Error code	Displayed message	Status (LED) display	Problems/countermeasures
F0H00* F0H01*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit	Contact Konica Minolta technical representatives.
F0H022	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out	
F0H03*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.		
F0I00* F0I01* F0I028 F0I03*	[AeroDR Detector name] power source error There is a fault in the power source unit. Contact a service representative.	Busy/error LED (orange): lit	
F0I048 F0I058	[AeroDR Detector name] capacitor error There is a degradation or fault in the capacitor. Contact a service representative.		
F0I06* F0I07*	[AeroDR Detector name] capacitor error There is a fault in the capacitor. Contact a service representative.		
F0I08* F0I09* F0I10* F0I11* F0I12* F0I13*	[AeroDR Detector name] capacitor error There is a degradation in the capacitor. Contact a service representative.	LED displayed at that point in time.	Lithium ion capacitor is degraded, or the power consumption of the whole of this device has increased. Continued use is possible, however contact Konica Minolta technical representatives.
F0I16*	[AeroDR Detector name] power source error There is a fault in the power source unit. Contact a service representative.	Busy/error LED (orange): lit	Contact Konica Minolta technical representatives.
F0I17*	Battery Charger error There is a fault in the Battery Charger. Contact a service representative.		
F0I18*	[AeroDR Detector name] power source error There is a fault in the Battery Charger or the power source unit of the [AeroDR Detector name]. Contact a service representative.		
F0I19*	[AeroDR Detector name] power source error There is a fault in the power source unit. Contact a service representative.	LED displayed at that point in time.	
F0I24* F0I25* F0I27*	[AeroDR Detector name] capacitor error There is a fault in the capacitor. Contact a service representative.	Busy/error LED (orange): lit	
F0I28*	Battery Charger error There is a fault in the Battery Charger. Contact a service representative.		
F0I34* F0I35* F0I36* F0I37* F0I38*	[AeroDR Detector name] cable connection error There is a cable connection error. Please reconnect.		Remove the wired connection cable and reconnect it.

Continued on the next page

Error code	Displayed message	Status (LED) display	Problems/countermeasures
F0I39*	[AeroDR Detector name] power source error There is a fault in the power source unit. Contact a service representative.	Busy/error LED (orange): lit	Contact Konica Minolta technical representatives.
F0J09* F0J10*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.		
F0J112	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out	Contact Konica Minolta technical representatives.
F0P015	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At shutdown of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out	
F2C02* F2C03*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit	
F2C04*	[AeroDR Detector name] wireless LAN error Wireless LAN has malfunctioned. Contact a service representative.	LED displayed at that point in time.	Wireless connection cannot be used, but continued operation is possible with the wired connection.
F2C05*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.		
F2C07*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit	Contact Konica Minolta technical representatives.
F2C08*	[AeroDR Detector name] wireless LAN error Wireless LAN has malfunctioned. Contact a service representative.	LED displayed at that point in time.	Wireless connection cannot be used, but continued operation is possible with the wired connection.
F2C13* F2E0**	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit	
F2E17H	[AeroDR Detector name] version update error It has failed to update the version. Contact a service representative.		
F2E18* F2E19*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.		
F2E20*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out At other time than startup of the AeroDR Detector→ Busy/error LED (orange): lit	Contact Konica Minolta technical representatives.
F2L213 F2M19*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit	
F32051 F35001	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): fast flashing Busy/error LED (orange): out Battery LED (blue): out	
F3600* F3601* F3602* F3603* F3604* F3605*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit	

Continued on the next page

6.2 Error codes and countermeasures

Error code	Displayed message	Status (LED) display	Problems/countermeasures
F36061 F37001	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): fast flashing Busy/error LED (orange): out Battery LED (blue): out	Contact Konica Minolta technical representatives.
F3800* F3801*	[AeroDR Detector name] internal error Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out At other time than startup of the AeroDR Detector→ Busy/error LED (orange): lit	
F38022 F38032 F38042	[AeroDR Detector name] wired LAN error Wired LAN has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out	
F39***	[AeroDR Detector name] wireless error Wireless communication has malfunctioned. Contact a service representative.	LED displayed at that point in time.	Wireless connection cannot be used, but continued operation is possible with the wired connection.
F3C002	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out	Contact Konica Minolta technical representatives.
F3C01* F3C06* F3C1** F3C2**	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit	
F3J00*	[AeroDR Detector name] wireless error Wireless communication has malfunctioned. Contact a service representative.	LED displayed at that point in time.	Wireless connection cannot be used, but continued operation is possible with the wired connection.
F3J012 F3J022 F3J032	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out	Contact Konica Minolta technical representatives.
F3J043 F3J053 F3J063	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Status LED (green): lit Busy/error LED (orange): lit Battery LED (blue): fast flashing	
F3J07* F3J08*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit	
F3K00*	[AeroDR Detector name] internal error Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out At other time than startup of the AeroDR Detector→ Busy/error LED (orange): lit	
F3K01* F3K02* F3K03* F3K04* F3K05* F3K06*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): lit Battery LED (blue): fast flashing At other time than startup of the AeroDR Detector→ Busy/error LED (orange): lit	Contact Konica Minolta technical representatives.
F3K11* F3K12*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit	

Continued on the next page

Error code	Displayed message	Status (LED) display	Problems/countermeasures	
F3K13* F3K14*	[AeroDR Detector name] wireless LAN error Wireless LAN has malfunctioned. Contact a service representative.	LED displayed at that point in time.	Wireless connection cannot be used, but continued operation is possible with the wired connection.	
F3K15* F3K16* F3K17*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit	Contact Konica Minolta technical representatives.	
F3K18* F3K19*	[AeroDR Detector name] wireless LAN error Wireless LAN has malfunctioned. Contact a service representative.	LED displayed at that point in time.	Wireless connection cannot be used, but continued operation is possible with the wired connection.	
F3K20* F3K21*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit	Contact Konica Minolta technical representatives.	
F3K22* F3K23*	[AeroDR Detector name] wireless LAN error Wireless LAN has malfunctioned. Contact a service representative.	LED displayed at that point in time.	Wireless connection cannot be used, but continued operation is possible with the wired connection.	
F3L2**	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out	Contact Konica Minolta technical representatives.	
F3M003	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Status LED (green): lit Busy/error LED (orange): lit Battery LED (blue): fast flashing		
F3M01* F3M02* F3M03* F3M04* F3M05* F3M06* F3M073 F3M08* F3M09* F3M1**	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit		
F3P002	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out		
F3Y00* F3Z02* F3Z03*	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	Busy/error LED (orange): lit		
F3Z042	[AeroDR Detector name] internal error Internal device has malfunctioned. Contact a service representative.	At startup of the AeroDR Detector→ Status LED (green): lit Busy/error LED (orange): fast flashing Battery LED (blue): out		
F4I32*	[AeroDR Detector name] power switch push error Power switch of the AeroDR Detector is held pressed. Release the power switch.	Busy/error LED (orange): lit		The power switch of the AeroDR Detector is pressed and held for 30 seconds or longer. Release the power switch.
F4I33*	[AeroDR Detector name] Selection switch push error The selection switch on AeroDR Detector is continually pressed. Release the selection switch.			The selection switch of the AeroDR Detector is pressed and held for 30 seconds or longer. Release the selection switch.

6.2.2 AeroDR Generator Interface Unit errors

Error codes starting with “G” are indicating AeroDR Generator Interface Unit errors.

Error code	Displayed message	Status (LED) display	Problems/countermeasures
G2010S G2011S	[AeroDR Generator Interface Unit name] startup error There is a startup error. Release the exposure switch.	Busy/error LED (orange): lit	Initial state error of the AeroDR Generator Interface Unit occurred. Release the exposure switch.
G2012S	[AeroDR Generator Interface Unit name] startup error There is a startup error. Contact a service representative.		Contact Konica Minolta technical representatives.
G2020S G2021S	[AeroDR Generator Interface Unit name] startup error There is a startup error. Release the exposure switch.		Initial state error of the AeroDR Generator Interface Unit occurred. Release the exposure switch.
G2022S	[AeroDR Generator Interface Unit name] startup error There is a startup error. Contact a service representative.		Contact Konica Minolta technical representatives.
G2030S G2031S	[AeroDR Generator Interface Unit name] startup error There is a startup error. Release the exposure switch.		Initial state error of the AeroDR Generator Interface Unit occurred. Release the exposure switch.
G2032S	[AeroDR Generator Interface Unit name] startup error There is a startup error. Contact a service representative.		Contact Konica Minolta technical representatives.
G22000	[AeroDR Generator Interface Unit name] exposure error There is a X-ray device exposure link error. Restart the X-ray device and AeroDR Interface Unit.		X-ray device link error of the AeroDR Generator Interface Unit occurred. Restart the X-ray device and the AeroDR Interface Unit.
G2201U	[AeroDR Generator Interface Unit name] exposure error There is a X-ray device exposure link error. Contact a service representative.		Contact Konica Minolta technical representatives.
G230**	[AeroDR Generator Interface Unit name] exposure error There is an internal device error or a communication error. Restart the AeroDR Detector, AeroDR Interface Unit and CS-7.		System error of the AeroDR Generator Interface Unit occurred. Restart the AeroDR Detector, the AeroDR Interface Unit, and the CS-7.
G2500U	[AeroDR Generator Interface Unit name] exposure error There is an internal device error. Restart the AeroDR Interface Unit.		System error of the AeroDR Generator Interface Unit occurred. Restart the AeroDR Interface Unit.
G2501U	[AeroDR Generator Interface Unit name] exposure error There is an internal device error or a communication error. Contact a service representative.		Contact Konica Minolta technical representatives.
G410**	[AeroDR Generator Interface Unit name] exposure error Exposure switch is not released. Release the exposure switch.		Release the exposure switch.

Chapter 7

Maintenance

This chapter describes the items that require periodic maintenance.

7.1 • Maintenance and inspection items

This chapter describes the inspections and cleaning required in order to maintain the use of this device in an optimum condition.

7.1.1 Maintenance schedule

The maintenance and inspection items that the user should perform are as follows.

Maintenance task	Maintenance interval
Checking and cleaning the surface of the AeroDR Detector	Weekly
Checking for external damage to the AeroDR Detector	Weekly
Cleaning the spring connectors of the AeroDR I/F Cable and AeroDR UF Cable	Weekly
Cleaning the wired connection connectors of the AeroDR Detector and AeroDR UF Cable	Weekly
Cleaning the AeroDR Battery Charger	Weekly
Full charge of the AeroDR Detector	Monthly
Offset calibration	Every 3 months
Gain calibration	Every 3 months

IMPORTANT

- To ensure optimum use of this device, be sure to perform periodic maintenance.
- The above task intervals are estimates and vary according to usage.

Reference


- For the offset calibration and gain calibration, refer to the "CS-7 User Tool Operation Manual".

7.1.2 Cleaning

The cleaning methods of the respective devices are as follows.

● AeroDR Detector


- Clean the exterior with a soft lint-free cloth dampened with a small amount of anhydrous ethanol and wrung well.
- To clean the gap between the exterior of the AeroDR Detector and the protective cover, remove dirt using a commercial plastic brush.

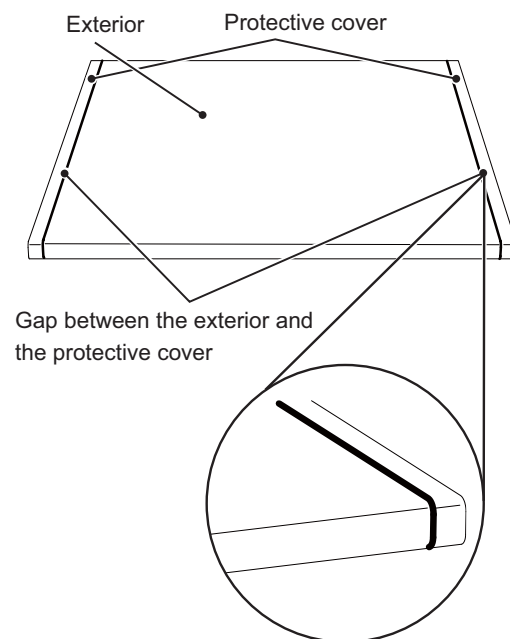


● Spring connector

- If foreign material has adhered to the spring connectors of the AeroDR I/F Cable and AeroDR UF Cable, remove it with a commercial plastic brush.

AeroDR I/F Cable

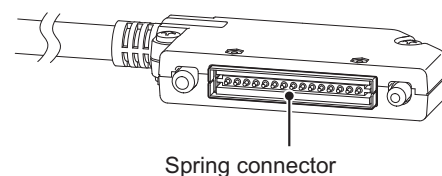


AeroDR UF Cable

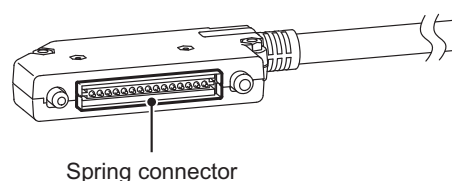

● **Wired connection connector**


- If foreign material has adhered to the wired connection connectors of the AeroDR Detector and AeroDR UF Cable, remove it with a commercial plastic brush.

AeroDR Detector

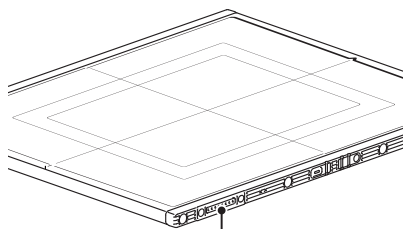

Wired connection connector

AeroDR UF Cable

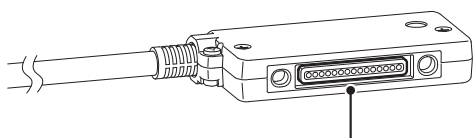


Wired connection connector

● **AeroDR Battery Charger**

- Clean dust on the insert table of the AeroDR Battery Charger with a soft cloth moistened with anhydrous alcohol or water.

AeroDR Detector insert table (front)

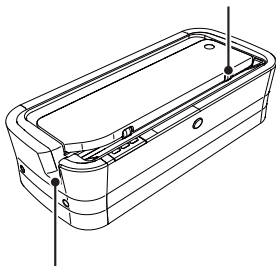

AeroDR Detector insert table (side)

IMPORTANT

- Be careful not to apply any cleaning chemical or liquid onto the spring connectors, the wired connection connectors, and the LEDs.
 - Do not clean with sharp or hard metal objects. If you cannot remove stains, contact Konica Minolta technical representatives.
 - Wear and deformation of the protective cover will occur with the AeroDR Detector due to the way it is handled. The protective cover can be replaced for a fee when the damage becomes extensive, so contact Konica Minolta technical representatives.
-

7.1.3 Disinfection of the AeroDR Detector

If bodily fluid or blood from a patient has contaminated the surface of the AeroDR Detector, disinfect with a soft lint-free cloth dampened with a small quantity of the following disinfectant and wrung well.

- Ethanol for disinfection
- Isopropanol for disinfection
- Commercial chlorine bleach, or 0.5% hypochlorite (10-fold dilution of household bleach)

IMPORTANT

- Bleach and hypochlorite are corrosive, so wash the bleach off well to avoid corrosion.
 - Be careful not to apply any chemical for disinfection onto the wired connection connectors and the LEDs.
-


Chapter 8

Specifications

This chapter describes the specifications of this device.


8.1 • Specifications

8.1.1 AeroDR Detector

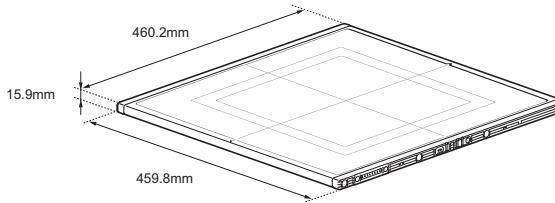
Item	Description
Product name (model name)	AeroDR 1717HQ (AeroDR P-21)
Detection method	Indirect conversion method
Scintillator	CsI (Cesium Iodide)
External dimensions	459.8(W)×460.2(D)×15.9(H)mm 
Weight	AeroDR 1717HQ: 3.6 kg
Pixel size	175 μm
Image area size	425.25×425.95 mm(2,430×2,434 pixels)
AD conversion	16 bit (65,536 gradients)
Usable grid frequency	AeroDR 1717HQ: 40lp/cm, 34lp/cm
Maximum patient weight	Point load: 150 kg@Φ40mm Face load: 300 kg@effective image area overall * Deadweight, even when loaded on the AeroDR Detector, has no effect on images and the AeroDR Detector. The measurement method is based on KM standards.
Communication	Dedicated wired Ethernet connection / wireless LAN (IEEE802.11a compliant)
Drive power	Dedicated power cable drive / battery drive
WLAN encryption	Wireless encryption method : AES Authentication method : WPA2-PSK
Dynamic range	4 digits
Preview display time	On the order of 1 second
Cycle time	Approx. 12 seconds when connected with the dedicated wired connection / approx. 19 seconds when connected with wireless LAN connection
Required time for wired/wireless switching	On the order of 1 second
Battery type	Lithium ion capacitor
Battery charging time empty to full	Within 30 minutes (when using the AeroDR Battery Charger) Within 60 minutes (when using the dedicated wired cable)
Number of exposable images*	120 images/2 hours * 60 second cycle exposure time
Battery duration in standby status*	Approx. 16 hours
Number of days with power off until electricity stored in the battery is gone*	Approx. 5.4 days
Battery expected lifetime	Above the AeroDR Detector product lifetime

* The described performance is all performance after fully charged. Additionally, described performance will vary depending on the usage environment and the usage frequency. (This is not a guarantee of performance.)


8.1.2 AeroDR Interface Unit

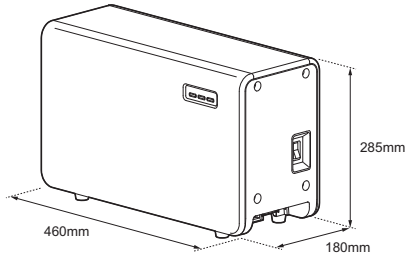Item	Description
Product name (model name)	AeroDR Interface Unit (AeroDR B-1)
Amount of connectable AeroDR Detectors	Wired connection: Up to 2 Wireless connection: Up to 4 AeroDR Access Point is necessary when operating wireless.
Power requirements	AC 100/110/115/120/200/220/230/240 V \pm 10%, single phase 50/60 Hz
Power consumption	With the AeroDR Detector connected : Approx. 80 VA (100-240 V) Without the AeroDR Detector connected: Approx. 33 VA (100-240 V)
External dimensions	460(W) \times 180(D) \times 285(H)mm 
Weight	11.5 kg


8.1.3 AeroDR Generator Interface Unit

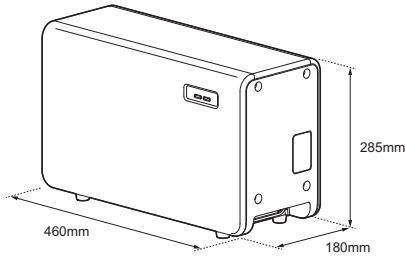Item	Description
Product name (model name)	AeroDR Generator Interface Unit (AeroDR X-1)
Power supply	Supplied from the AeroDR Interface Unit via the Ethernet cable.
External dimensions	460(W) \times 180(D) \times 285(H)mm 
Weight	7.3 kg



8.1.4 AeroDR Battery Charger

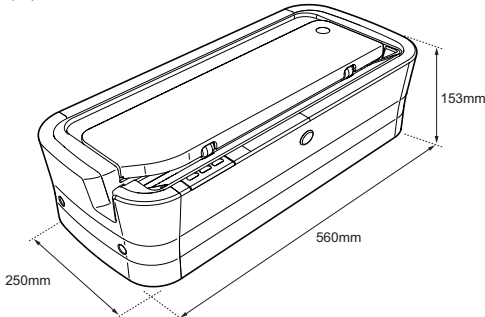Item	Description
Product name (model name)	AeroDR Battery Charger (AeroDR D-1)
Battery charging system	Automatic charging
Power requirements	AC 100/110/115/120/200/220/230/240 V ± 10%, single phase 50/60 Hz
Power consumption	Charging: Approx. 161 VA (100-240 V) Standby : Approx. 25 VA (100-240 V)
External dimensions	560(W)×250(D)×153(H)mm 
Weight	7.2 kg


8.1.5 AeroDR Access Point

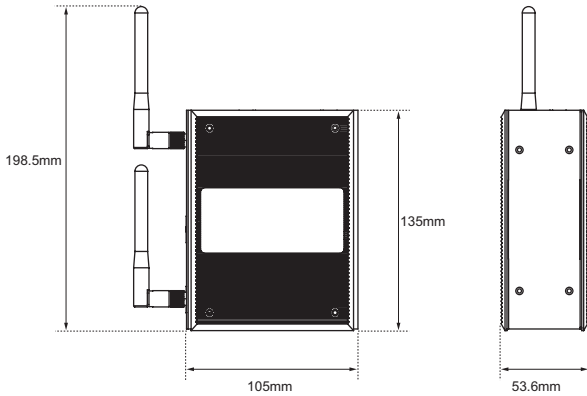Item	Description
Product name (model name)	AeroDR Access Point (AeroDR C-1)
External dimensions	53.6(W)×135(H)×105(D)mm (without antenna) 
Weight	850g

8.1.6 AeroDR UF Cable

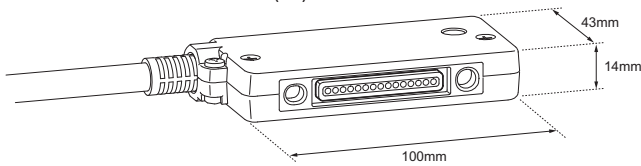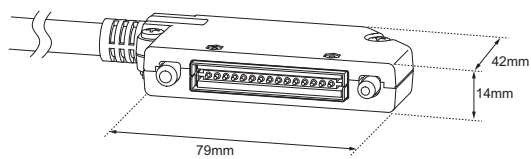Item	Description
Product name	AeroDR UF Cable
Cable length	2m/5m
External dimensions	<p>Wired connection connector (UIF) side</p>  <p>Spring connector (PIF) side</p> 

8.1.7 AeroDR I/F Cable

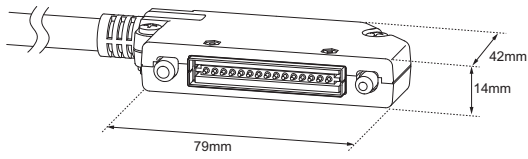Item	Description
Product name	AeroDR I/F Cable
Cable length	10m/20m
External dimensions	



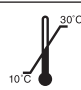

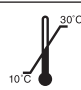







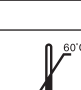

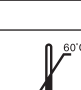

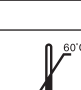

8.1.8 AeroDR XG Cable

Item	Description
Product name	AeroDR XG Cable Set 100V AeroDR XG Cable Set 120V AeroDR XG Cable Set 220V AeroDR XG Cable Set 230V AeroDR XG Cable Set 240V AeroDR XG Cable Set DC24V

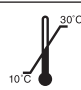8.1.9 AeroDR S-SRM Cable

Item	Description
Product name	AeroDR S-SRM Cable GEX1 AeroDR S-SRM Cable CPX1 AeroDR S-SRM Cable PKX1 AeroDR S-SRM Cable DEX1 AeroDR S-SRM Cable HIX1 AeroDR S-SRM Cable TOX1 AeroDR S-SRM Cable PHX1 AeroDR S-SRM Cable DEX2 AeroDR S-SRM Cable TOX2 AeroDR S-SRM Cable TOX3 AeroDR S-SRM Cable TOX4 AeroDR S-SRM Cable NC

8.1.10 General AeroDR SYSTEM

Item	Description					
Recommended storage and usage environment conditions	When operating	<table border="1"> <thead> <tr> <th>Temperature</th> <th>Humidity</th> </tr> </thead> <tbody> <tr> <td>10 to 30°C </td> <td>35 to 80% RH (ensure no water condensation) </td> </tr> </tbody> </table>	Temperature	Humidity	10 to 30°C 	35 to 80% RH (ensure no water condensation) 
	Temperature	Humidity				
	10 to 30°C 	35 to 80% RH (ensure no water condensation) 				
When not operating	<table border="1"> <thead> <tr> <th>Temperature</th> <th>Humidity</th> </tr> </thead> <tbody> <tr> <td>-10 to 40°C </td> <td>20 to 90% RH (ensure no water condensation) </td> </tr> </tbody> </table>	Temperature	Humidity	-10 to 40°C 	20 to 90% RH (ensure no water condensation) 	
Temperature	Humidity					
-10 to 40°C 	20 to 90% RH (ensure no water condensation) 					
In storage	<table border="1"> <thead> <tr> <th>Temperature</th> <th>Humidity</th> </tr> </thead> <tbody> <tr> <td>-20 to 60°C* </td> <td>20 to 90% RH (ensure no water condensation) </td> </tr> </tbody> </table> <p>* However, performance warranty period when storing at 60°C is 6 months after packing.</p>	Temperature	Humidity	-20 to 60°C* 	20 to 90% RH (ensure no water condensation) 	
Temperature	Humidity					
-20 to 60°C* 	20 to 90% RH (ensure no water condensation) 					
Classification	Safety IEC60601-1 Class I					
Operation mode	Continuous operation					



KONICA MINOLTA

Konica Minolta Medical Imaging U.S.A., Inc.
411 Newark Pompton Turnpike, Wayne, NJ 07470, U.S.A.
TEL. 973-633-1500



EU Authorized Representative:
Konica Minolta Medical & Graphic Imaging Europe B.V.
Frankfurtstraat 40, 1175 RH Lijnden,
The Netherlands
TEL.+31-20659-0260

Konica Minolta Healthcare India Pvt. Ltd.
Office No. 515, 5th Floor, C-Wing, 215-Atrium Centre,
Andheri (East), Mumbai 400 059 India
TEL.+91-22-61916969

Konica Minolta Business Solutions (Canada) Ltd.
369 Britannia Road East, Mississauga,
Ontario, L4Z 2H5, Canada
TEL. 905-890-6600

Konica Minolta Medical & Graphic (Shanghai) Co., Ltd.
11F-C1 Junyao International Plaza, No.789
Zhaojiabang Road, Shanghai, China Postcode 200032
TEL. 021-6422-2626

Production Site:
Konica Minolta Techno Products Co., Ltd.
2-2-1, Hirose-dai, Sayama-shi, Saitama 350-1328, Japan

A5ATYC220A

20110912CT