

# User Manual


Product Name: The Wearable Smart Phone Alert  
Model: iL-100

Brand: SERENE, Aurismed, Amplicomms

Manufacture: INTELLI INNOVATIONS(USA), LTD




## Feature Identifications:

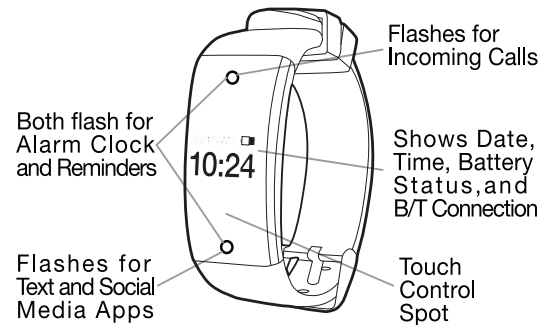

## Charging instaLINK:

Charge instaLINK for 2 hours before first use as shown below.

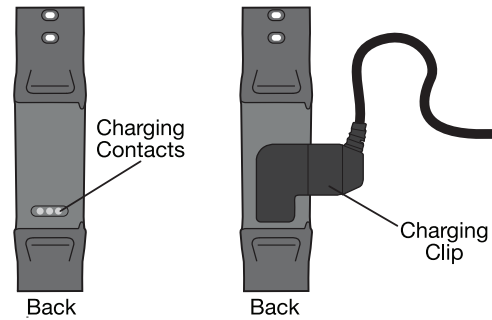

## Download App:

1. Go to the App Store or Google Play and search for the "LINK" App by Serene Inovations Inc.
2. Download and install the free App on your iOS or Android device.

## Pairing instaLINK to your Smartphone:

1. Turn the instaLINK "ON" by placing finger on touch control spot until the word "Welcome" appears. (fig. A)
2. Make sure your smartphone's Bluetooth™ function is turned "ON".

## Pairing instaLINK to your Smartphone (CONT.):

3. Tap and open the "LINK" App. The App will start searching for the nearby instaLINK to complete the pairing process. Press "PAIR" when it appears on screen. (Note: Pairing is done via the App only)
4. When pairing is complete, a small Bluetooth icon will appear on the instaLINK's display.



## Setting Up Alert Notifications:

Follow the App to:

1. Set Date and Time (12 or 24 Hour Clock)
2. Set Alarm Times (Up to 8 Alarms)
3. Set Medicine Reminder Times (Up to 8 Reminders)
4. Select Notifications for:
  - Incoming Calls
  - Text/SMS
  - Social Media Apps: Facebook, Twitter, Skype, Whatsapp, etc.

## Alert for Incoming Calls:

1. instaLINK will Vibrate up to 5 Times and then stop or touch the Touch Control Spot to stop the vibration. After vibration has stopped, nothing will be shown on the display.

## Reviewing Missed Calls:

1. When nothing is on display, tap the Touch Control spot once to show display, then tap touch control spot again.
2. A handset icon will appear on the display.
3. Place finger on Touch Control Spot for 2 seconds again and missed call will appear.
4. Tap the Touch Control Spot once to cycle through 5 most recent missed calls.

## Clearing Missed Calls:

1. While reviewing missed calls, place finger on Touch Control Spot for 2 seconds.
2. The message "DELETE ALL" will appear on the display.
3. Place finger on Touch Control Spot for 2 seconds again and all calls will be cleared.
4. After calls are cleared, there will be nothing on the display.

## Alert for Text or Social Media Apps:

1. instaLINK will Vibrate 3 Times and then stop or touch the Touch Control Spot to stop the vibration. After vibration has stopped, nothing will be shown on the display. Check missed texts and social media App notifications on phone.

## Alert for Alarm Clock/Medicine Reminders:

1. InstaLINK will Vibrate up to 5 Times and then stop or touch the Touch Control Spot to stop the vibration. After vibration has stopped, nothing will be shown on the display.

## FCC Caution.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.