SERENE®

**TV SoundBox®**  
Wireless TV Speaker

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Model  
**TV-SB**

Thank you for purchasing this TV SoundBox®. Please read this operating guide very carefully before use and keep it for future reference. If you have any questions, please feel free to contact our Customer Care Department.

**Toll-Free Help Line:** 866.376.9271 (9AM-5PM PST, M-F)

## **IMPORTANT SAFETY INSTRUCTIONS:**

Please read these instructions carefully before using the TV SoundBox®.

- Always keep the system dry. Do not immerse in water.
- For your safety, do not attempt to open the unit or battery compartment. Attempting to open or opening the battery compartment violates your warranty.
- TV SoundBox® uses a lithium ion battery. Please observe the safety and handling instructions for the lithium ion batteries. Please dispose of the unit/used batteries properly, following any local regulations. Do not dispose of the unit/used batteries in the household trash. Do not incinerate. Take the equipment to a nearby recycle center.


- Do not unplug by pulling on cord. To unplug, grasp the plug, not the cord. Unplug from outlet when not in use and before servicing or cleaning. Do not operate any device with a damaged cord or plug, or after the device malfunctions, or is dropped or damaged in any manner. Return device to the nearest authorized service facility for examination, repair or electrical or mechanical adjustment.
- TVSoundBox® is intended for home and personal use only.

### **FCC.Radiation.Exposure.Statement**


The antennas used for this transmitter must be installed to provide a separation distance of at least 20 cm from all persons and must be co-located for operating in conjunction with any other antenna or transmitter.

### PACKAGE CONTENT CHECK:

Your TV SoundBox® is packaged with all of the items shown. If anything is missing, please contact our Customer Care Department immediately.

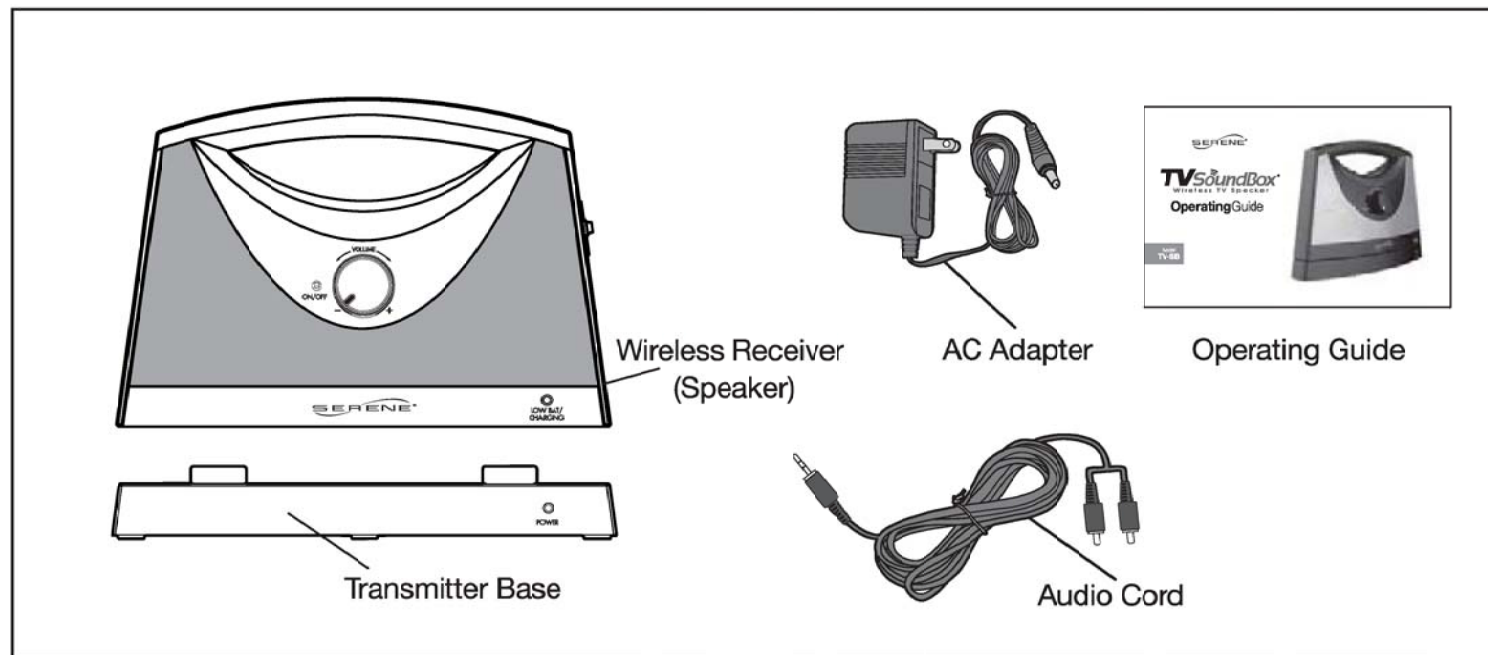
### FEATURE IDENTIFICATIONS:
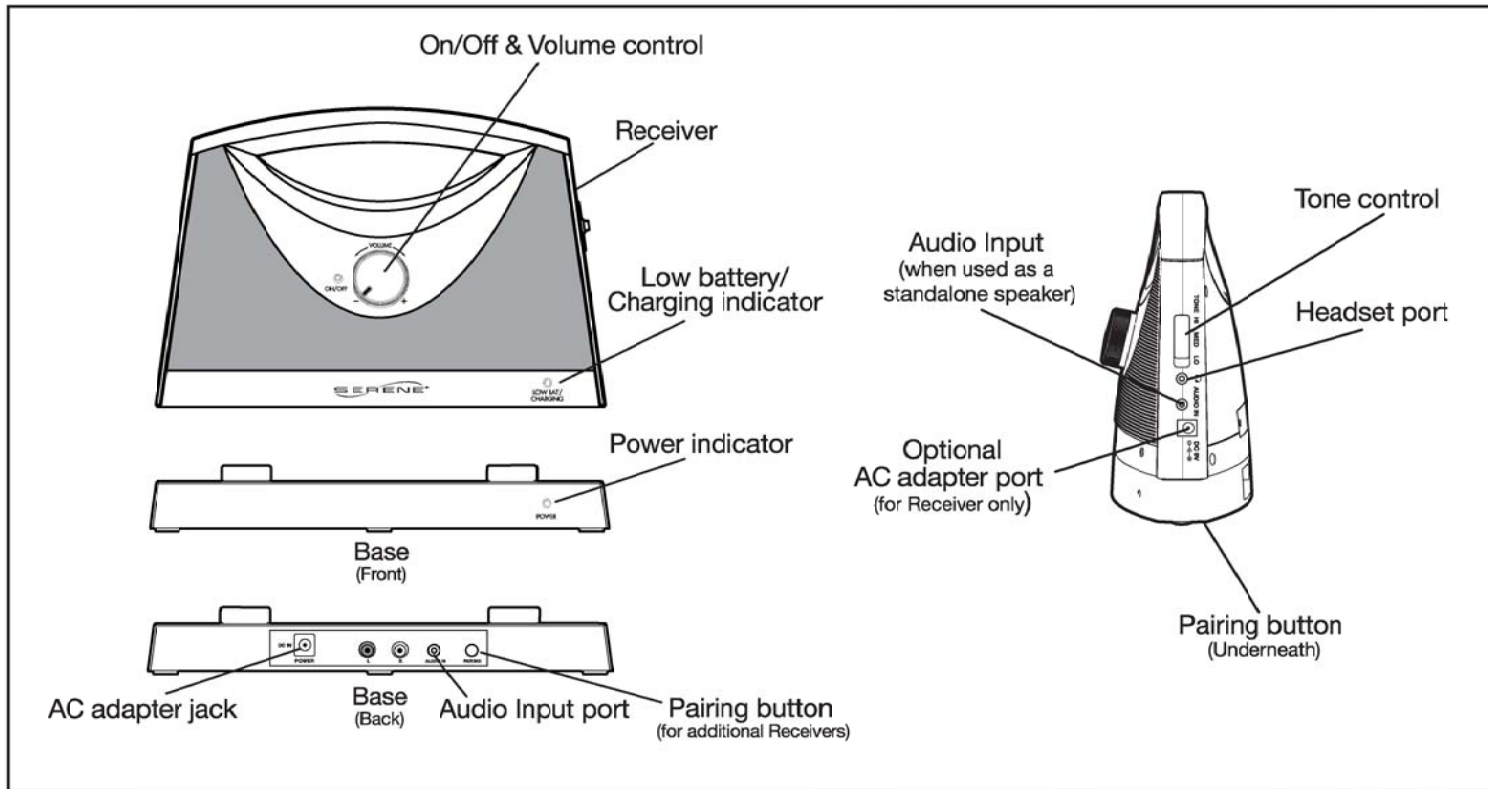
### CONNECTION & SET UP:


#### For TV with Headphone:

If your TV has a Headphone jack, connect the Base of the TVSoundBox® to your TV as shown below.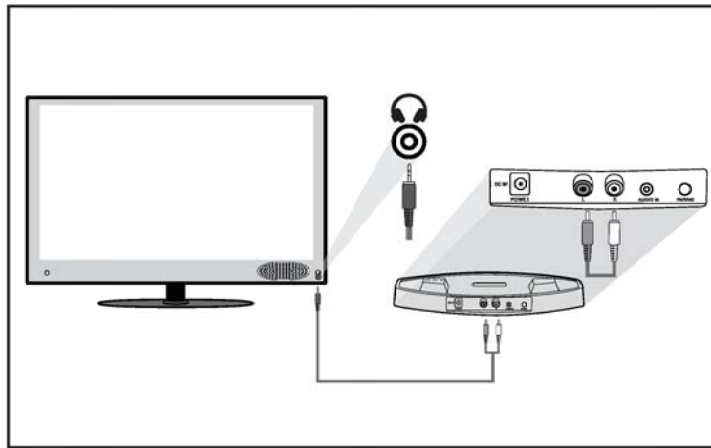


#### For TV's with Audio-Out Ports:

If your TV does not have a Headphone jack, but does have Audio-Out ports (look for these at the back of your TV), connect the Base of the TVSoundBox® to your TV as shown below.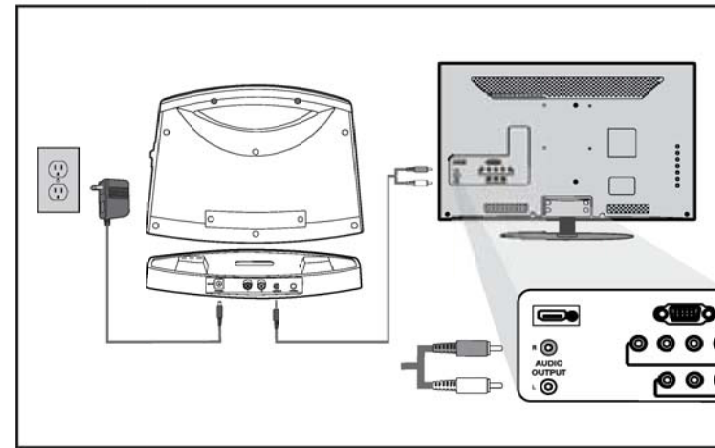


### CONNECTION & SET UP (con't):

**For TV without Headphone and Audio-Out ports:**  
If your TV has a digital Optical-Out port, use a TV Connector (sold separately, available from Serene, and connect it to the TVSoundBox® as shown below.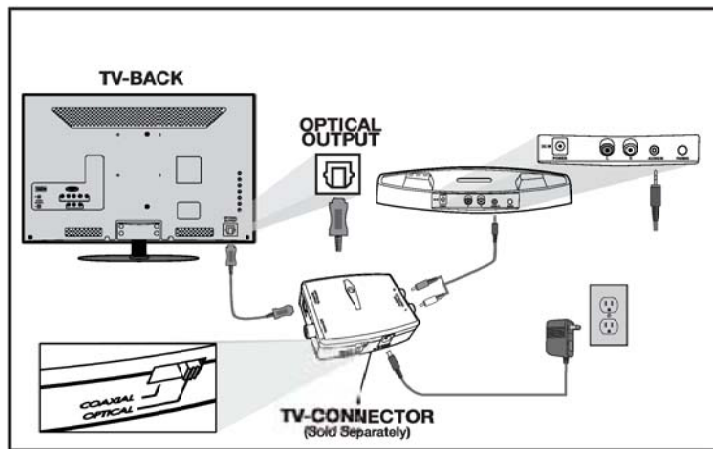

### Placement of Base:


Place base on top of your TV stand or on top of your TV.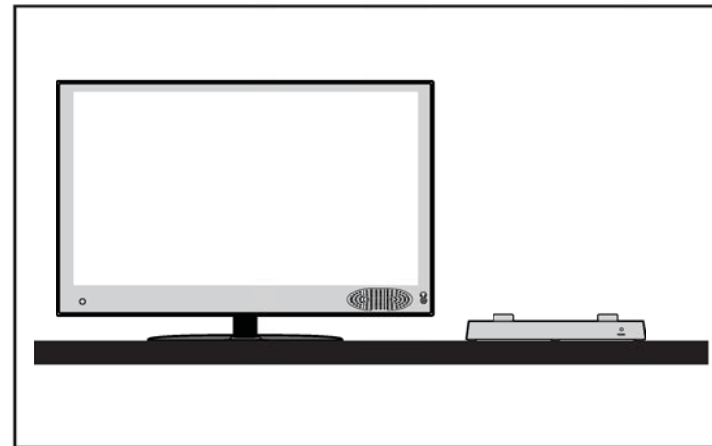

### BATTERY CHARGING:

Place the Receiver on the Base as shown for charging for 6+ hours before first use. During charging, the Low Battery/Charge indicator blinks. When it is fully charged, the indicator light will stop blinking and stay steadily lit. A fully charged battery will give you up to 8 hours of listening time.


When the Receiver is not in use or when the Low Battery indicator blinks, place the Receiver on the Base for recharging.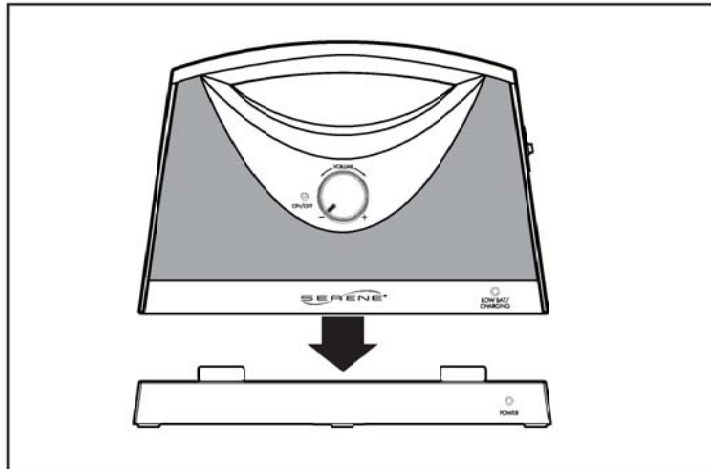

### OPERATION:

#### On/Off, Volume and Tone control:

1. Rotate the knob to turn Receiver On or Off and to adjust its volume.
2. You can turn your TV speaker to low or to off. On some TV's, you may need to use your Remote to go into the TV Menu to achieve an optimal audio/speaker setting. Please refer to your TV user manual for details.
3. To conserve battery power, the Receiver automatically turns off if no signal is received from the Base (after

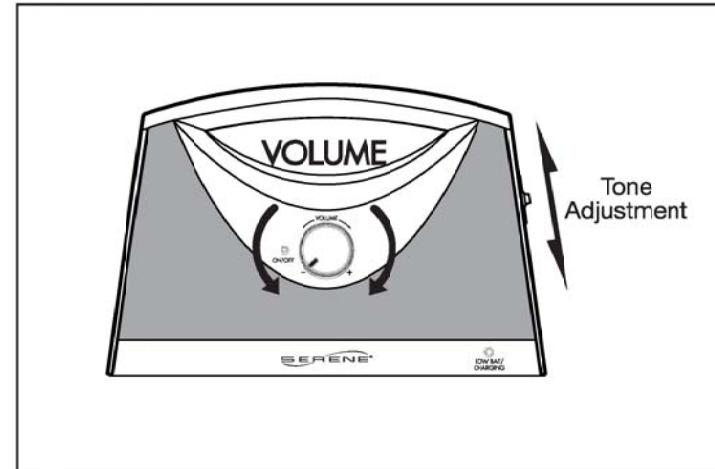### OPERATION (con't):


your TV is turned off) for a period longer than three minutes. To turn the Receiver on again after you have turned on your TV, rotate the knob to Off and then back to On. The Base will go into standby mode automatically when your TV is turned off.

4. Set the Tone Control to a position that enables you to hear and understand the dialog best.


### ADDITIONAL RECEIVERS:

You can add up to 50 Receivers to work with your Base. For each added new Receiver, you will need to “pair” it with the Base as follows:

1. Turn the new Receiver On.
2. Press and hold at least 2 seconds the PAIRING button at the back of the Base to initiate pairing. The Power Indicator will blink while it is looking for the new Receiver.
3. Turn on the Receiver and press and hold the PAIRING button on the underside of the Receiver. It will emit a beep sound. The Base’s indicator light will stop blinking and stay on when it is successfully paired with the Base.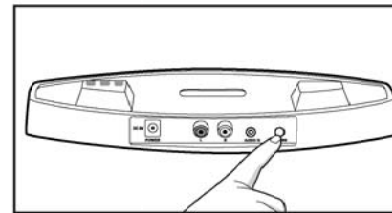

Back of transmitter (Base)

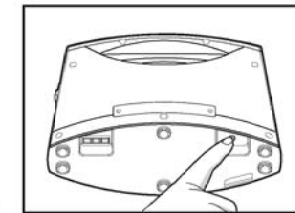
Bottom of receiver (Speaker)


### USING A HEADPHONE:

- Plug in your headphone to the Headphone jack. No sound will be emitted from the speakers. Adjust the headphone volume by rotating the Volume knob.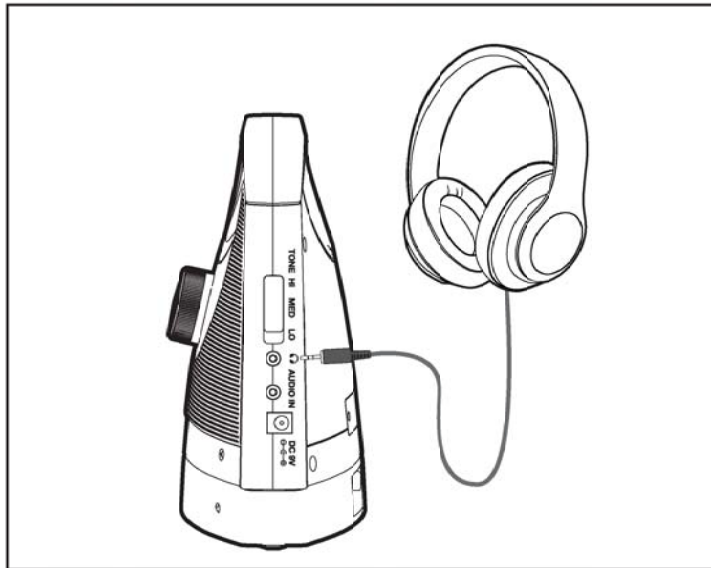


### USING THE RECIEVER AS A STAND-ALONE SPEAKER:

- Connect your audio source to the Audio Input jack as shown.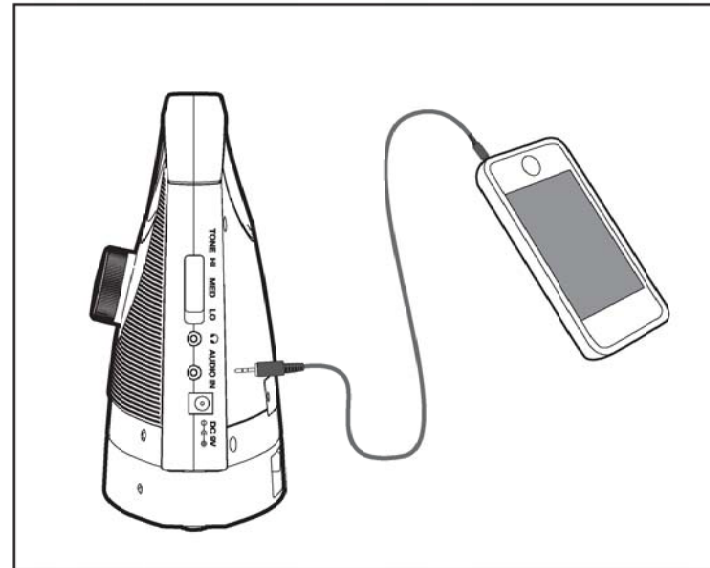

### ADDITIONAL POWER SOURCE FOR YOUR RECEIVER:

- Purchase an additional AC Adapter (available from Serene) and plug it into the Receiver as shown. This way, you can place and keep the Receiver at the same location without the need to return it to the Base for recharging.

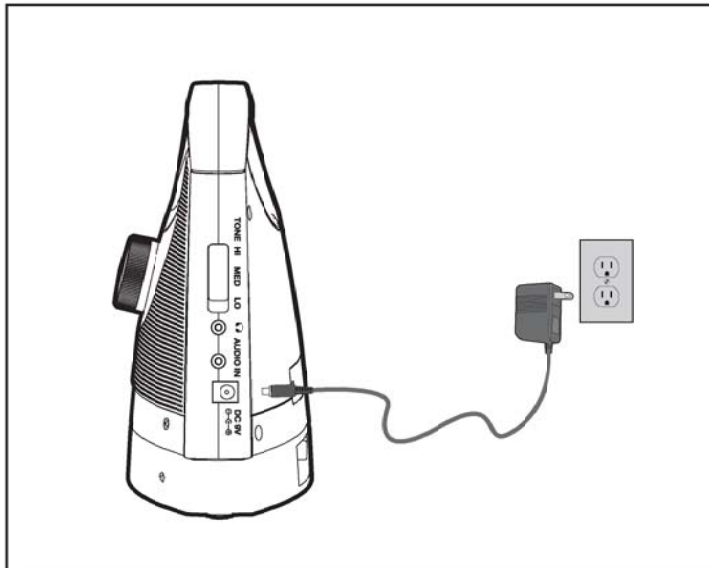### TROUBLESHOOTING:

- No sound from the Receiver- Check all connections to ensure that all the cords are firmly connected at the back of the Base and TV. Also, confirm that the Receiver is not more than 100 feet away from the Base.
- There is sound from the TV speaker, but no sound from the Receiver- Check your TV speaker setting by going into the TV Menu, using your TV remote and ensuring the TV is sending audio signals to the Base.
- The Low Battery/Charging indicator keeps blinking on the Receiver- This indicates that the battery is low and needs to be placed on the Base for recharging. When fully charged, the blinking will stop (in about 3 hours). If the blinking does not stop, check to make sure the Receiver is firmly connected and properly aligned to the Base.

### CLEANING AND MAINTENANCE:

Unplug the AC Adapter from the wall. Then, use a dry, soft cloth to clean and dust the outside surfaces and the charging contacts on the Base. Do not immerse the Receiver in water.

**OPTIONAL ACCESSORIES:**

The following accessories enhance your TV listening experience. Please contact Serene Innovations or your nearest dealer to order:

- Additional Receiver to use with your Base.  
Model TVSB-RX, Item #SB1002
- Additional AC adapter for your Receiver.  
Model TVSB-APTR, Item #SB1003
- TV Connector for TV with no Audio Out ports.  
Model TV-DACUS, Item #TV6007 (US Version)  
Model TV-DACEU, Item #TV6008 (European Version)

**WARRANTY/SERVICE INFORMATION:**

Your TV SoundBox® comes with a one year limited warranty. We warrant during this warranty period from the date of purchase to the original consumer that TV SoundBox® will be free from defects in materials and workmanship under intended home use. In the event that TV SoundBox® fails to function properly within one year of original purchase due to defects in materials or workmanship, return the unit (freight prepaid) with proof of purchase (sales receipt or packing slip, no exceptions) to Serene Innovations.

Serene Innovations will either repair or replace the unit (with a refurbished unit or unit of equal condition) and return it to you (using UPS/USPS ground shipping) at no cost to you, if the unit is returned within 30 days of purchase. If the unit is returned after 30 days of purchase but within the warranty period, there will be a warranty handling charge of U.S. \$9.95 for each return.

This warranty does not apply to any product that has been accidentally damaged due to: abuse, misuse, negligence, use on electrical frequency or voltage other than marked on product and/or described in this manual. Defects or errors caused by unauthorized alterations, repairs and/or tampering are also not covered by this warranty. This warranty gives you specific legal rights. You may have other legal rights that vary from state to state. If a defect covered by this warranty should occur, promptly contact a Customer Care Representative by phone or log on to [www.sereneinnovations.com](http://www.sereneinnovations.com) to obtain a Return Merchandise Authorization number and shipping instructions before returning the product to us. Any shipment without a RMA will not be accepted and will be returned to you at your expense.

Any authorized returned product must be accompanied with proof of purchase and a brief description of the problem. For out of warranty repair and service, please contact our Customer Care Department for directions.

**SPECIFICATIONS:**

(Final product may vary.)

- Transmission:.....2.4 GHz Radio Frequency
- Speakers:..... 2x5W Class D
- Speaker Battery:..... Built-in Lithium Ion Battery
- Operating Range:..... Over 30m (100 feet) in an open field
- AC Adapter Power:..... 100-240VAC/50-60 Hz input; 9VDC 1.5A output
- Listening Time:.....Up to 8 hours (at 50% max. volume level) when fully charged
- Dimensions:..... 24x17x8 cm (9x7x3.25")
- Weight:..... 800 gm. (1.7 lb.)

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This device complies with Industry Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

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#### **Part 15 of FCC Rules Information**

This device complies with Part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. This equipment has been tested and found to comply with the limits of a Class B digital device, pursuant to Part 15 of FCC rules. These limits are designed to provide reasonable protection against harmful interference in residential installation.

NOTICE: The Industry Canada label identifies certified equipment.

This Class B digital device complies with Canadian ICES-003.

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Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Serene Innovations, Inc.

14731 Carmenita Road, Norwalk, CA90650 Toll Free: 866.376.9271

Local 562.407.5400 [www.sereneinnovations.com](http://www.sereneinnovations.com)