iHealth™

Wireless Body Analysis Scale (HS5)

Balance d'analyse corporelle sans fil (HS5)

OWNER'S MANUAL

MODE D'EMPLOI

Table of Contents

INTRODUCTION	3
PACKAGE CONTENTS	3
INTENDED USE	3
ASSESSING BODY WEIGHT	4
CONTRAINICATION.....	7
PARTS AND DISPLAY INDICATORS	7
SET UP REQUIREMENTS	8
SET UP PROCEDURES.....	9
MEASUREMENT PROCEDURES.....	13
SPECIFICATIONS.....	16
GENERAL SAFETY AND PRECAUTIONS.....	18
TROUBLESHOOTING	20
CARE AND MAINTENANCE.....	24
WARRANTY INFORMATION.....	24
EXPLANATION OF SYMBOLS.....	25

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EXPLICATIONS DES SYMBOLES.....25

INTRODUCTION

Thank you for selecting the iHealth Wireless Body Analysis Scale. The Scale is designed for home use so you can measure, track and share vital body composition. This Owner's Manual will guide you through the setup procedure and highlight the Scale's key features. Please keep it handy for future reference.

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PACKAGE CONTENTS

- iHealth Wireless Body Analysis Scale
- Owner's Manual
- Quick Start Guide
- 4 AA Batteries

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INTENDED USE

The iHealth Wireless Body Analysis Scale is a precision electronic instrument intended for adult use. This Scale employs full electronic methodology and a pressure sensor to measure body composition automatically in a non-invasive manner. The measurements are displayed on an iPod touch, iPhone, or iPad and stored in the device's memory with a date and time stamp.

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IMPORTANT NOTE FOR USERS

Pregnant women need to consult their doctor before use. A person who is ill, feverish, has a chronic or acute disease, or a condition that affects hydration levels may get inaccurate results. Please consult your physician or healthcare provider for more information.

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Always store the iHealth Wireless Body Analysis Scale in a dry place. To ensure accurate results, keep the device away from magnetic fields as this may adversely affect communications or possibly damage the device.

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ASSESSING BODY WEIGHT

The Body Mass Index (BMI), or Quetelet Index, is a heuristic measure of body weight based on a person's weight and height. Body Mass Index is defined as the individual's body weight divided by the square of his or her height. The formulae universally used in medicine produce a unit of measure of kg/m².

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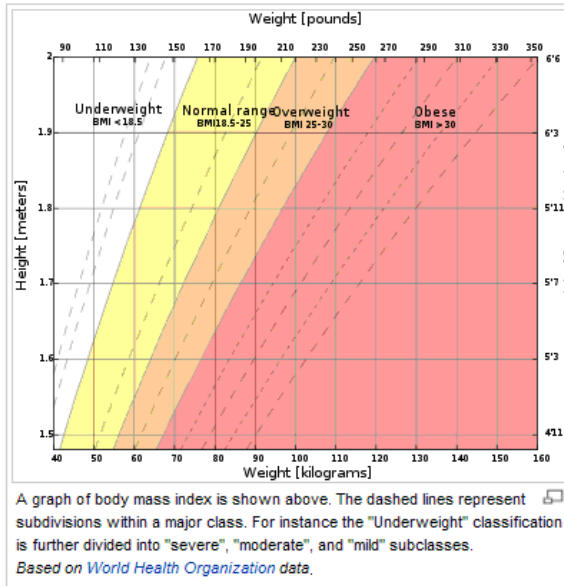
BMI can also be determined using a BMI chart, which displays BMI as a function of weight (horizontal axis) and height (vertical axis) using contour lines for different values of BMI or colors for different BMI categories.

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English	French
Weight (pounds)	Poids (livres)
Underweight IMC<18.5	Insuffisance pondérale IMC<18,5
Normal Range IMC18.5-25	Plage normale IMC18,5-25
Overweight IMC25-30	Surcharge pondérale IMC25-30
Obese IMC>30	Obèse IMC>30
Height { meters}	Taille { mètres}
Height {feet and inches}	Taille {pied et pouces}
Weight {kilograms}	Poids {kilogrammes}
A graph of body mass index is shown above. The dashed lines represent subdivisions within a major class. For instance the "Underweight" classification is further divided into "Severe", "moderate", and "mild" subclasses. Based on World Health Organization data.	Un graphique de l'indice de la masse corporelle est affiché ci-dessus. Les lignes pointillées représentent les sous-divisiones au sein d'une classification plus large. Par exemple la classification "Insuffisance pondérale" est davantage divisée en sous-classes "Grave", "modérée" et "légère". En fonction des données de l'Organisation Mondiale de la Santé.

CONTRAINDICTION

Never use this unit in combination with medical electronic devices such as:

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 (1) Medical electronic implants such as pacemakers.

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 (2) Electronic life support systems such as an artificial heart/lung.

 (3) Portable electronic medical devices such as an electrocardiograph.

This unit could cause these devices to malfunction, posing a considerable health risk to users of these devices.

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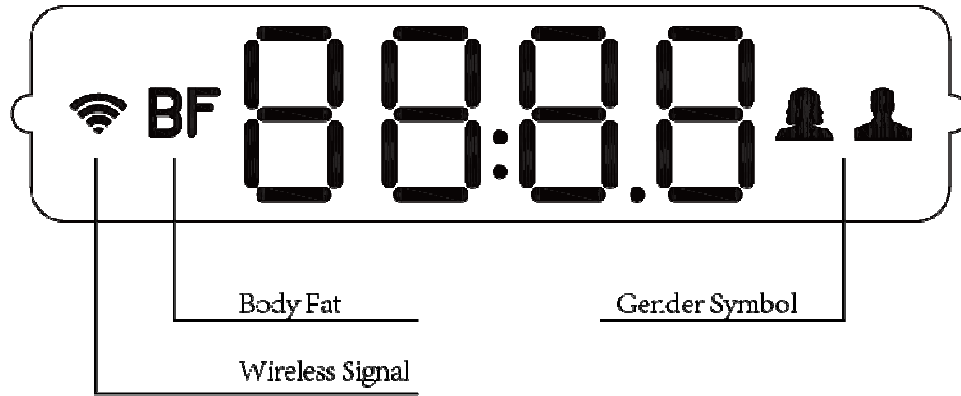
SETUP REQUIREMENTS

The iHealth Wireless Body Analysis Scale is designed to be used with the following iPod touch, iPhone and iPad models:

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iPod touch (4th generation)
 iPhone 4S
 iPhone 4
 iPhone 3GS
 iPad (3rd generation)
 iPad2
 iPad

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The iOS version of these device should be V5.0 or higher.

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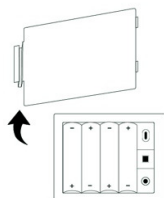
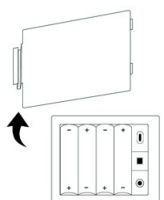
- b. Install four "AA" batteries.
- ⚠ If "Lo" appears on the LED, replace all the batteries.
 - ⚠ Rechargeable batteries are not suitable for this Scale.
 - ⚠ To avoid damage as a result of battery leakage, remove the batteries if the Scale is not going to be used for 3 months or more.
 - ⚠ Avoid getting battery fluid in your eyes. If it should get in your eyes, immediately rinse with plenty of clean water and contact a physician.

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—The Scale and the batteries must be disposed of according to local regulations at the end of their service life.

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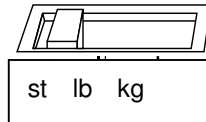
Select Weight Unit

Select a weight unit by adjusting the switch under the battery cover on the back of the Scale.

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Download the Free iHealth App

Prior to first use, download and install the iHealth App from the App Store. Use keyword search terms "iHealth", "HS5" or "Wireless Body Analysis Scale."

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Connect to a New Wireless Router

STEP 1: Make sure your iOS device is compatible and is version V5.0 or higher.

STEP 2: Turn **Bluetooth** "On" under the "Settings" Menu (Settings>General>**Bluetooth**>On).


STEP 3: Press the "SET" key under the battery cover on the back of the Scale.

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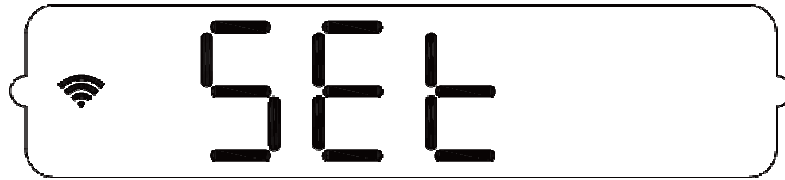
STEP 4: When using the Scale for the first time with your iOS device, wait until the model name, "HS540000", and "Not Paired" appear in the Devices list on your device's Bluetooth screen.

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Every subsequent time you use the Scale, "Not Connected" will be displayed and then continue from STEP 5.

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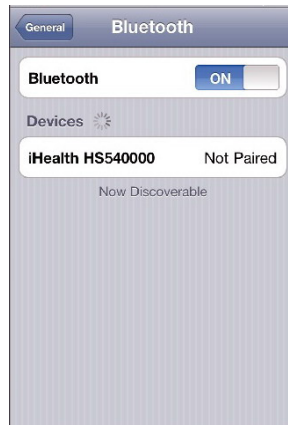STEP 5: Ensure that the Scale is powered on and select the model name "HS5 xxxx" to pair and connect.

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STEP 6: Select "Allow" in the pop-up window, as shown below.

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
STEP 7: The wireless signal should flash for a few seconds and then stop flashing when the Wi-Fi connection is successful.

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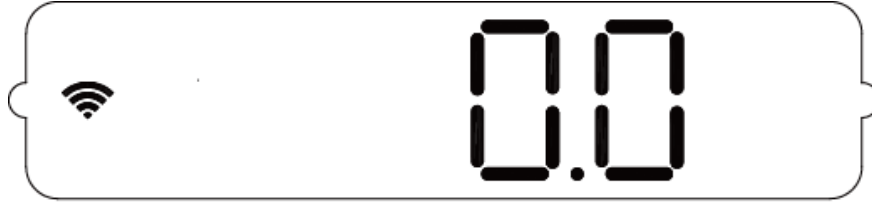STEP 8: Launch the App.

Note: When pairing the Scale for the first time, it may take up to 30 seconds for your iOS device to detect the **Bluetooth** signal during the pairing process.

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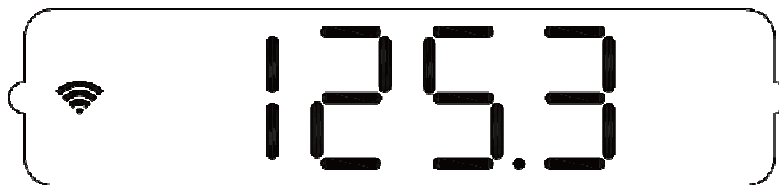
MEASUREMENT PROCEDURES


- Step on the Scale to turn it on, and wait until "0.0" appears on the display.
- Stand on all four electrodes with bare feet. Your weight will be shown on the Scale's display.

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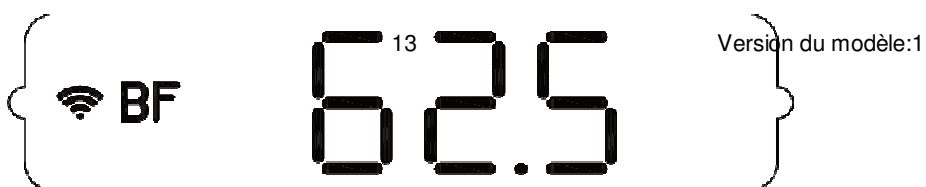
Note: If the Body Fat measurement fails, your weight will still display but the Body Fat data and symbol (BF) will not appear on the screen.

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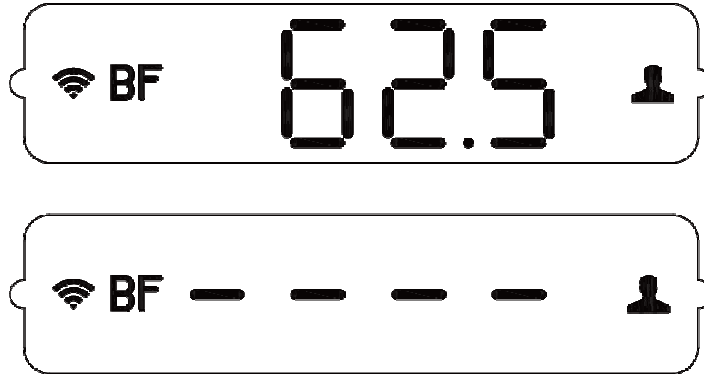If the Scale is connected to an iOS device and the App is running, the App will show detailed body composition data including Body Weight, Body Fat, Lean Mass, Body Water Percentage, Body Muscle Mass, Bone Mass, BMI, Visceral Fat Rating and Daily Calorie Intake (DCI).

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User information such as ID, height, gender, and age as well as time and date will be synchronized with the iOS device after the Scale's initial use.

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MEASURING WITHOUT AN iOS DEVICE

MESURER SANS UN APPAREIL iOS

The Scale can also be used without being connected to an iOS device. In this case, after your weight and other body composition parameters are measured, the Scale will automatically attach the measurements to the appropriate user.

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If two or more people with similar weights are using the Scale, you will need to select the correct user. The Scale will display the user symbol and flash the user ID number. Step on the lower left corner of the Scale lightly to select the user ID number and then step on the lower right corner of the Scale to confirm it. Body Weight and Body Fat will then be displayed on the screen alternatively.

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If the Scale is connected to the Internet through a router, current measurement data will automatically be sent to the iHealth Cloud.

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If not, measurement data will be stored in the Scale's memory and can be uploaded to your iOS device when the connection is re-established.

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The Wireless Body Analysis Scale can store up to 200 weight results for up to 20 users. When each user's memory is full, any new measurements will overwrite the oldest ones.

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- a. The Scale shuts down automatically after 2 minutes following completion of the measurement. If a new measurement is started or any App

operation is performed (such as uploading the memory contents of the Scale to an iOS device) during this time, the shutdown timer will be reset to two minutes.

b. If abnormal measurement conditions are present (such as a low battery, etc.), the Scale also will shut down after 10 seconds.

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OPERATING INSTRUCTIONS

For detailed operating instructions, please visit www.ihealthlabs.com or the FAQ section of the iHealth Wireless Body Analysis Scale Application on your mobile device.

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10. Operating temperature: $10^{\circ}\text{C} - 35^{\circ}\text{C}(50^{\circ}\text{F}\sim 95^{\circ}\text{F})$
11. Operating humidity: 20 - 85%RH
12. Environmental pressure for operation: 86~106kPa
13. Storage and transport temperature: $-20^{\circ}\text{C} - 60^{\circ}\text{C}(-4^{\circ}\text{F}\sim 140^{\circ}\text{F})$
14. Storage and transport humidity: 10 - 95%RH
15. Environmental pressure for storage: 50~106kPa
16. Battery life: approx.6 months with daily usage

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Note: These specifications are subject to change without notice.

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the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

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TROUBLESHOOTING

DEPANNAGE

PROBLEM	POSSIBLE CAUSE	SOLUTION
No module name in iOS device	Bluetooth connection failed	Check that the module is on, and restart the iPod touch, iPhone or iPad.
Connection unsuccessful	Bluetooth connection failed	Connect again, restart the iPod touch, iPhone or iPad.
The symbol for the	Bluetooth connection failed	Reestablish the Bluetooth connection and

Bluetooth signal in the App disappears		restart the App. If the problem persists, restart the iPod touch, iPhone or iPad.
The App displays "Timeout"	Bluetooth connect failed	Re-upload data. If the problem persists, reestablish the Bluetooth connection.
The App displays "Wi-Fi connection failed, please ensure the connection Wi-Fi of iOS device"	The iOS device is not connected to a router	Establish a Wi-Fi connection between the iOS device and a router, and then launch App again.
The App displays "Wi-Fi connection failed, please ensure sharing Wi-Fi setting, share Wi-Fi setting again"	User didn't allow sharing in the Wi-Fi settings of the App	Select "Allow sharing" in the Wi-Fi settings of the App.
Wireless signal flash momentarily then disappear	User has not allowed sharing in the Wi-Fi settings or there is no connection between iOS device and the router	Check the connection between the iOS device and the router, and allow sharing in the Wi-Fi settings.
Communication failed	If this occurs while setting up Wi-Fi, the product may be out of range of the router. A Wi-Fi connection may also have been established with the wrong router.	Check the distance between the Scale and the router and that the correct router has been connected to.

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PROBLEM	POSSIBLE CAUSE	SOLUTION
No response when you perform operations or load batteries	Incorrect operation, or strong electromagnetic interference	Remove the batteries, and then re-install the batteries after one minute.
Low Battery indication: "Lo" No power	Low battery	Check that batteries are installed correctly (correct polarity). If the problem persists, please replace all four batteries.
Scale displays "Er 0"	Failed to initialize	Remove the batteries, and then re-install the batteries after one minute.
Scale displays "Er 1"	The current weight is beyond the measurement range	The Scale cannot exceed 150kg.
Scale displays "Er 2"	The current weight is unsteady	Place feet in the center of the Scale, stand still, and allow the Scale to shut off completely after weighing. Do not move the Scale unnecessarily.
Scale displays "Er 3"	The Scale's Wi-Fi module failed to power on.	Remove the batteries, and then re-install the batteries after one minute.
Scale displays "Er 4"	The connection between the Scale and the access point has failed	Keep your wireless access point in good condition. Remove the batteries, and then re-install the batteries after one minute.
Scale displays "Er 5"	Connection failed between MCU and Cloud Server	Keep your wireless access point in good condition. Remove the batteries, and then re-install the batteries after one minute.
Scale displays "Er 6"	EEPROM access error	Remove the batteries, then re-install the batteries after one minute and re-test.

Scale displays "Er 7"	Device parameter check error	
Scale displays "Er 8"	FLASH access error	
Scale displays "- - -"	Failed to detect body impedance	Review the Measurement Instructions and re-test.

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WARRANTY INFORMATION

INFORMATIONS SUR LA GARANTIE

The iHealth Wireless Body Analysis Scale is guaranteed to be free from defects in materials and workmanship within one year from the date of purchase when used in accordance with the provided instructions. The warranty covers the cost of components and shipping/handling. The warranty extends only to the end user. We will, at our option, repair or replace without charge the iHealth Wireless Body Analysis Scale. Repair and/or replacement are the only options covered by the warranty.

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EXPLANATION OF SYMBOLS

EXPLICATIONS DES SYMBOLES

Symbol for "THE OWNER'S MANUAL MUST BE READ"

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Symbole "AVERTISSEMENT"

Symbole "PIECES APPLIQUEES DE TYPE BF"

Symbol for "ENVIRONMENT PROTECTION – Waste electrical products should not be disposed of with household waste. Please recycle where facilities exist. Check with your local Authority or retailer for recycling advice".

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Symbol for "MANUFACTURER"

CE 0197

Symbol for "COMPILES WITH MDD93/42/EEC REQUIREMENTS"

Symbol for "Year of Manufacture"

Symbole " FABRICANT "

CE 0197

Symbole " CONFORME AUX MDD93/42/EXIGENCES CEE "

Symbole " Date de Fabrication "

Symbol for "EUROPEAN REPRESENTATION"

Lot

Symbol for "Lot Number"

Symbol for "KEEP DRY"

ANDON HEALTH CO., LTD.
No. 3 Jinping Street, YaAn Road, Nankai District, Tianjin 300190, China

Symbole "REPRESENTANTION EUROPEENNE"

Lot

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Symbole " CONSERVER AU SEC "

ANDON HEALTH CO., LTD.
No. 3 Jinping Street, YaAn Road, Nankai District, Tianjin 300190, Chine

Manufactured for iHealth Lab Inc.
Mountain View, CA 94043, USA 1-855-816-7705
www.ihealthlabs.com

Fabriqué par iHealth Lab Inc.
Mountain View, CA 94043, USA 1-855-816-7705
www.ihealthlabs.com

iHealth is a trademark of iHealth Lab Inc.

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Bluetooth® associated logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by iHealth Lab Inc. is permitted under license.

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“Made for iPod”, “Made for iPhone”, and “Made for iPad” mean that an electronic accessory has been designed to connect specifically to iPod, iPhone, or iPad, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPod, iPhone, or iPad may affect wireless performance. iPad, iPhone, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries.

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Other trademarks and trade names are those of their respective owners.

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iHealth est une marque commerciale de iHealth Lab Inc.